



WARNING

- Do not use this product or any available option equipment without first completely reading and understanding this instruction manual.
- Do not use the product for any purpose not indicated in this manual.
- Only qualified personnel trained in the treatment and prevention of bedsores should operate this device.
- Dynarex declines all liability for any consequences resulting from incorrect use of this product and from unauthorized alteration of the product.
- The manufacturer reserves the right to change the information contained in this document without prior notice.
- Do not allow children to play with or operate the pump or mattress.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination or repair.
- Keep the unit and cord away from heated surfaces and liquids.
- Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair and the like.
- Never drop or insert any object into any opening or hose.
- Do not use outdoors.
- Connect this product to a properly grounded outlet only.

INTENDED USE

The DynaRest™ Airfloat 300 Air Mattress with Pump is intended to reduce the incidence of pressure ulcers while optimizing patient comfort.

PRODUCT FUNCTIONS – Pump

The functions of the pump are described below. Please refer to the figures of the pump.

(1) Power Switch

- Turn ON/OFF the power, the pump will start/stop operation.

(2) Pressure Adjust Knob adjustable by patient's weight

- Turn the Pressure Adjust Knob to set a comfortable pressure level by using the weight scale as a guide.

(3) Normal Pressure Indicator

- A visible indicator (green) tells the pressure has reached a preset level.

(4) Low Pressure Indicator

- A visible indicator (amber) warns the pressure is below a preset level.

(5) Alarm Reset

- The audible/visible alarm turns on when the pressure is low.
- To mute the audible alarm, press the reset button. The visible alarm indicator will flash.
- Re-press the reset button to reactivate the alarm.
- The air pressure issue must be corrected in order for the visible alarm to shut off. Once corrected the audible/visible alarm function will automatically reset.


(6) Static Control

- Press the Static button to set it in static mode, and the Static indicator will come on. The static mode will be started within approximately 6 minutes. Press the Static button again to switch back to alternating mode.


Note: In static mode, the mattress provides a firm surface that makes it easier for the patient to transfer or reposition. The static mode helps prevent the patient from bottoming out when in a sitting position.

(7) Air Pressure Valve

- The air pressure valve is located on the left side of the pump.
- Connect the air tubes securely into the air pressure valve of the pump.

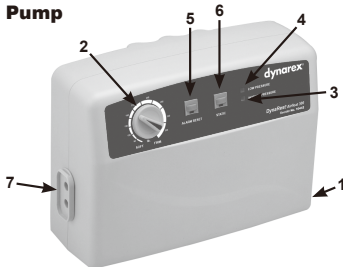

- (1) Power switch
- (2) Pressure adjust knob
- (3) Normal Pressure Indicator (green)
- (4) Low Pressure Indicator (amber)
- (5) Alarm Reset
- (6) Static Control
- (7) Air Pressure Valve

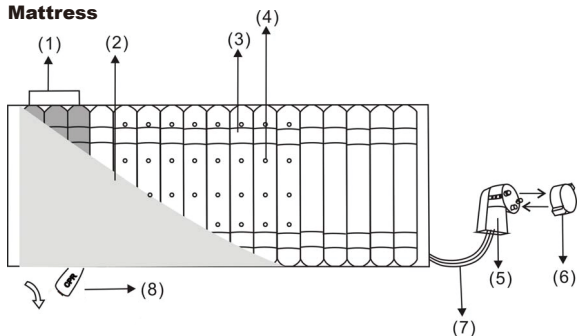

- (1) Pillow
- (2) Cover
- (3) Fixing belt
- (4) Low air loss holes
- (5) Male quick connector
- (6) Protective cap of connector
- (7) Air tubing
- (8) CPR valve

