


Reorder No. 12960

Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com
Made in China




Universal Rotating Assist Bar

User Manual

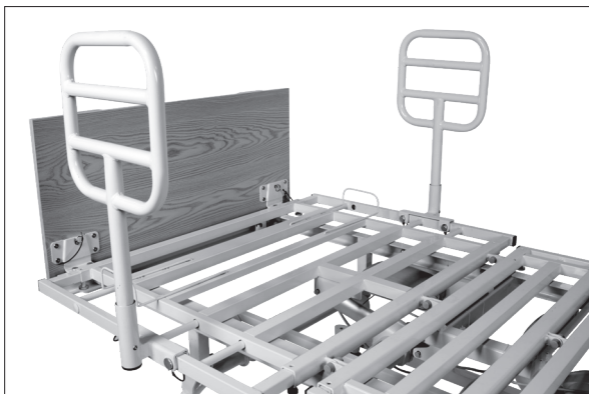


MOUNTING INSTRUCTIONS

1. Before installing the rails, the bed should be in the flat position. The carton should include 2 bed rails as shown:

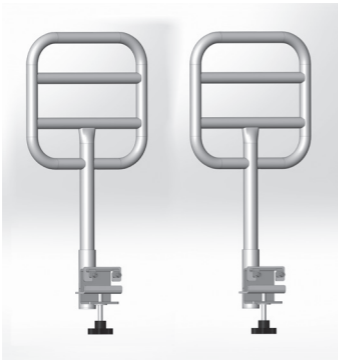

2. Clamp the rail bracket over the edge of the bed deck with the vertical sliding steel plate positioned on the inside of the bed frame. Tighten the two Allen screws and rotate the turn knob underneath the bracket clockwise until the rail is secured in place. Repeat the procedure to install the bed rail on the other side.

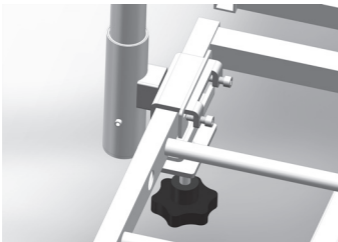

The rails can be positioned at the head or foot section.

RAIL OPERATION

PARALLEL RAIL ALIGNMENT: Lift up and rotate the bed rail until it is parallel to the bed edge and release the bed rail. Ensure it is securely locked in place.

PERPENDICULAR RAIL ALIGNMENT: Lift up and rotate the bed rail until it is perpendicular to the bed edge, then release the bed rail. Ensure it is securely locked in place.

WARNINGS

- Read the instructions carefully before installing.
- Before operating, ensure that the bed rails are assembled and installed correctly as instructed. Ensure that all the attaching hardware are tightened and that the bed rails are securely in place. Inspect the hardware frequently to make sure all the components are tightened securely.
- **DO NOT** use the bed rails as a handle for getting into or out of the bed. This could result in personal injury and damage to the bed rails.
- **DO NOT** apply side pressure to the bed rails. This could deform or even break the bed rails.
- **DO NOT** use the bed rails as a push handle for moving the bed. This could deform or even break the bed rails.
- **DO NOT** use the bed rails if proper installation cannot be achieved. Contact your dealer or manufacturer.

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.



SYMBOL GLOSSARY

For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

R220324