

[www.drivemedical.com](http://www.drivemedical.com)

**drive**  
MEDICAL DESIGN & MANUFACTURING

med aire plus 8" defined perimeter  
low air loss and alternating pressure  
mattress replacement system with  
low pressure alarm



EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

[www.drivemedical.com](http://www.drivemedical.com)

**drive**  
MEDICAL DESIGN & MANUFACTURING

## operating instructions

1. Place the pump on a flat surface or suspend pump on end of bed with attached hook.
2. Place the mattress on top of bed mattress. Note foot end.
3. Connect air hoses to pump and mattress.
4. Check and ensure the air hoses are not kinked or tucked under mattress.
5. Turn pressure to firm. Plug the power cord into outlet. Switch on the pump. The green switch will light when unit is operating.
6. Adjust pressure for patient comfort

### **DANGER:**

DO NOT use in the presence of flammable anesthetic:

It may cause possible explosive hazard.

### **CAUTION:**

Risk electric shock. DO NOT remove cover (or back). Refer to your qualified service personal for assistance.

## 1 year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China