MOBILITY DESIGNED

EXCURSION™ X20

Rollator

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Andador

Instructions For Use
Instructions D'utilisation
Instrucciones De Uso

Item # / Article n° / Artículo n.º:
MD700920BK, MD700920BL, MD700920PR

VER.A.9.25

Parts List

- 1 Backrest
- 2 Removable Storage Bag
- 3 Hand Grip
- 4 Brake Handle
- 5 Seat
- 6 Seat Handle
- 7 Closure Latch
- 8 Wheel


Excursion X20 includes:

- Removable Storage Bag
- Backrest
- Operating Instructions


Unfolding the Excursion X20

To unfold, position the rollator on all four wheels and ensure all four wheels rest firmly on the ground. Unlatch to locking hook. Hold each hand grip and pull outward until the rollator unlocks and opens fully. Gently push down on each side of the seat rails near the vertical frame to ensure the unit is fully opened. An audible 'click' confirms that the latches are secure, and the seat is fully engaged in a locked position.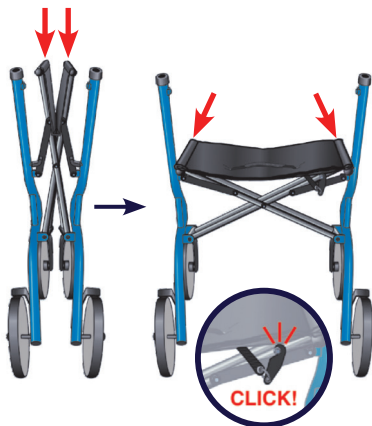


Assembly Instructions


- 1** Remove contents from carton.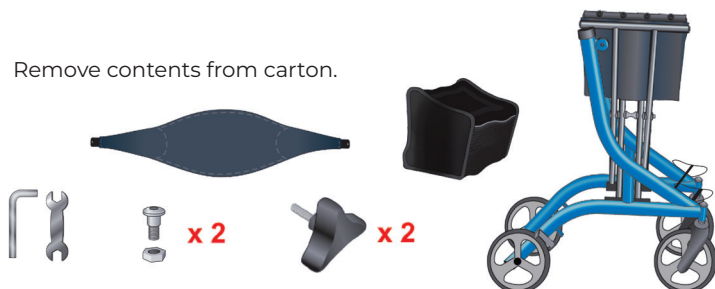

- 2** Unfold the frame. Push down on the seat supports to expand and open the frame. The safety lock will click into place when the rollator is fully unfolded.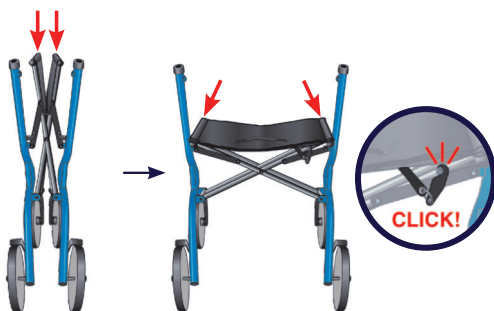



- 3** Insert the handlebars. Ensure the brake cables are on the outside of the frame and that they pass through the cable guide.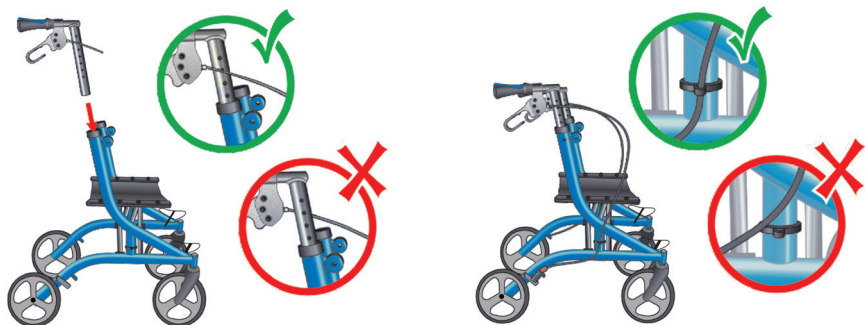


Assembly Instructions

- 4** For height adjustment, stand upright and set the hand grips even with your wrists. Hand tighten the triangular hand screws.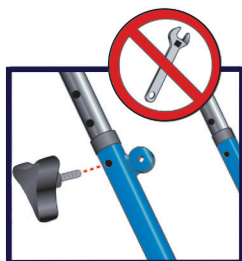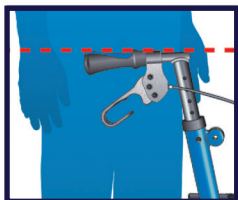

- 5** Install the backrest and the storage bag. Insert the backrest so that the logo faces the front wheels. Slide one end of the backrest into the fan bracket. Insert the screw from the inside of the unit out and repeat. Install the storage bag by sliding it onto the mounts starting from the back and then pulling to the front.

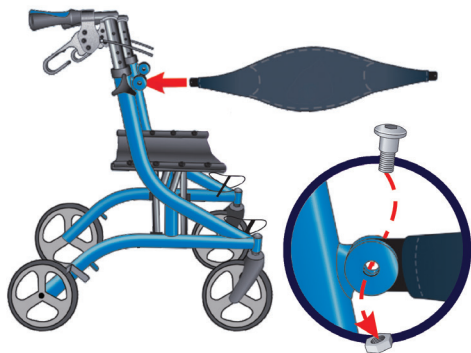Install the backrest

Install the storage bag

Directions

Folding: Pull the folding strap on the seat to release the safety lock and initiate folding. Use the folding clip to lock the rollator in the folded positions.



Using the brake handles: Squeeze up to slow down. To lock, push down until they click. To release, pull back up. ALWAYS lock the brakes before using the seat.

Brake Adjustment: Push down the adjustment knob and rotate. Clockwise will loosen the brakes, counter-clockwise will tighten them. Your brakes must be checked periodically for safety.

IMPORTANT: Before using your rollator again, ensure the brakes are working properly. DO NOT use if the wheels rotate while the brakes are locked.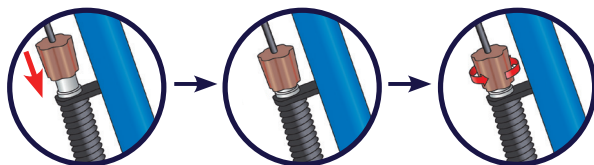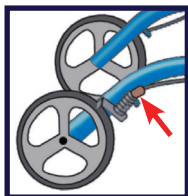

Safety Instructions

WEIGHT CAPACITY: 300 LB / 136 KG

- This is a walking aid only and is not to be used as a transportation device.
- Use only on even and solid surfaces.
- Maximum user weight: 300 lb.
- Regularly check the screws and fasteners on rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lb.
- Do not expose the product to temperatures above 104°F for an extended period as this can cause parts to overheat and burn the user.
- Please make sure the rollator is fully opened, seat is down and the folding bar under the seat is locked in the down position before using or sitting in the rollator.
- Be sure both hand grips are at equal heights. Test rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the rollator. When wheels are locked, the brake pad should prevent the wheels from turning. When the brakes are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.

DO NOT MOVE ROLLATOR WHILE SEATED.

Cautions:

- Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
- Before each use, make sure that all parts are secure and that the frame is fully opened.
- Risk of fall and serious injury may result if brakes are not locked while the user attempts to sit down.
- Do not navigate independently while seated. Do not attempt to sit on the seat while on an incline. Do not lean when you are seated. Doing so may cause the walker to tip and a serious risk of injury may result.
- Falls are possible if you use the seat without installing the backrest.
- Do not use your rollator while folded. The stability of any rollator is compromised when the frame is folded, increasing the likelihood of a fall.
- Do not exceed 300 lbs weight limit or serious injury could result.

Maintenance and Cleaning

Brush off dust and dirt or wipe with a damp cloth. Clean regularly by hand with a damp cloth and mild detergent only. Do not use aggressive detergents such as chlorine, bleach or abrasive cleaners. M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.