



LYSOL® IC™

Disinfectant Spray

This tuberculocidal, virucidal, fungicidal and bactericidal formula kills 99.9% of germs on hard, nonporous surfaces and is effective against over 50 microorganisms including H1N1 (Influenza A virus) MRSA, Norovirus, Rhinovirus, Poliovirus Type 1, Hepatitis A virus, Hepatitis B virus and HIV-1 (AIDS Virus).* This product eliminates odors and prevents the growth of mold and mildew. Lysol® disinfectant spray is a hospital-grade spray and kills 99.9% of viruses, including norovirus, which causes the stomach flu.* It also eliminates 99.9% of bacteria and fungi, helping to control the growth of mold and mildew, remove odors, and prevent seasonal colds, flu, and allergies.*

Features & Benefits

- Control Flo Valve produces a wetter spray, enhancing surface coverage
- Meets CDC, EPA & OSHA Bloodborne Pathogen Standards
- Use on showers, sinks, door handles & around toilet areas, Suitable for waiting rooms, sick rooms & laboratories
- Hospital use disinfectant: Bactericidal, Mildewstat



* When used as directed.





LYSOL® IC™ Disinfectant Spray

Use Instructions

- Preclean surfaces prior to use. Hold can upright 6" to 8" from surface. Spray 3 to 4 seconds until covered with mist.

To Disinfect:

- Surfaces must remain wet for 3 minutes then allow to air dry. For Norovirus and Mycobacterium bovis BCG surfaces must remain wet for 10 minutes then allow to air dry.

To Sanitize:

- Surfaces must remain wet for 10 seconds then allow to air dry

Fabric Sanitizer: To Sanitize Soft Surfaces:

- Spray until fabric is wet
- DO NOT SATURATE
- Fabric must remain wet for 30 seconds. Let air dry.
- For difficult odors, repeat application
- Rinse toys and food contact surfaces with potable water after use

To Control and Prevent Mold & Mildew:

- Apply to pre-cleaned surface
- Allow to remain wet for 3 minutes
- Let air dry
- Repeat applications in weekly intervals or when mold and mildew growth appears

To Deodorize:

- Spray on surfaces as needed
- Do not use on polished wood, painted surfaces, leather, rayon fabrics, or acrylic plastics

Technical data	LYSOL® IC™ Disinfectant Spray
Color/Form	Clear, Aerosol
pH	10.8 to 11.8
Shelf Life	2 Years

Product	Pack size	Dilution	Product code
LYSOL® IC™ Disinfectant Spray w/ Control Flo Valve	12 x 19 oz. Aerosol	Ready-to-use	CB950290

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.