



Transport Bar

User Manual



INTRODUCTION

The Transport Bar serves as a link for the different bed components. The assembly can be wheeled using the casters on the head and footboard frames.

Push the head/footboard to maneuver the assembly.

During the transportation, ensure all cables are tidy and not on the floor.

Once the bed is transported to the intended location, reverse the mounting procedure to unload the parts. Refer to the bed user manual for bed installation.

WARNINGS

Read instructions carefully before installing.

Before use, ensure that the transport bars and all attaching hardware are assembled as described. Inspect prior to transportation to ensure all attaching hardware is secured in position and will not fall off during use.

DO NOT push the bed platforms when moving the assembly as this may cause the bed platforms to fall off.

DO NOT use the Transport Bar if the bed platform end rivets are not properly mounted in the notches. (see figure 3.1 on pg. 3)

MOUNTING INSTRUCTIONS

1. Ensure the bed is disconnected from power. Follow the bed user manual and disassemble the bed into four sections – headboard, footboard, and two bed platform halves. Depending on the bed model, the motor shafts may also require disassembling.
2. Hook one end of the Transport Bar onto the headboard frame and the other end onto the footboard frame (see figure 2.1). Insert the wing screws to lock the transport bar in place (see figure 2.2). Repeat the installation on the other side (see figure 2.3).
3. Turn one of the bed platform halves vertically. With the spring side facing outwards and end rivets at the bottom, mount the bed platform onto the Transport Bar (see figure 3.1). The outer rivet should sit in the notch on the Transport Bar. Repeat the procedure and mount the other bed platform half (see figure 3.2).

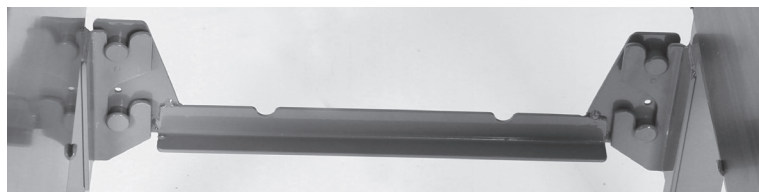


Figure 2.1

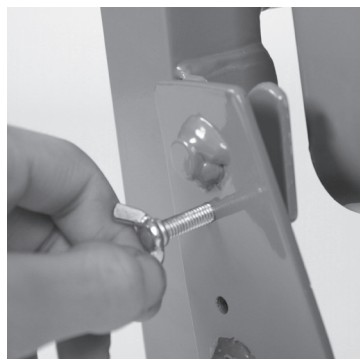


Figure 2.2



Figure 2.3



Figure 3.1



Figure 3.2

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced

- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

 **Manufactured for:**
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com
Made in China

 **WARNING:**
Cancer and Reproductive Harm
www.P65Warnings.ca.gov



SYMBOL GLOSSARY
For an explanation of symbols
used in Dynarex packaging, visit
dynarex.com/symbols.php