



# Accel PREvention™ RTU

## One-Step Surface Cleaner and Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A ready-to-use cleaner disinfectant based on proprietary AHP® technology to deliver fast, effective cleaning performance.

### Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus, HIV and Poliovirus providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 3 minutes. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces, while harnessing the power of AHP® technology. The AHP® active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPEs. It is non-irritating to eyes and skin
- Cleans and disinfects in one labor-saving step

### Applications

- Clean and disinfect high-touch surfaces in 3 minutes
- Non-irritating to eyes and skin





# Accel PREvention™ RTU

One-Step Surface Cleaner and Disinfectant

## Use instructions

### As a One-Step Surface Cleaner Disinfectant\*:

- Pre-clean heavily soiled areas.
- Apply product to hard, non-porous inanimate surfaces.
- All surfaces must remain wet for 3 minutes.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

\* Refer to product label for complete use instructions

Technical data	Accel PREvention™ RTU
DIN	02448521
Certifications	EcoLogo
Color/Form	Colorless, liquid
pH	2.1
Scent	Characteristic of ingredients
Shelf Life	2 years

Product	Pack size	Dilution	Product code	
Accel PREvention™ RTU	12 x 946 mL / 1 U.S. Qt. Bottles	Ready-to-use	101103731	🇨🇦
Accel PREvention™ RTU	4 x 3.78 L / 1 U.S. Gal. Containers	Ready-to-use	101103732	🇨🇦

## Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.