



Oxivir® Plus

30

Disinfectant Cleaner Concentrate

Dilutable one-step disinfectant cleaner, hydrotherapy tub cleaner and sanitizer, formulated with proprietary hydrogen peroxide technology. Ecologo certified.

Features & Benefits

- Quick disinfection in just 5 minutes against Human Immunodeficiency Virus (HIV-1) (AIDS Virus), Polio virus, Influenza A Virus, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Salmonella enterica*, MRSA and VRE
- 5 minute Bactericidal, Tuberculocidal, Virucidal, Fungicidal with a 30 sec 1:128 non food contact sanitizer claim
- AHP® active breaks down into water and oxygen after use
- No VOCs, no added fragrance and no NPEs
- Delivers fast and effective cleaning performance while being non-irritating to eyes and skin in use dilution

Applications

- Clean and disinfect non-critical medical devices, equipment and hard, non-porous surfaces





Use instructions

Oxivir® Plus Disinfectant Cleaner Concentrate is a one-step disinfectant cleaner using patented AHP® technology for use on walls, stainless steel, glazed porcelain, plastic surfaces, hydrotherapy tubs and other hard, nonporous surfaces.

- To disinfect, apply a 1:40 dilution to thoroughly wet surface.
- Let solution remain on surface for a minimum of 5 minutes.
- Air dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.


USE DIRECTIONS FOR HYDROTHERAPY TUBS AND FOOT BATHS

- To reduce the generation of foam, add an anti-foam agent at the rate of 5% of the amount of this product. Heavily soiled tubs require pre-cleaning prior to disinfection.
- **Initial Purging and Disinfecting** Fill tub with cool water to above the water jet line. Add this product at 1:40.
- Turn on the pump and allow the water to circulate for 5 minutes. Drain then rinse. Broad-Spectrum Sanitizing - Maintenance Fill tub with cool water to above the water jet line. Add this product at 1:128.

- Turn on the pump and allow the water to circulate for 30 seconds. Ensure surfaces have been scrubbed.
- Drain then rinse.

Dilutable Disinfectant Cleaner

- 1:40 dilution – Bactericidal and virucidal, including antibiotic-resistant bacteria, in 5 minutes.
- 1:128 dilution – Non-food contact sanitizer in 30 seconds or general purpose cleaning.

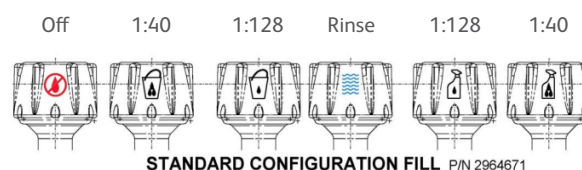

Technical data	Oxivir® Plus
DIN	02403684
Certifications	Ecologo, LONO
Color/Form	Colorless, liquid
pH	0.8 (Concentrate) 1.9 (Use Dilution 1:40)
Scent	Characteristic of ingredients
Shelf Life	2.5 years (Concentrate), 30 days (Use Dilution)

Product	Pack size	Dilution	Product code
Oxivir® Plus	4 x 3.78 L / 1 U.S. Gal. Containers	1:40, 1:128	5919024
Oxivir® Plus	2 x 1.5 L RTD® Units / 50.7 oz.	1:40, 1:128	101109764
Oxivir® Plus	2 x 2.5 L J-Fill® / 84.5 oz.	1:40	101109765

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design and Oxivir® are trademarks of Diversey, Inc.