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49551 & 49659 User Guide

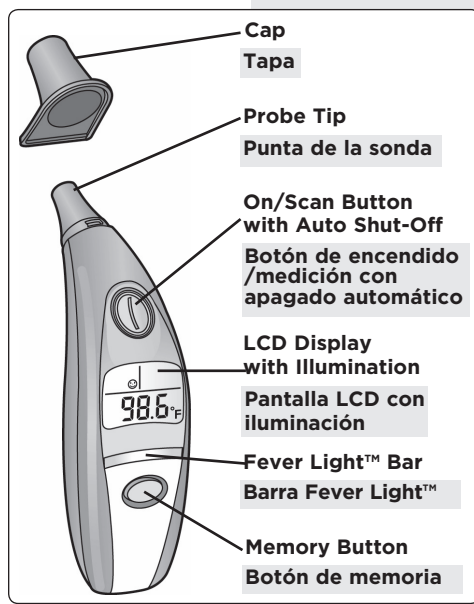
! WARNING:

- NEVER allow a child to play with this thermometer. It is not a toy. It is a sensitive electronic device and may be damaged if not used properly under parental supervision.
- ALWAYS supervise your child when using thermometer.
- Avoid sudden shock and dust.

Know Your Unit

How Does it Work?
 The *Fever Light™ Ear Thermometer* obtains a reading in 1 second by measuring the infrared heat given off by the eardrum and surrounding tissues.

Temperature From The Ear
 Clinical research has shown that the ear is an ideal site for taking body temperature. The eardrum shares the same blood supply with the hypothalamus, the part of the brain that controls body temperature. Therefore, the ear is an accurate indicator of internal body (core) temperature.

Important Information

- Read these instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading, and technique.
- High or prolonged fever requires medical attention, especially for young children. Contact your physician.
- For an infant, it is best to have the child laying flat with his head sideways so the ear is facing upwards. For an older child or adult, it is best to stand behind and slightly to the side of the child. It is recommended that three temperatures in the same ear be taken and the highest one used as the reading.
- Keep in mind that the thermometer will automatically shut-off after one minute.
- If you question any of the readings you get, check to see that the ear canal is free of excessive amounts of earwax and that the probe tip is clean (see “Care and Cleaning”).
- To ensure an accurate measurement, clean probe tip after each measurement. Store the thermometer in a dry place at room temperature.
- Proper placement within the ear canal is essential for an accurate reading. If the thermometer is not inserted properly and firmly into the ear canal, a lower reading might be obtained. Remember to always use the same ear. The temperature between the right and the left ear may differ.
- Do not return this product to the place of purchase. If any parts are missing, email consumer@djgusa.com, call Consumer Relations at (800) 544-1108, or fax at (800) 207-8182. You can also visit our website at www.safety1st.com. Have the model number (49551 or 49659) and date code (manufacture date located on back of thermometer) ready.
- DO NOT expose thermometer to extreme temperatures, humidity, direct sunlight or shock.

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Operating Thermometer

- 1 Turn On:** Press and release the on/scan button. All segments are displayed for 1 second (**Figure 1**). A short beep will occur and the last temperature measured appears (**Figure 2**). Backlight goes off after 3 seconds.
- When the backlight goes off the thermometer is ready to measure temperature.
- Gently pull ear up and back to straighten the ear canal. Insert the thermometer until the probe tip seals the ear canal.
 - **For children under 1 year:** Pull the ear straight back.
 - **Children ages 1 year to adult:** Pull the ear up and back.
- Press and hold the on/scan button for 1 second once you have the thermometer in place. During measuring the  icon will appear (**Figure 3**). After 1 second, release button. One beep will sound and the temperature with  icon will appear for 1 minute, and backlight illuminates for 5 seconds (**Figure 4**).

After the “clean me”  icon flashes several times, the unit is ready to take another measurement. Auto-shut off will occur after 1 minute. **NOTE:** Thermometer should be cleaned between uses.

FEVER ALERT NOTE: If patient has an elevated temperature 99.5 (37.5°C) or above, the red Fever Light™ bar will light up, the  icon will appear and multiple beeps will sound.

Changing from Fahrenheit to Celsius

To switch the display between °F and °C and vice-versa: While the thermometer is off, press and hold the on/scan button for about 5 seconds until you see the °F or °C. Release button. Unit has been reset.

Memory Function

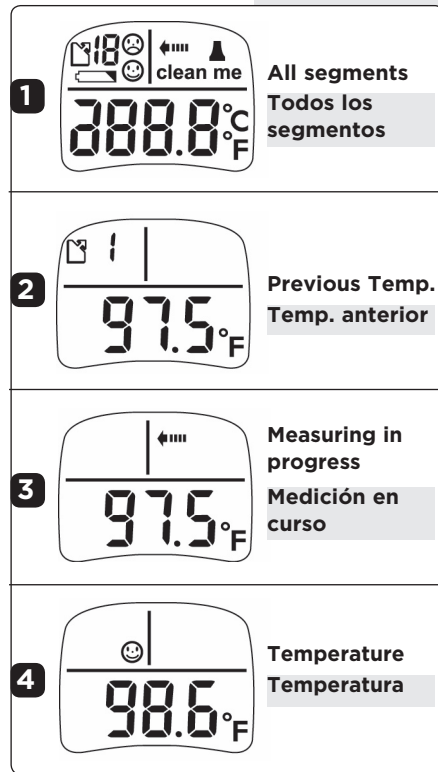
The thermometer memory function stores the last 12 measurements. The last measurement will appear for approximately 3 seconds after turning the thermometer on and after it runs its built-in test. The ready display will then show after last reading appears. This feature allows you to compare your most recent measurement to the prior ones.

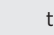
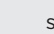
To scroll through the last 12 measurements: With the thermometer on, press and release the memory button. The screen will display the  icon and the number of the reading along with that measurement (**see Figure 2 above**). Press the memory button, screen will scroll to reading number “0”. Press the memory button again, and it will scroll up to number “1”, and so on. When 12 spaces are filled the oldest measurement will be the next to be deleted.

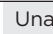
Fever Light™

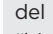
The Fever Light™ occurs during temperature measurement when the temperature is over 99.5°F (37.6°C). If your child has an elevated temperature the red Fever Light™ bar will light up, reading will blink and multiple beeps will sound. High or prolonged fever requires medical attention, especially for young children. Contact your physician. Fever chart included on package.

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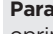
Care and Maintenance

- Clean the probe and tip with an alcohol swab before and after each measurement.
- Use a soft, dry cloth to clean thermometer body. Never use abrasive cleaning agents, thinners or benzene for cleaning. Do not scratch the surface of the probe lens or the display. Do not expose the thermometer to extreme temperatures, humidity, direct sunlight, or shock. Store at room temperature.

To Replace Battery

When the “low battery” icon appears (**Figure 1**), the battery is nearly exhausted, but a few measurements may still be taken. When the “Lo” appears (**Figure 2**), the battery is exhausted and needs replacing.

- Use a small screwdriver to loosen screw. Slide cover off (**Figure 3**).
- Press release tab with a small tool and slide battery out (**Figure 3**).
- Insert a new 3-volt battery (battery number CR2032-3V), correctly positioned with the positive “+” side facing up (**Figure 4**).
- Slide the battery cover on and secure the screw.

CAUTION: Replace only with 3V “CR2032” lithium battery. Use of other types of batteries may present a risk of fire or rupture.

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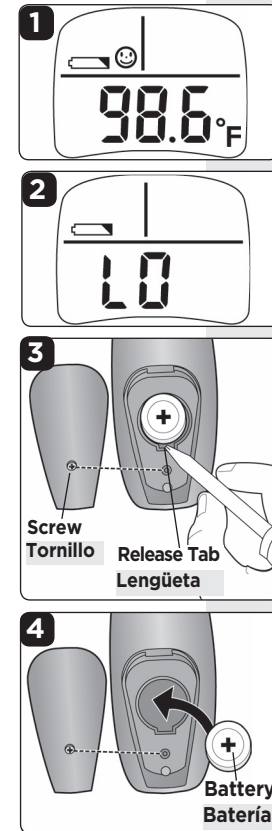
⚠ WARNING:

- Keep battery out of child’s reach. Swallowing battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode.
- Used batteries should be disposed of properly. Do not dispose of batteries in fire, recharge, put in backwards, disassemble, mix with used or other battery types – they may explode or leak and cause injury.

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©2009 Dorel Juvenile Group. All Rights Reserved. Todos derechos reservados.