

## LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments

- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

Reorder No. 10466

Manufactured for:  
Dynarex Corporation  
10 Glenshaw Street  
Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)  
Made in China



# Homecare Half-Length Bed Rail Cross Bar Mount



**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)



**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R191018

## WARNINGS

Read instruction carefully before installing. Before use, ensure that the bed rail is assembled and installed as described, that all attaching hardware is securely tightened, and that the bed rail is securely in place.

**DO NOT** use the bed rail as a handle for getting into or out of bed; this could result in personal injury and damage to the bed rail.

**DO NOT** exert side pressure on the bed rail; this could deform or break it.

**DO NOT** use the bed rail as a push handle for moving the bed; this could deform or break it.

**DO NOT** use the bed rail if a proper fit in installation cannot be achieved. Contact your dealer.


## INSTALLATION

1. Prepare the bed in a flat position. The carton should include the following:




2. Place crossbrace on the head end of the frame between the first and second spring.
3. Turn the barrel so that the lock plate on the locking collar is facing down.

4. Insert the lock plate on the far side of the bed so that it is under the lip of the frame.

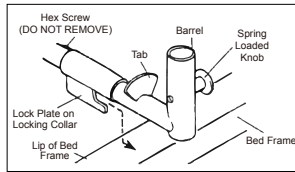


5. Push the crossbar together to allow for the locking collar to enter inside the frame.
6. Slowly release the crossbar so the locking plate inserts into the lip of the bed frame on the near side.


7. Rotate the barrel so that the tab is facing down thereby locking the crossbar into position.

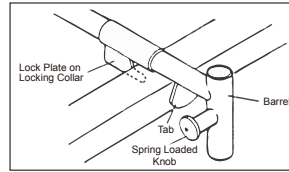

8. Place the second crossbrace on the bottom of the back rest panel of the frame. Repeat steps 2-6 to secure the crossbrace to the bed.

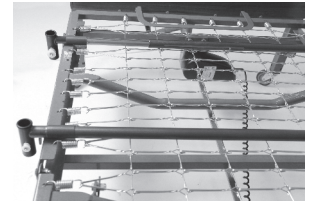

9. Insert one bed rail into the barrels on one side of the bed, ensuring that the spring loaded knob is fully engaged.
10. Insert the other bed rail into the barrels on the other side of the bed, ensuring that the spring loaded knob is fully engaged.

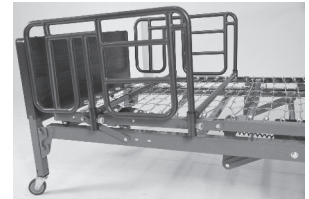

## OPERATING INSTRUCTIONS

### LOWERING/RAISING THE RAIL

1. Hold the rail, while pulling the spring loaded knobs.
2. Lower or raise the rail to the desired height.
3. Release the knobs and lower or raise the rail until the spring loaded knob "clicks" into position (unless the rail is at the lowest position).
4. Repeat steps 1-3 for the opposite end of the bed rail.