
Ear Otoscope


User's Guide IH481

Thank you for purchasing this Safety 1st product. We're proud to be your choice to help you keep your child healthy and safe. Please read these instructions completely before using this product.

Basics of Your Ear Otoscope

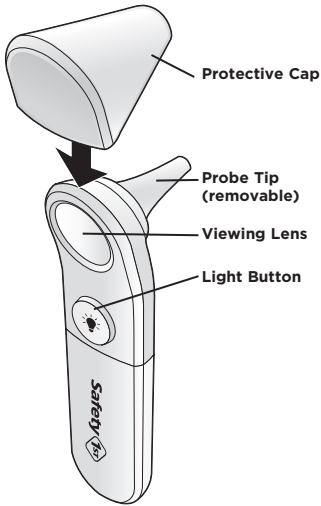How to Replace Batteries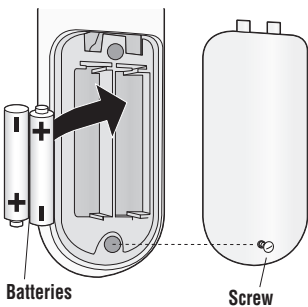

- Use a small screwdriver to loosen screw. Remove battery cover.
- Insert 2 AAA alkaline batteries (included). Position batteries correctly with the positive "+" side facing as shown on the battery cover.
- Place the battery cover on the unit and secure the screw.

FCC Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

RADIO AND TELEVISION INTERFERENCE

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes and Modifications not expressly approved by the manufacturer or registrant of this equipment can void your authority to operate this equipment under Federal Communications Commissions rules.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Ear Otoscope), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse. This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages. **SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.**

How to Use Your Ear Otoscope

Please read this section carefully and all the way through before using this product.

- Position ear canal for viewing** (Figures A and B).
 - 0 - 12 months:** Gently pull the ear straight back.
 - 1 year and older:** Gently pull the ear up and back to straighten the ear canal.


- Gently insert probe tip into ear canal.**
If pain occurs, remove probe tip immediately and consult physician.


- Press the light button and look through the viewing lens.**
Review "Reference Images" in the package (or scan this QR code) to see examples of what a healthy and infected ear looks like.

- Contact your physician if:**
 - Signs of ear infection are detected. Not all ear infections will be immediately visible. You may want to check again later or contact your physician if symptoms persist.
 - Your child has ear tubes that are no longer visible on the eardrum.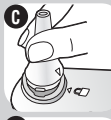

- Remove probe tip for cleaning** (Figures C and D).
 - To unlock, rotate tip to align notch on base unit and remove. See "Cleaning Probe Tip" section.
 - To lock, align tab on tip with notch on base unit.
 - Insert tip and rotate to align triangle indicators.

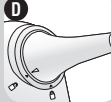NOTES: We recommend taking a "baseline" viewing of your child's ear when all is well. You will see what his or her eardrum looks like normally and where the ear tubes are located.

This device should not be used as a substitute for a medical diagnosis. Only a medical professional can make a diagnosis of a medical condition. It is important that you always seek medical treatment from a qualified medical professional.

DO NOT use any replacement parts not provided and approved by Dorel Juvenile Group. **DO NOT** substitute parts or try to modify the product in any way. Parts not provided and approved by Dorel Juvenile Group may result in an unsafe product.

WARNING:

- Keep these instructions as they contain important information.
- NEVER** push the probe tip too far into the ear.
- Always wash and dry tip thoroughly before and after each use.
- NEVER** use in baby's mouth or nose
- This product is not a toy. Do not allow children to play with it. When not in use, keep out of reach of children.
- This product contains small parts. Keep away from children at all times.

BATTERY CAUTION:

- We recommend alkaline batteries for longer battery life. **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon zinc) or rechargeable (nickel-cadmium) batteries. **Non-rechargeable batteries are not to be recharged.** **Rechargeable batteries are to be removed from the unit before being charged.** **Rechargeable batteries are only to be charged under adult supervision.** **Only batteries of the same or equivalent type as recommended are to be used.** **Batteries are to be inserted with the correct polarity.** **Exhausted batteries are to be removed from the unit.** **The supply terminals are not to be short-circuited.** **Check that all contact surfaces are clean and bright before installing batteries.** **Used batteries should be disposed of properly.** **Remove batteries when stored for long periods of non-use.**

Care and Maintenance

Cleaning Lens:

- Breathe onto lens surface, then wipe gently with lens tissue, soft cotton T-shirt type fabric or micro-fiber cloth.
- If that doesn't work, apply a small amount of lens-cleaning solution to the lens-cleaning tissue or soft cloth and gently wipe lens.
- Use of facial tissue, paper towels or polyester-based fabric may scratch lens and is not recommended.

Cleaning Probe Tip:

- Remove tip from unit to clean.
- Wash in warm soapy water and rinse well.
- Air dry thoroughly before storing.

Cleaning Base Unit:

- DO NOT** submerge base unit with electronics in water.
- DO NOT** place in microwave.
- Wipe with damp cloth.

Troubleshooting

- Install new batteries and retry function before calling Consumer Care on electronics.
- To address any issues, please visit us at www.safety1st.com to chat with us there. You can also text us at (812) 652-2525 or call Consumer Care at (800) 544-1108.

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