



Accel[®]
PREvention^{™/MC}
RTU
One-Step
Surface
Cleaner and
Disinfectant



Tuberculocidal, Virucidal, Bactericidal, Fungicidal,
Broad-Spectrum Sanitizer.

A ready-to-use cleaner disinfectant based on
proprietary AHP[®] technology to deliver fast,
effective cleaning performance.

Features

- One-step cleaner disinfectant
- Powered by AHP[®] technology
- No VOCs, no added fragrance and no NPEs
- Environmentally responsible - active ingredient breaks down to oxygen and water

Benefits

- Effective against key pathogens including MRSA, VRE, TB, and Norovirus
- Kills Norovirus, Poliovirus, and HIV and is Tuberculocidal, all in 3 minutes
- 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces
- Cleans, disinfects, and deodorizes in one labor saving step

Applications

- Clean and disinfect high-touch surfaces in 3 minutes
- Non-irritating to eyes and skin



Accel® PREvention™/MC RTU

One-Step Surface Cleaner and Disinfectant



Use Directions

As a One-Step Cleaner Disinfectant*:

- Pre-clean heavily soiled areas.
- Apply product by coarse trigger spray to hard, non-porous inanimate surfaces.
- All surfaces must remain wet for 3 minute.
- Wipe surfaces dry, rinse, or allow to air dry.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey Care representative.

* Refer to product label for complete use instructions

Product Specifications

| Description | |
|----------------|-------------------|
| DIN | 02448521 |
| Certifications | EcoLogo |
| Color/Form | Colorless, liquid |
| pH | 2.1 |
| Scent | Characteristic |
| Shelf Life | 2 years |

Available Items

| SKU Code | Description/Package Size | Dilution |
|-----------|-------------------------------------|--------------|
| 100906589 | 12 x 1 L / 1.06 U.S. Qt. Bottles | Ready-to-Use |
| 100908439 | 4 X 4 L / 1.06 U.S. Gal. Containers | Ready-to-Use |

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at www.sealedair.com or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.

www.sealedair.com

© 2016 Sealed Air Corporation. All Rights Reserved. PSS90512 (16/293)