



# Tracheostomy Adult Flex Connector

15 mm OD × 22 mm OD/15 mm ID

## DESCRIPTION

The flex connector is to provide added flexibility and mobility for ventilator circuits. The flex connector can extend and contract for easy length adjustment. The flex connector can expand from 5 cm to 6.5 cm.

## INSTRUCTION FOR USE

1. Install the Flex Tubing between the endotracheal or tracheostomy tube and the breathing circuit.
2. Adjust Flex Tubing for length and position.

## WARNINGS

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Do not use excessive force when expanding tubing. This can result in loss of pressure and/or volume.
- Do not soak, rinse, wash or sterilize this product.

## TECHNICAL DATA

<b>Dead Space</b>	8 mL
<b>Weight</b>	6.5 g
<b>Connections</b>	22M/15F-15M

## SYMBOLS

SYMBOL	SYMBOL NOTES
	Consult instructions for use
	Cautions and Warnings
	Keep dry
	Keep away from heat and sunlight
	Do not re-use
	Do not use if package is damaged
	Not made with natural rubber latex
	Not Manufactured with di(2-ethylhexyl)phthalate
	Batch code
	Prescription use only



**SYMBOL GLOSSARY**  
 For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)