

 Manufactured for:  
Dynarex Corporation  
10 Glenshaw Street  
Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)  
Made in Turkey

Reorder No. 30400



# Advantage Filter

## Universal Respiratory B/V Filter

### Bacterial: >99.998% • Viral: >99.96%



**CAUTION:** Federal (USA)  
Law restricts this device to sale  
by or on the order of a physician



**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

## DESCRIPTION

The filter is a disposable single-use device indicated for patient during the delivery of ventilator gases. The filter provides filtration for reducing possible cross contamination between patient and equipment. The filter is for use in hospital, ICU, anesthesia, respiratory therapy, during transport and with resuscitators.

## INSTRUCTIONS FOR USE

Place the filter between the breathing circuit and the equipment being used. Always replace the filter after each patient. When used continuously on a single patient, change the filter every 24 hours or more frequently as required.

## WARNINGS

Please perform control tests for any blockage and leakage on the product and all respiratory line before starting the treatment.

1. The filter has been manufactured for single patient use and maximum recommended hours of use is 24 hours.
2. This filter is only used between the equipment being used and breathing circuit.
3. The product is manufactured under clean room conditions. Do not use if the product and product package is damaged.
4. This product should be used only and exclusively under Physician or licensed practitioner.
5. Replace the filter in the event of increased resistance due to secretion from the patient.
6. The additional dead space of this device must be taken into consideration when used.

## TECHNICAL DATA

<b>Tidal Volume Range</b>	150-1500 mL
<b>Resistance</b>	at flow 30 L/min 0.68 cm H <sub>2</sub> O at flow 60 L/min 1.4 cm H <sub>2</sub> O
<b>Dead Space</b>	45 mL
<b>Weight</b>	21 g
<b>Connections</b>	Patient Side: 22M/15F Machine Side: 22F/15M
<b>Filtration Efficiency</b>	Bacterial: >99.998% Viral: >99.96%

## SYMBOLS

SYMBOL	SYMBOL NOTES
	Consult instructions for use
	Cautions and Warnings
	Keep dry
	Keep away from heat and sunlight
	Do not re-use
	Do not use if package is damaged
	Not made with natural rubber latex
	Not Manufactured with di(2-ethylhexyl)phthalate
	Batch code
	Prescription use only