



Stand Up Cane

Black



- Three contact points provide added stability
- Comfortable rubber hand grip
- Wrist carry strap
- Self-standing
- Adjustable height (33"-37")
- 300 lb. weight capacity



***No bending to pick up
fallen cane***



Self-standing



CAUTION:
This product contains natural rubber latex which may cause allergic reactions in some individuals.

Intended Use and Limitations

- This cane is a walking aid and is not intended to support the full weight of the user
- It is not intended for use on ice or other slippery surfaces and will not stand alone on uneven surfaces or plush carpets

Safety Precautions

- This product should not be used without proper instruction from a healthcare professional
- DO NOT use unless individual segments are securely joined together and push button is fully engaged
- DO NOT use if elastic cord is damaged
- When assembling segments, keep fingers away to avoid being pinched
- DO NOT over stretch
- DO NOT use cane if handle or base are loose or damaged
- Applying downward pressure on the base while lifting up on the handle can cause the cane segments to separate and the cane to collapse
- Inspect carefully before each use

Set Up Instructions

- Remove cane from packaging
- Unfold cane and ensure that cane segments are securely joined

To Adjust Height

- Loosen locking collar by twisting clockwise
 - Depress push button and slide lower section upward or downward to desired height
 - Secure locking collar by turning counterclockwise until tight
- WARNING:** Before use, ensure that the push button protrudes fully through adjustment hole and that locking collar is securely tightened

Maintenance

Check elastic cord before every use. DO NOT use cane if cord is cut, frayed or damaged. Periodically check the tip for wear or damage. Replace as needed. Prior to each use, ensure push button and locking collar are secure and functioning properly.

 **Manufactured for:**
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China R200224



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php