Invacare® Court-Side Glides™

EN Assembly, Installation and Operating Instructions

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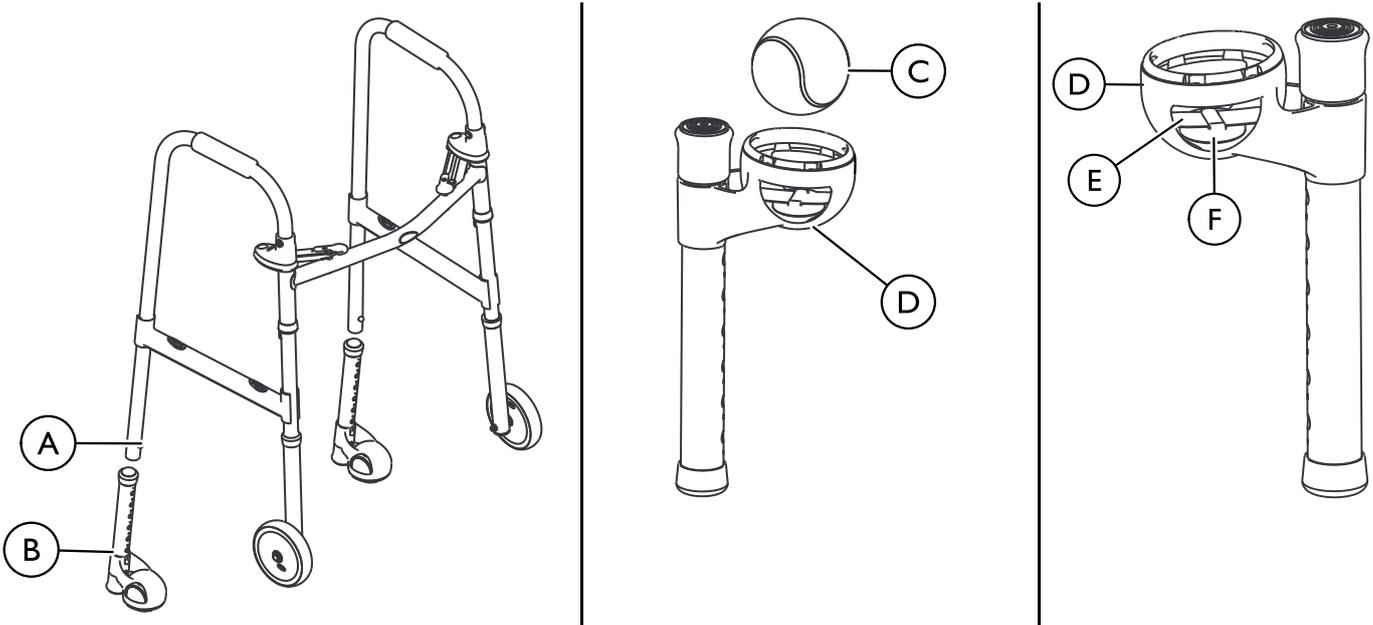
I.1 Figures

I.2 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING! Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION! Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

2 Safety

2.1 General Guidelines

WARNING!

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Court-Side Glides are to be used ONLY on Invacare Walkers that are equipped with 3 inch or 5 inch fixed wheeled front leg extensions.
- Court-Side Glides are not for use on Rollites™ or Walkers with 3 inch or 5 inch swivel casters.

WARNING! STABILITY

- Check to make sure that rubber tips and/or tennis balls are not ripped, worn or missing. Replace them before using the walker.
- Court-Side Glides are used on rear legs only and should be positioned toward the interior of the walker.
- Extra caution should be taken when using Court-Side Glides as they are not intended to grip the floor.
- All four legs must be placed on the floor for stability and support. Short steps should be taken to ensure that weight is directly over the walker.
- After installation and before use, make sure that the walker and accessories are properly and securely locked in place and hardware is tight.

WARNING! ACCESSORIES

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

3 Usage

3.1 Installing the Court-Side Glides (CSG)

WARNING!

- Make sure all snap buttons fully protrude through adjustment holes of each leg extension and are securely locked in place.
- Make sure tennis ball is secure inside housing and will not fall out.

1. Press the snap buttons **A** on the rear legs to remove the two existing rear leg extensions. Keep for future use.
2. Press the snap buttons and install the CSG leg extensions **B** onto the rear legs.
3. Adjust the CSG leg extensions to one of the adjustment holes. Refer to 3.2 Height Adjustment, page 2.

3.2 Height Adjustment

All client height ranges are approximate.

MODEL NO.	NEW APPROX. CLIENT HEIGHT
629I, 6240, 628I with 5 inch wheels	5 ft 6 in to 6 ft 4 in 168 cm to 193 cm
629I, 6240, 628I with 3 inch wheels	5 ft 4 in to 6 ft 4 in 163 cm to 193 cm
629IJR, 6240JR, 628IJR with 5 inch wheels	4 ft 8 in to 5 ft 6 in 142 cm to 168 cm
629IJR, 6240JR, 628IJR with 3 inch wheels	4 ft 6 in to 5 ft 6 in 137 cm to 168 cm
6300-JRA WalkLite	4 ft 11 in to 5 ft 6 in 150 cm to 168 cm
6300-ATA WalkLite	5 ft 4 in to 6 ft 163 cm to 183 cm

WARNING!

- The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one inch lower than the front.

Press the snap button and adjust CSG leg extensions to proper height adjustment hole.

Different hole positions may be used on front and rear leg extensions to level the walker. Make sure all four legs are in contact with the floor.

If the rear of the walker is 1 inch lower than the front of the walker, less pressure is required to engage the rubber tip.

3.3 Rotating/Replacing Tennis Ball

The degree of free rotation depends on the pressure applied to the rear legs of the walker and on the surface the tennis ball contacts.

Perform one of the following:

- If the tennis ball **C** is worn, rotate it in the housing **D** so that the desired surface of the ball contacts the ground when in use.
- Replace the tennis ball:
 1. Push the tennis ball out of the housing.
 2. Press a new tennis ball into the housing.

3.4 Installing/Removing Locking Clip

1. Remove tennis ball from housing **D**. Refer to 3.3 Rotating/Replacing Tennis Ball, page 2.
2. Lift up the loose side of the metal clip **E**.

WARNING!

- DO NOT force the metal clip as it may result in injury or damage to the CSG leg extension

3. Perform one of the following:
 - **To Install:** Slide new locking clip **F** through top opening in housing and onto the metal clip. Position the clip to the center of the housing.
 - **To Remove:** Slide locking clip out from underneath metal clip and out top opening in the housing.

4. Press the metal clip into original position and replace tennis ball. Refer to 3.3 Rotating/Replacing Tennis Ball, page 2
5. Repeat steps 1-4 on remaining rear leg extension as necessary.

4 Warranty

4.1 Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the Court-Side Glides (excluding tennis balls) to be free from defects in materials and workmanship for five years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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