

LIMITED LIFETIME WARRANTY

Reorder No. 10167

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.



Travel Walker

With 5" Wheels



WEIGHT CAPACITY: 300 LB.

WARNING:

Cancer and Reproductive Harm
www.P65Warnings.ca.gov



SYMBOL GLOSSARY

For an explanation of symbols
used in Dynarex packaging, visit
dynarex.com/symbols.php



 **Manufactured for: Dynarex Corporation**
10 Glenshaw Street • Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

R220914

WALKER INSTRUCTIONS

Easy “pull up” trigger release for opening and closing.

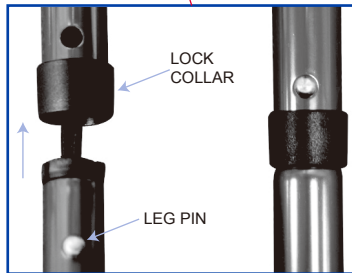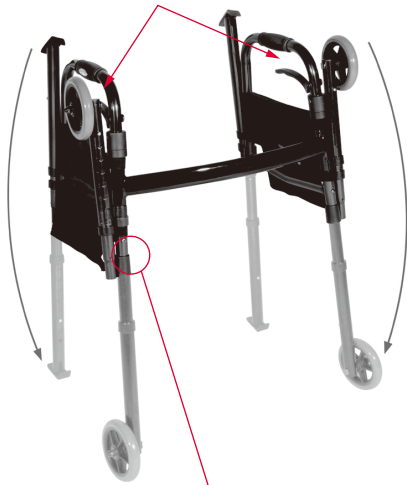

GLIDE SKIS


Be sure glide ski is centered on leg and facing forward with lip turned up at the front as shown.

IMPORTANT SAFETY NOTICE:

Your walker features flip up legs that can be attached to the frame of the walker for easy storage. Be sure that when the legs are turned down for use as a walker, the leg pin is inserted into the lock collar as shown above. Be sure to check the leg height adjustment hole position when setting up or folding for storage in the carry bag to be sure all hole adjustments are the same.

CUSTOM WALKER ADJUSTMENTS

With your arm extended down, handle should be at wrist height. With your hand placed on the hand-grip, your elbow should be flexed 20° - 30°.

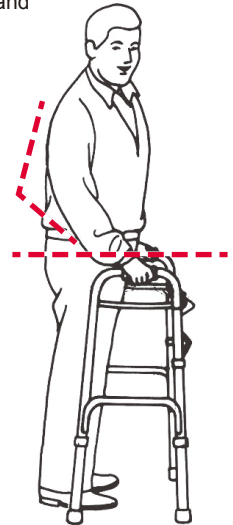

CAUTION:

Be aware of the following:


- Electrical cords in way
- Liquids on floor
- Throw rugs
- Toys or debris
- Furniture
- Doors and door steps
- Gaps in floor

WARNING:

1. Use wheels on front legs only or on all four legs with appropriate rear wheel brakes.
2. Do not push one side of the walker ahead and then other side without lifting the walker first.
3. Use Dynarex accessories only. Use of other manufacturers' accessories can result in improper fit and risk fall and/or injury.
4. Do not use on stairs or escalators.

Wheelchair to Walker Transfer

Making certain wheels on the wheelchair are locked, lift your body by pushing down on the armrests and carefully transfer one hand at a time to the walker.


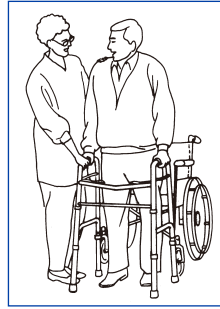


Wheelchair to Walker Transfer

(with lower extremity injury)

Make sure the wheels on the wheelchair are locked.

- For injuries to the right leg or foot: Lift your body by pushing down on the armrests and carefully transfer first the left hand, then the right hand to the walker.
- For injuries to the left leg or foot: Lift your body by pushing down on the armrests and carefully transfer first the right hand, then the left hand to the walker.

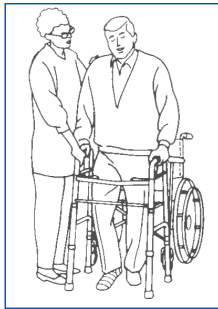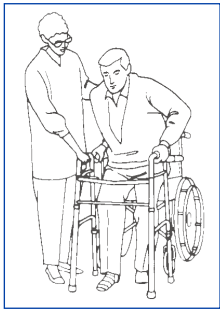

Three-point Gait

Step 1

Align the middle of your foot with the back legs of the walker.



Step 2

Slide the walker forward and place it comfortably in front of you.



Step 3

Carefully, walk forward one step at a time.



FOLDING

1. Pull up on release lever.



2. Fold the back frame legs, on both sides.



5


3. Fold the front frame legs, on both sides.



4. To make the walker more compact for storage and transport, identify push pin above leg collar, depress pin and pull leg out of tube.



5. Fold legs up and insert into receptacle on frame. Repeat for each leg.

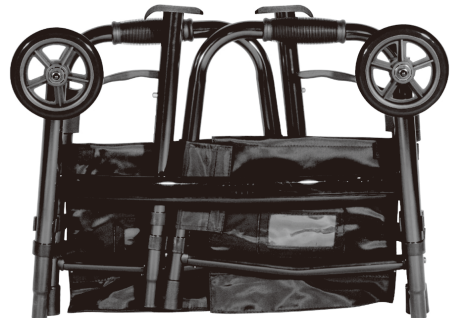

6