



# Oxivir® HC Wipes

**Sanitizer • Cleaner • Disinfectant • Deodorizer \*Virucidal • Bactericidal**

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance with broad surface compatibility.

Equipment surface compatible formula is gentle on surfaces, safe on gloves and designed for sensitive equipment surfaces.

## Features & Benefits

- One-step cleaner disinfectant powered by AHP® technology
- Effective against key pathogens – MRSA, Norovirus, and VRE
- Kills HBV, HCV and HIV in 1 minute
- No VOCs, no added fragrance and no NPEs. Environmentally-responsible – active ingredient breaks down to oxygen and water
- Compatible with most hard, non-porous surfaces
- Convenient – pre-wetted wipes make even the toughest job easier

## Applications

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin





# Oxivir® HC Wipes

Sanitizer • Cleaner • Disinfectant • Deodorizer \*Virucidal • Bactericidal

## Use Instructions

### As a One-Step Cleaner Disinfectant

- Pre-clean visibly soiled areas.
- Pull towelette from dispenser and wipe hard, non-porous environmental surfaces.
- All surfaces must remain visibly wet for 1 minute.
- Allow to air dry or rinse with potable water if necessary.

**Note:** All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

**Note:** Prior to use on electronic equipment or components, please refer to manufacturer's instructions for appropriate use.

Technical data	Oxivir® HC Wipes
EPA Reg. No.	70627-80 (Canisters, Buckets, Refills)
Color/Form	Colorless, wipe
pH	3
Scent	Characteristic of ingredients
Shelf Life	3 years

Product	Pack size	Dilution	Product code	
Oxivir® HC Wipes	12 x 160 ea, Small Wipes, Canisters	Ready-to-use	101103738	
Oxivir® HC Wipes	4 x 160 ea, XLarge Wipes, Buckets	Ready-to-use	101103753	
Oxivir® HC Wipes	4 x 160 ea, XLarge Wipes, Refills	Ready-to-use	101103752	

## Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

AHP® and Design are trademarks of Diversey, Inc.