MOBILITY DESIGNED

AIR-GO™

Walker

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Andador

Instructions For Use

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
MD30100BK, MD30100BL

VER.A.6.25

Where Real Life Inspires Innovation

Designed with insight from real lives, elevated by next-generation engineering and innovative materials — because everyday support should feel extraordinary.

- The M+D by Drive® Team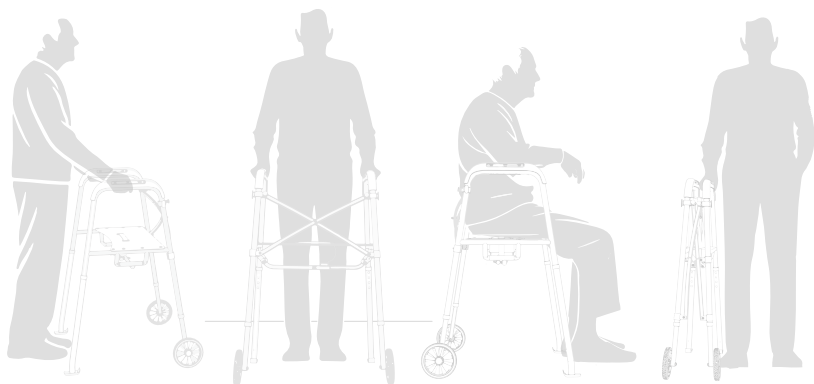

Parts List

1 Air Handle Grip

2 Seat

3 Seat strap

4 Cross bar

5 Cross bar
locking clip

6 Cross bar lever

7 Scissor frame

8 Left frame upper

9 Left front leg

10 Left wheel

11 Right rear leg

12 Left rear leg

13 Right frame
upper

14 Right front leg


15 Right wheel

16 Glide Ski

17 Accessory clip

18 Bag clip

19 Snap button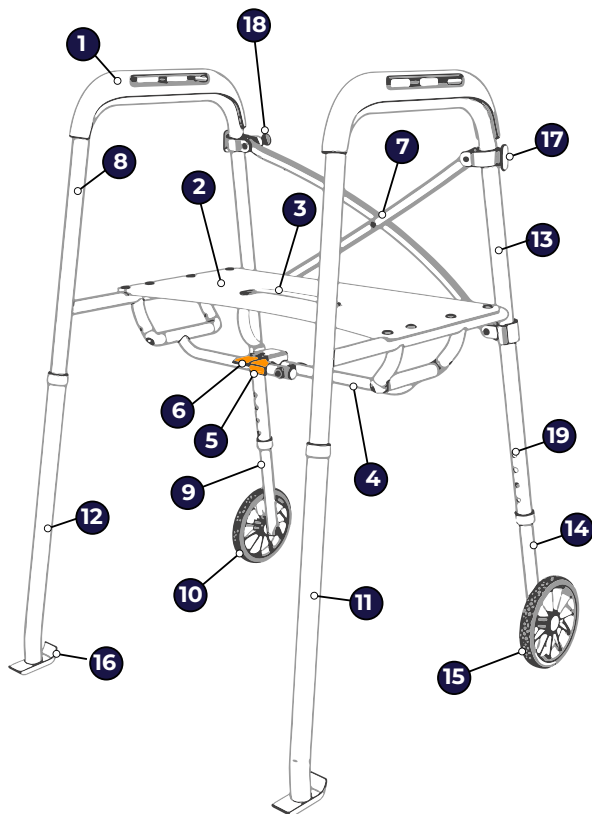

WEIGHT CAPACITY: 300 LB / 136 KG

USER HEIGHT: 4'9" - 6'7" (1.44 m - 2.0 m)


Instructions



Tutorial Videos

Scan QR code for link to tutorial videos

- 1 Assembly:** Click in the legs. Legs with wheels go in the front, and legs with skis go in the back.
NOTE: Wheels should be on outside of the leg, facing away from the middle of walker to help align the letter on the leg with the correct hole for each leg.

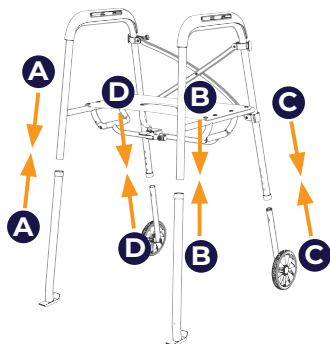
- 2** To open the walker for walking mode, pull the handles away from each other. Push down on the seat or cross bar lever, to help open the walker.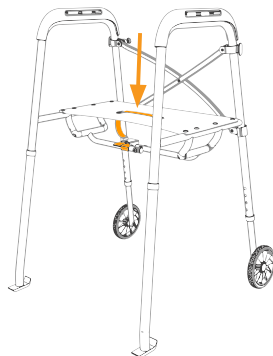
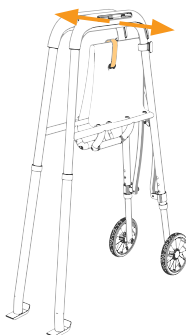


Instructions

- 3** To lock the walker in the open position, push down on the orange lever under the seat. You should feel a click as the cross bar locks in position.

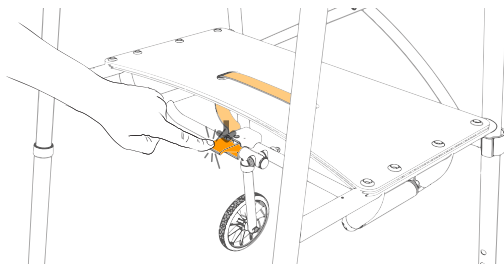
- 4** To fold the walker, pull the orange lever located on the cross bar under the seat upwards, and then pull up on the seat strap to fold the walker.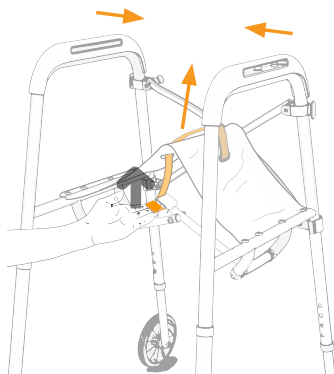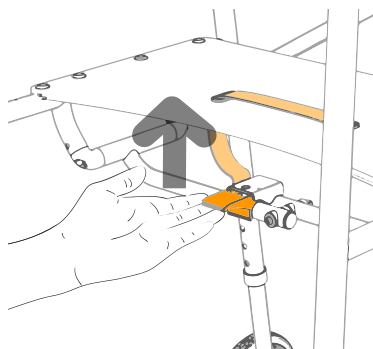



Instructions

- 5 Seat Mode:** The Air-Go Walker™ should only be used as a seat on stable, level surfaces. Only use the seat when the walker is open and the cross bar is locked. Seat height can be adjusted by adjusting leg length. All legs should be at the same height for use.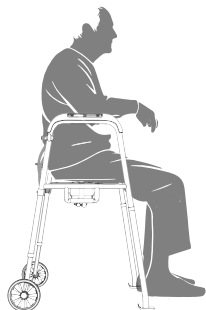

Height Adjustment

- 1** The ideal ergonomic height for a user is one in which the user is not bending over to reach the handles, but is in a stable vertical position with their elbows slightly bent to touch the handles.
- 2** Push the snap button on each leg and extend or retract the leg tubes to desired height.
- 3** To lock the walker in the open position, push down on the orange lever under the seat. You should feel a click as the cross bar locks in position. NOTE: The seat height changes depending on the overall height of the walker. You may want to adjust the height differently for sitting or walking.

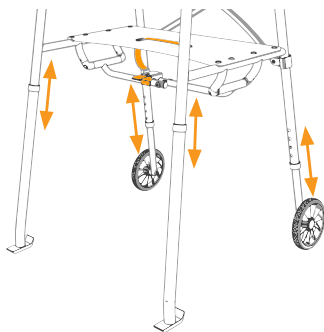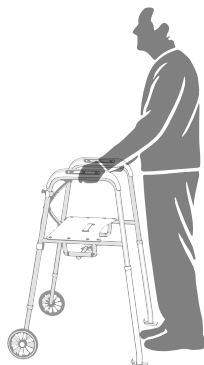Maintenance and Cleaning

Brush off dust and dirt or wipe with a damp cloth.

M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
Mobility Designed® es una marca registrada de Medical Depot, Inc.
2025 Medical Depot, Inc. exerçant son activité sous le nom
Depot, Inc.