**Important:** Be sure to retain this manual for future reference and read it completely before using Evenflo Milk Storage Bag Adapters. Refer to your pump manual for additional instructions and warnings.

## BEFORE FIRST USE AND AFTER EACH USE

For best results and to avoid potential damage, hand wash (avoid boiling and high temperature dish washing). Thoroughly dry before use. This product contains metal - **DO NOT MICROWAVE.**

## INSTRUCTIONS FOR USE


### Installation:

1. Screw adapter **A** into pump flange base **B**.
2. Pinch adapter clips to open position.
3. Slide open milk storage bag (not included) **C** between clips with adapter funnel below zipper.
4. Release adapter clips to closed position.

### Removal:

1. Hold flange with one hand around the clip area – this same hand will be used to open clips.
2. Press open clips and slide bag down, off of the adapter.
3. Put flange down and seal milk storage bag according to instructions provided by your milk storage bag manufacturer.

*Note: If double pumping, you can secure one flange to your body with your forearm while removing the other flange from your breast with the same hand. Follow Steps 1-3 above.*

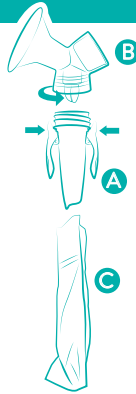**WARNING:** DO NOT use this product if it is broken or any parts are missing. The spring clips are a potential pinch point. Contains small parts which could pose a choking hazard to small children. Keep pump parts and accessories out of children's reach. Pump and accessories are personal care items, for one user only.

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