SAVE THESE INSTRUCTIONS

Safety Summary

To ensure the safe and proper installation of the WARNING/CAUTION labels, these instructions MUST be followed:

⚠ WARNING

DO NOT install or use this kit without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may occur.

DO NOT use the patient lift without installing these Warning and Caution labels.

The label kit contains the following labels:

PART NUMBER	DESCRIPTION OF LABEL	REPLACE LABEL
I130267B	Patient Lift Warning Label 450	I073698B
I130229B	Patient Lift Warning Label 600	I070965A
I130265A	Mast Bolt Warning Label	I130200A

Installing Warning/Caution Labels

1. Determine the proper location for each label. Refer to FIGURE 1.

NOTE: The English labels are included in FIGURE 1 for ease in determining proper location.

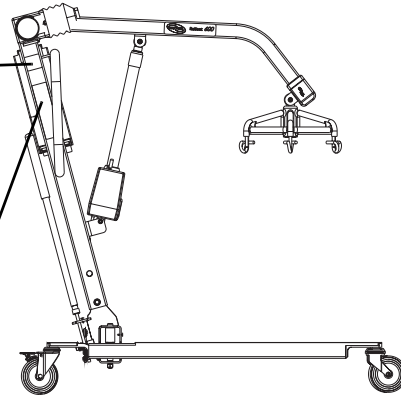
2. Peel one (1) label from the adhesive backing.
3. Position the appropriate warning or caution label directly over the English label at the location determined in STEP 1.
4. Repeat STEPS 1-3 for each label.

For more information regarding Invacare products, parts, and services,
please visit www.invacare.com

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⚠ WARNING
Mast pivot under rubber boot must be tight to ensure safe use of your patient lift. Bolt must be checked at least every six (6) months in conjunction with periodic maintenance. See Owner's Manual.
P/N 1130200 Rev. A 01/05

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NOTE: Electric Lift shown. Locations are the same on the Hydraulic/Manual lift.

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⚠ WARNING

BEFORE using the Patient Lift, **READ** and **UNDERSTAND** the Owner's Manual for proper operation and safety procedures.
⚠ WARNING
WEIGHT LIMITATION 600 lbs.
The Invacare Patient Lift is **NOT** a transport device. Moving a person suspended in a sling over **ANY** distance is **NOT** **RECOMMENDED**.
DO NOT roll casterbase over deep carpet, raised carpet bindings, door frames or any uneven surfaces that may cause the Patient Lift to tip over.
DO NOT lock the casters of the Patient Lift when lifting an individual. Casters **MUST** be left unlocked to allow Patient Lift to stabilize during lifting procedures.
USE ONLY genuine **INVACARE SLINGS** and **LIFT ACCESSORIES**.
USE ONLY **RELIANT 600 Model** Components for maintenance and replacement.
Refer to Owner's Manual for periodic maintenance procedures.
Customer Service: 1-800-333-6900
Rev. A 04/05 1070965

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⚠ WARNING

BEFORE using the Patient Lift, **READ** and **UNDERSTAND** the Owner's Manual for proper operation and safety procedures.
⚠ WARNING
WEIGHT LIMITATION 450 lbs.
The Invacare Patient Lift is **NOT** a transport device. Moving a person suspended in a sling over **ANY** distance is **NOT** **RECOMMENDED**.
DO NOT roll casterbase over deep carpet, raised carpet bindings, door frames or any uneven surfaces that may cause the Patient Lift to tip over.
DO NOT lock the casters of the Patient Lift when lifting an individual. Casters **MUST** be left unlocked to allow Patient Lift to stabilize during lifting procedures.
USE ONLY genuine **INVACARE SLINGS** and **LIFT ACCESSORIES**.
USE ONLY **RELIANT 450 MODEL** Components for maintenance and replacement.
Refer to Owner's Manual for periodic maintenance procedures.
Customer Service: 1-800-333-6900
1073698 REV. B 07/05

FIGURE I Installing Warning/Caution Labels/Installer les étiquettes avertissements/attention

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© 2005 Invacare Corporation
trademarks of Invacare Corporation.
Invacare Corporation www.invacare.com