

# LIMITED LIFETIME WARRANTY

our Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.


The Warranty shall not apply under the following conditions: Problems arising from normal wear; Problems arising from failure to adhere to the product instructions; Problems arising from misuse, negligence, accident or improper operation, maintenance or storage; Problems arising from modifications or unauthorized repairs, parts or attachments; Products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)



**SYMBOL GLOSSARY**  
For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R181224

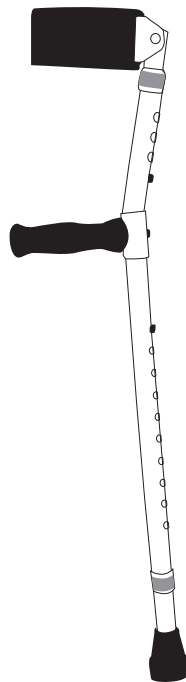

## Forearm Crutches




Manufactured for: Dynarex Corporation  
10 Glenshaw Street • Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)

Made in China

## ADJUSTING THE HEIGHT

1. Before use, adjust height of leg section between the tip and the hand grip to your specific needs by loosening the anti-rattle collar then depressing the push button, (Figure 1).
2. When desired height is reached, lock leg into place by aligning button into proper hole, (Figure 2).
3. Be sure leg is locked into place by pushing down on or pulling leg, then tighten the anti-rattle collar.
4. Repeat the same steps with the section between the hand grip and forearm collar.

**FIGURE 1**

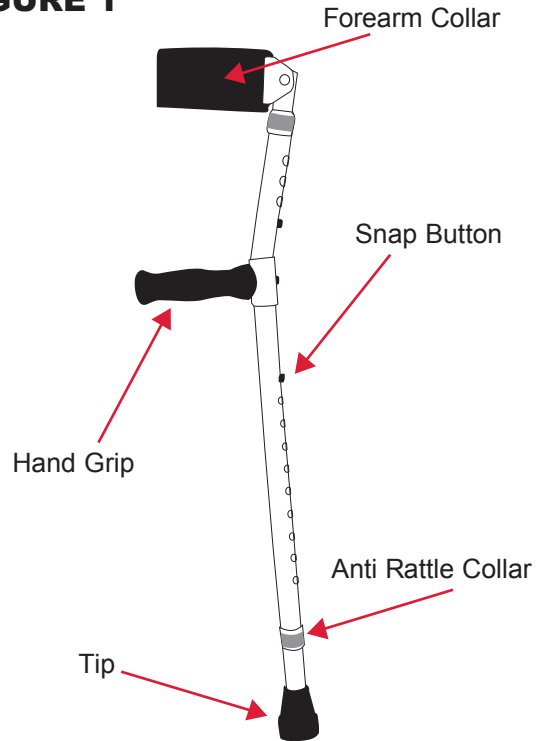

## CLEANING INSTRUCTIONS

1. Use a non-abrasive mild detergent or a household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product as damage may ensue.

**FIGURE 2**

