


 Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

Overbed Table

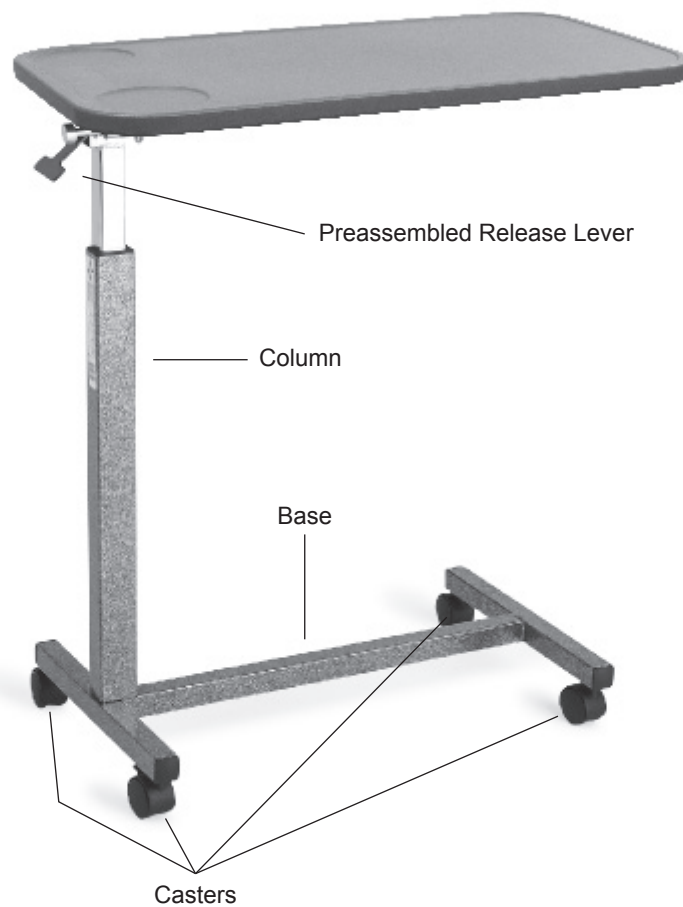

⚠️ WARNINGS

- **DO NOT** assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment, otherwise, injury to the user or damage to the table may occur.
- The adjustable support column is compressed during shipment. Once the stretch wrap is cut/removed, the column will spring out. **USE EXTREME CAUTION** when removing the stretch wrap from the column or possible injury may occur.
- After assembly and **BEFORE** use, ensure that **ALL** attaching hardware is securely tightened.
- When not in use, store out of the walking path.
- When in use, the weight should be evenly distributed on the table top. Distributed weight capacity is 50 lb.
- **DO NOT** use the overbed table as a stand assist or walking aid.

COMPONENTS

1 – Tabletop Assembly	2 – Non-Locking Casters	1 – Wrench	1 – 8 mm Allen Wrench
1 – Column w/Release Lever	2 – Locking Casters	2 – 8 mm Screws	1 – 6 mm Allen Wrench
1 – H-base	1 – Support Bracket	2 – 6 mm Screws	

ASSEMBLY

NOTE: The table will be assembled upside down.

- STEP 1** Position the tabletop assembly on a flat surface with the pre-drilled holes facing upward.



- STEP 2** Line up the plate on the column with the pre-drilled holes on the tabletop. Insert three 6 mm screws and tighten with appropriate allen wrench.



- STEP 3** Align the base on the column ensuring the threaded screw inserts are facing up. Line the support bracket with the holes and insert two 8 mm screws. Tighten with the appropriate Allen wrench.



- STEP 4** Insert the casters and tighten with the wrench supplied. It is recommended to assemble the locking casters on the column side.



- STEP 5** Position the overbed table upright and your table is ready for use.

ADJUSTING THE OVERBED TABLE

TO RAISE: Lift the tabletop UP to the desired height.

TO LOWER: Pull up on the release lever and adjust the table top DOWN to the desired height. **NOTE:** The column will lock into place when the release lever is



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

⚠️ WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov