

## LIMITED LIFETIME WARRANTY

Your Dynarex product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions: Problems arising from normal wear; Problems arising from failure to adhere to the product instructions; Problems arising from misuse, negligence, accident or improper operation, maintenance or storage; Problems arising from modifications or unauthorized repairs, parts or attachments; Products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

Reorder No. 10164



# Side Style Walker One Arm



#### SYMBOL GLOSSARY

For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

R181205

 Manufactured for: Dynarex Corporation  
10 Glenshaw Street • Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)

Made in China

## PRE-ASSEMBLY

Read the following instructions before attempting usage. If the directions are not clear, contact a healthcare professional or technician.


To achieve the proper adjustments and usage of the walker, consult with a therapist or other medical professional.

## OPENING / FOLDING THE WALKER

1. Grasp the Top Hand Grip of the walker. With other hand, push the Pivot Lock Handle down towards the Cross Brace.

NOTE: The Pivot Lock Handle MUST be directly on top of the Cross Brace.

2. Reverse step 1 to close the walker.

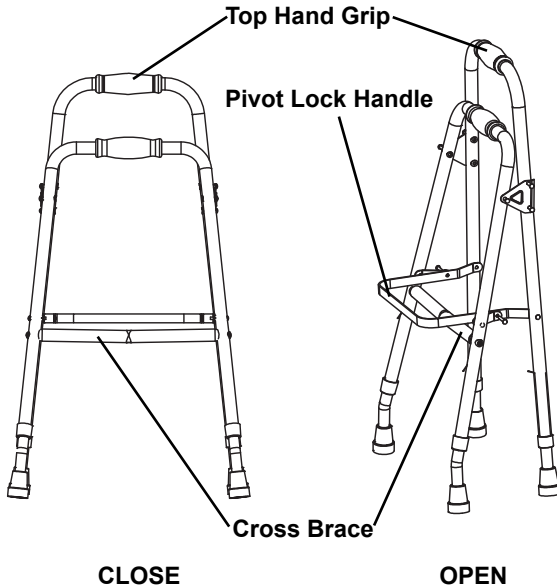


## ADJUSTING THE LEG EXTENSIONS

1. Adjust leg extensions by depressing the snap button through the appropriate extension and allowing it to snap through the appropriate hole on the walker leg frame.

NOTE: Make certain the snap buttons are fully engaged into the adjustment hole of each leg frame and are at the same height before using.

2. Repeat this procedure for each of the leg extensions.

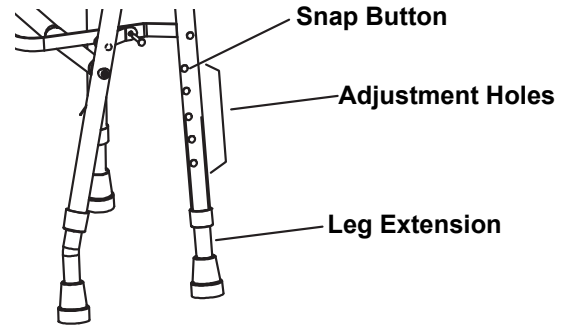

## ⚠️ WARNINGS

- Do not exceed the maximum user weight capacity marked on the product.
- Use for stability and assistance while walking; do not use to support full body weight.
- Make sure all snap buttons protrude all the way through the holes.
- All leg extensions must be set at the same height. Before using, double check that walker has been fully opened and is level to the floor or ground.
- The walker is not designed to hold items. Hanging objects on the walker could tip it over, potentially causing personal injury or damage.
- Periodically inspect rubber tips for wear and tear. If tips are damaged, replace them immediately.