AIR CANE™

Folding Cane

Canne Piante

Bastón Plegable

Instructions For
Use

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
Where Real Life Inspires Innovation

Designed with insight from real lives, elevated by next-generation engineering and innovative materials — because everyday support should feel extraordinary.

- The M+D by Drive® Team

Parts List

- ❶ Air Handle
- ❷ Snap Button
- ❸ Wrist Strap
- ❹ Anti-Rattle Collar
- ❺ Upper telescoping tube
- ❻ Lower collapsible tubes
- ❼ Air Tip

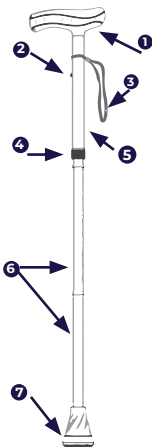Tutorial Videos:

Scan QR code for link to tutorial videos.

WEIGHT CAPACITY: 350 LB / 159 KG

USER HEIGHT: 4'9" - 6'7" (1.44 m - 2.0 m)

Proper Cane Height




For a more comfortable fit, ensure your shoe height is the typical height of the shoes you wear most frequently when using cane.

- 1 Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
- 2 Adjust your cane so that the top portion of the handle touches the inside of your wrist where it meets the palm of your hand.

Height Adjustment

- 1 The cane includes three folding sections that snap out automatically.
- 2 Turn the anti-rattle collar clockwise to loosen.
- 3 Depress the snap button.
- 4 Slide extension up or down until the snap button protrudes through the desired adjustment hole on the handle end of cane.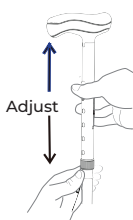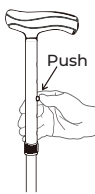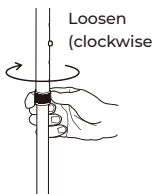


NOTE: An audible “CLICK” will be heard when the snap button securely protrudes through the adjustment hole.

Height Adjustment

- 5** Tighten the anti-rattle collar by turning it counterclockwise.
- 6** Repeat steps 2-5 until the cane is at the height determined as the proper cane height in the instruction manual.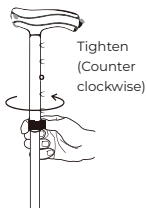

Cautions

- Use only after proper training under the direction of your doctor or physical therapist.
- Replace Cane Tips if they appear worn.
- Use extra caution on wet and slippery surfaces.


Safety Check

- With cane positioned per diagram, confirm that you can lean forward comfortably on the handle for weight support.
- The cane was not designed to bear a user's full weight. Improper use could lead to injury.
- Use extra caution on wet and slippery surfaces.
- If the handle or bottom base becomes loose, stop using cane immediately and contact customer service.

Safety Check


WARNING: Lifting handle while stepping on base may cause cane to separate and collapse.

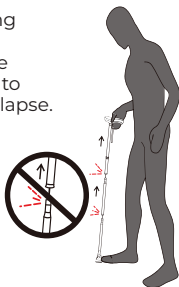Keep Fingers Away and Do NOT Over Stretch

IMPORTANT: Be sure segments are securely joined and that push button is fully engaged.

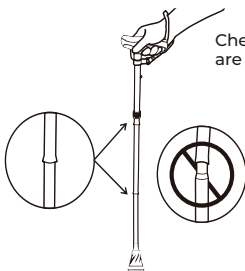Check that Joints are Secure

Maintenance and Cleaning

Brush off dust and dirt or wipe with a damp cloth.

Ensure the snap button releases and re-engages properly.

Inspect cane tip for wear and damage.

Replace any broken, damaged or worn parts immediately.

Maintenance and Cleaning

M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

Air-Cane™ Accessories:

Replacement Air-Go Cane Tip MD10202BK
Scan QR code for link.

Patents: www.drivemedical.com/patents

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
2025 Medical Depot, Inc. exerçant son activité sous le nom de
Inc.
Depot, Inc.