



Accel PREvention™ RTU

31

One-Step Surface Cleaner and Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A ready-to-use cleaner disinfectant based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus, HIV and Poliovirus providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 3 minutes. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces, while harnessing the power of AHP® technology. The AHP® active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPEs. It is non-irritating to eyes and skin
- Cleans and disinfects in one labor-saving step

Applications

- Clean and disinfect high-touch surfaces in 3 minutes
- Non-irritating to eyes and skin





Accel PREvention™ RTU

One-Step Surface Cleaner and Disinfectant

Use instructions

As a One-Step Surface Cleaner Disinfectant*:

- Pre-clean heavily soiled areas.
- Apply product to hard, non-porous inanimate surfaces.
- All surfaces must remain wet for 3 minutes.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

* Refer to product label for complete use instructions

Technical data	Accel PREvention™ RTU
DIN	02448521
Certifications	EcoLogo
Color/Form	Colorless, liquid
pH	2.1
Scent	Characteristic of ingredients
Shelf Life	3 years

Product	Pack size	Dilution	Product code	
Accel PREvention™ RTU	12 x 946 mL / 1 U.S. Qt. Bottles	Ready-to-use	101103731	🇨🇦
Accel PREvention™ RTU	4 x 3.78 L / 1 U.S. Gal. Containers	Ready-to-use	101103732	🇨🇦

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.