

Headrest Adapter Plate



Assembly, Installation and Operating Instructions

Part No. HR15

Yes, you can.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, do not use. Contact the dealer for further instructions.

SAFETY SUMMARY

The following recommendations are made for safe installation and use of the Headrest Adapter Plate:

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these **Warnings, Cautions and Instructions**, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.

INSTALLATION WARNING

Make sure that all screws and adjustments are tightened properly, otherwise serious injury or damage may result.

INSTALLING THE ADAPTER PLATE (FIGURE 1)

1. Perform one (1) of the following:
ULTIMATE AIR OR KSS BACK - Turn the adapter plate so the "A" side faces up.
CONTOUR U[®] - Turn the adapter plate so the "B" side faces up.

2. Align the center mounting holes on the adapter plate with the threaded holes in the bracket base.


NOTE: The slotted side of the bracket base should face out.

3. Secure the adapter plate to the bracket base with the four (4) mounting screws. Securely tighten with the 1/8-inch allen wrench.
4. Align the four (4) slotted holes in the adapter plate with the holes in the back cushion plate. Ensure that the side determined in STEP 1 faces up.

5. Perform one (1) of the following:
ULTIMATE AIR OR KSS BACK - Secure the adapter plate to the back with the four (4) 3/4-inch cap screws. Securely tighten with the allen wrench.
CONTOUR U[®] - Secure the adapter plate and two (2) threaded backing plates to the back with the four (4) 1/2-inch cap screws. Securely tighten with the allen wrench.

NOTE: The Headrest Adapter Plate Kit contains two (2) types of screws for securing the adapter plate to the back: 3/4-inch screws for the Ultimate Air or KSS back and 1/2-inch screws for the Contour U[®].

KSS OR ULTIMATE AIR BACK

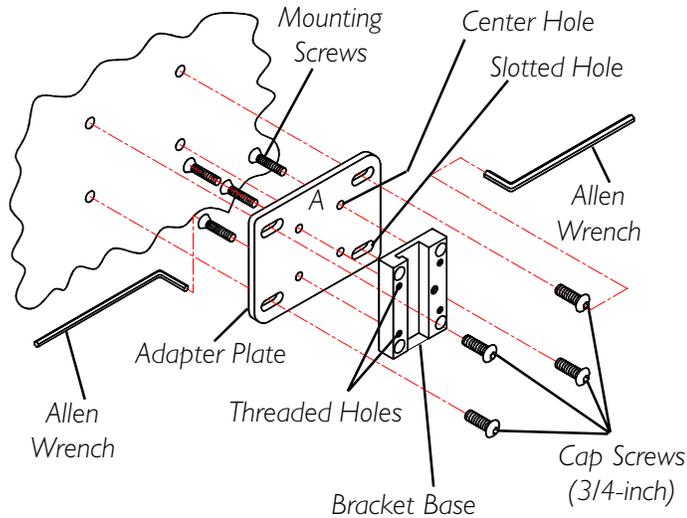

CONTOUR-U® BACK

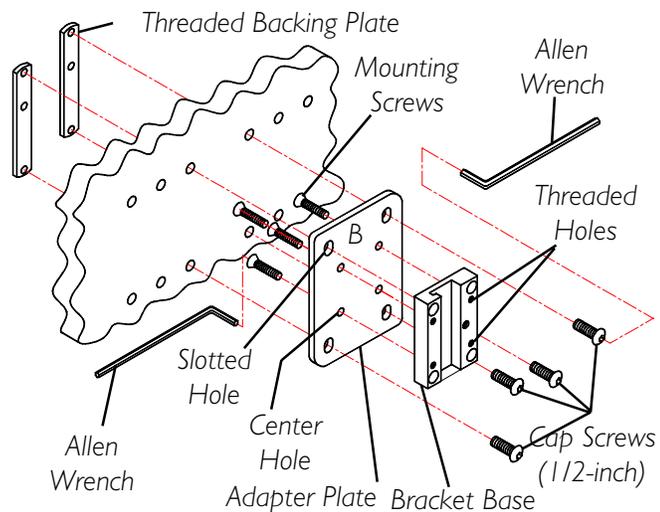


FIGURE I - INSTALLING THE ADAPTER PLATE



Yes, you can.™

Invacare Corporation www.invacare.com

USA
One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada
570 Matheson Blvd E Unit 8
Mississauga Ontario
L4Z4G4 Canada
800-668-5324

Invacare is a registered trademark of Invacare Corporation.
Yes, you can. is a trademark of Invacare Corporation.
© 2002 Invacare Corporation
Rev D (1) 2/02 Part No. 0900113