



neonatal
solutions

Directions for Use

Enteral Extension Set

ENFit®

R_x Only

Not made with natural rubber latex.

Not made with Diethylhexylphthalate (DEHP).

WARNING:

- The ENFit Enteral Extension Set is not intended for intravascular or intravenous use.
- This device has the potential to misconnect with small bore connectors of other healthcare applications.

CAUTIONS:

- Contents are sterile in an unopened, undamaged package. Do not use this device if opened or damaged.
- Do not re-sterilize.
- Do not remove the hub from the enteral extension set.
- The patient's age, condition, and ability to feed orally will determine the method of feeding.

INDICATIONS FOR USE:

- The ENFit Enteral Extension Set is intended to be attached to a nasogastric feeding tube and/or syringe for introduction of enteral fluids and/or oral medication. The ENFit Enteral Extension Set is disposable and single use only.

RECOMMENDED PROCEDURE:

- Use clean technique.
- Ensure this device is only connected to an enteral port and not to an I.V. set.
- Attach ENFit female connector to feeding tube.
- Attach ENFit male connector to syringe.

ENTERAL SET REMOVAL:

- Remove the enteral accessory and discard per hospital protocol.

 Medela LLC
1101 Corporate Drive,
McHenry IL 60050

Medela wordmark and logo are registered in the U.S. Patent and Trademark Office. ENFit® is a federally registered trademark of GEDSA and is used with permission.

1548587 A 0617 © 2017 Medela DFU-042