PersonalFit™ PLUS  
Breast Shield | Embudo | Téterelle

- EN Instructions for use
- ES Instrucciones de uso
- FR Mode d'emploi

Medela LLC  
1101 Corporate Drive  
McHenry, IL 60050  
(800) 435-8316  
customer.service@medela.com  
medela.us  
Medela Canada, Inc.  
4160 Sladeview Crescent, #8  
Mississauga, ON, L5L 0A1  
(800) 435-8316  
customer.service@medela.ca  
www.medela.ca

medela.us | medela.ca

YouTube Instagram Facebook

\*All parts that touch breast milk are made without BPA, PVC, natural rubber latex, or DEHP.  
\*Todos los partes que tocan la leche materna están fabricados sin BPA, PVC, natural rubber latex, or DEHP.  
\*Toutes les pièces qui entrent en contact avec le lait maternel ne contiennent pas de BPA, PVC, natural rubber latex, or DEHP.

Medela wordmark and logo is registered in the U.S. Patent and Trademark Office.  
Personnifit and Quick Clean are trademarks of Medela.  
La marca denominativa Medela y el logotipo están registrados en la Oficina de Patentes y Marcas de los EE. UU.  
Personnifit y Quick Clean son marcas comerciales de Medela.  
Le mot-symbole et le logo Medela sont enregistrés au Bureau américain des brevets et des marques de commerce.  
Personnifit et Quick Clean sont des marques de Medela.

15400246 A 0124 © 2024 Medela 101046459

EN Cleaning Guidelines

Wash Before First Use & After Each Use

1. **Separate parts for washing.**
2. **Rinse separated parts in cool water in order to remove breast milk residue.**
3. **Wash in a dedicated basin:**
  - Soak separated parts in warm, soapy water for 5 minutes.
  - Wash each part with a clean dish-cloth or soft brush.
  - Rinse separated parts with clear water.
- OR
- Wash in dishwasher:**
  - Wash all separated parts on top rack or in top rack cutlery section of dishwasher.
4. **Dry on a clean surface:**
  - Allow to air-dry on a clean, unused dish towel or paper towel.
  - Put the clean and dry parts in a clean storage bag or clean environment.
  - Do not store parts in an airtight container/bag if moist. It is important that all residual moisture dries.

Sanitize Before First Use & Once Per Day

1. **Separate parts for sanitizing.**
2. **Wash hands thoroughly.**
3. **Boil separated parts:**
  - Fill a pot with enough water to cover all parts.
  - Bring water to a boil.
  - Place parts in boiling water for 10 minutes.
4. **Allow water to cool and gently remove parts from water with tongs.**
5. **Dry on a clean surface:**
  - Allow to air-dry on a clean, unused dish towel or paper towel.
  - Put the clean and dry parts in a clean storage bag or clean environment.
  - Do not store parts in an airtight container/bag if moist. It is important that all residual moisture dries.

As an alternative, use the Medela Quick Clean™ Microsteam™ bag by following the instructions on the bag (sold separately).


EN Determine Your Fit

1. Understanding Medela's breast shield sizing

Pumping should **NOT** hurt. For maximum comfort and pumping efficiency, Medela offers multiple breast shield options and sizes.

Visit [medela.com/fittingguide](https://www.medela.com/fittingguide) to view the full guide to help you determine your optimal size based on your nipple diameter.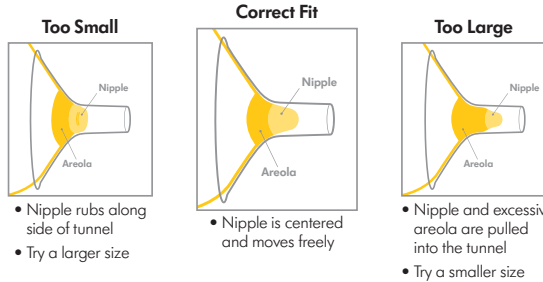

2. Test your breast shield size

1. Start with the breast shield that came with your pump, or the size determined by measuring.
2. Center nipple and gently hold breast shield against your breast.
3. Adjust for Maximum Comfort Vacuum to achieve optimum suction level.
4. Refer to images while pumping in the expression phase.

EN Warnings, Cautions & Notes

- WARNING**

  - PersonalFit™ Breast shields are a personal care item intended for use by a single user and should not be shared between users.
- CAUTION**

  - Only use drinking-quality tap or bottled water for cleaning.
  - Separate and wash all parts that come in contact with breast milk immediately after use to avoid dry up of breast milk residue and prevent growth of bacteria.
  - If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue with the breast shield tunnel, use of a lubricant such as Lanolin may be beneficial.

- NOTE**

  - If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommend that you boil parts with distilled water to prevent substantial mineral build up over time which may compromise parts.
  - If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.
  - Store in a clean non-airtight bag/container until next use.

EN Meaning of Symbols

-  Manufacture (ISO 15223-1\*, 5.1.1).
-  Catalogue number (ISO 15223-1\*, 5.1.6).
-  This symbol indicates a fragile device that should be handled with care (ISO 15223-1\*, 5.3.1).
-  Do keep the device away from sunlight (ISO 15223-1\*, 5.3.2).
-  This symbol indicates that the instructions for use should be followed (IEC 60601-1, Table D.2, Symbol 10).
-  This symbol indicates that the device should be kept dry (ISO 15223-1\*, 5.3.4).
-  This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.
-  This symbol indicates that the packaging is capable of being recycled (ISO 7000-1135)
-  Date of manufacture (ISO 15223-1\*, 5.1.3)
-  Batch code (ISO 15223-1\*, 5.1.5).

\*Medical devices - Symbols to be used with information to be supplied by the manufacturer -Part 1: General requirements

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