Step 1: Chose your program

Use **INITIATE** immediately after birth:

- Within the first 5 days, **OR**;
- Until you've pumped at least 20 mL in each of your last 3 sessions

Use **MAINTAIN** after milk comes in:

- Once you've pumped at least 20 mL in each of your last 3 sessions, **OR**;
- Beginning day 6, whichever comes first

Step 2: Select program and get started

To select the **INITIATE** program, press  while the screen displays:

After selecting the **INITIATE** program, the screen will display:

The **INITIATE** program alternates between the stimulation and expression phase, with several pauses and turns off automatically when the session is completed. Complete the entire program.

To select the **MAINTAIN** program, **wait 5-10 seconds**, while the screen displays:

The **MAINTAIN** program will start automatically. The screen will then display:

While pumping in the **MAINTAIN** program, the stimulation phase will last 2 minutes. If your milk starts flowing before the expression phase begins, press  to begin the expression phase.

Scan QR code for helpful videos

To achieve Maximum Comfort Vacuum™, adjust knob to optimum suction level.

Medela AG
Lättichstrasse 4b
6340 Baar, Switzerland

Distributed by:
Medela LLC
1101 Corporate Drive
McHenry, IL 60050
USA
+1 (800) 435-8316 or +1 (815) 363-1166
customer.service@medela.com
www.medela.us

Medela Canada, Inc.
4160 Sladeview Crescent, #8
Mississauga, ON, L5L 0A1
Canada
+1 (800) 435-8316 or +1 (905) 608-7272
info@medela.ca
www.medela.ca

The Quick Card does not replace the Symphony® Instructions for Use. Refer to Symphony® Instructions for Use for more detailed information.

Medela wordmark and logo and Symphony PLUS are registered in the U.S. Patent and Trademark Office. Science of Care and Maximum Comfort Vacuum are trademarks of Medela.

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© 2024 Medela 101046889/2024-06/A