

Waterless Milk Warmer™ Quick Start Instructions

Review instruction manual for important safeguards prior to first use.

Set-up




Expand liner prior to installing in warmer.




Bend insert at hinge and place into position.




Evenly spread liner within warmer cavity.

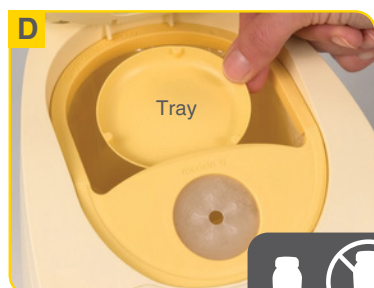

Push tray, flat-side down, into bottom of warmer.

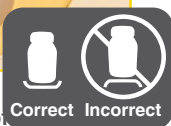


Load feed.
NOTE: Always use insert tray to warm container or syringe.

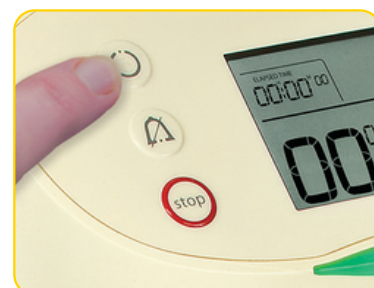


Press power button to turn on the device.

Program


Select Mode

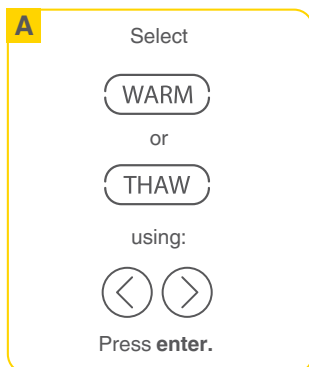


Select Starting Temperature

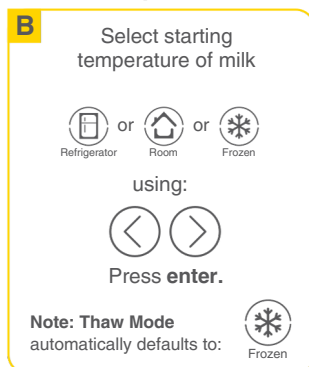

Select Volume

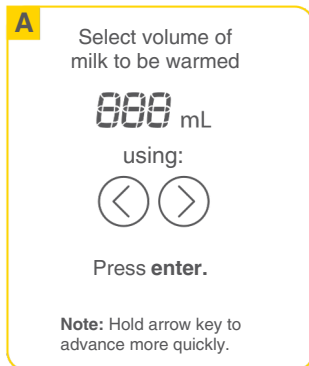

Confirm & Begin

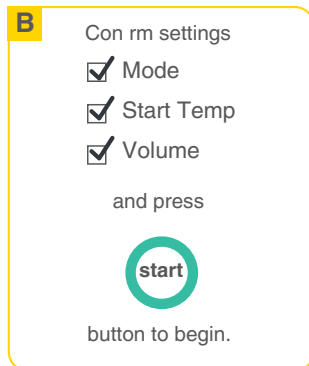

Ready

Flashing green light indicates milk is ready.



Remove

- Thaw complete.
- Remove milk before ice melts.

Note: Proceed to mode when ready to prepare for feeding.

Hold

- Feed will stay warm for up to 30 minutes.
- Remove container or syringe when ready to feed.

Note: Lift the lid to pause or stop the warmer.

Feed

TIP: If warming or thawing milk in the Colostrum Protector, refer to the How to Warm and Thaw using Colostrum Protector Section in the Instructions for use document.