Article #6100018S (US)
Produit #20018 (Canada)
Artículo #6100018S (EE.UU.)

medela 

medela 

www.medela.com

Manufactured by Medela, Inc.
P.O. Box 660, McHenry, IL 60051-0660
1101 Corporate Dr., McHenry, IL 60050
Phone: (800) 435-8316 or (815) 363-1166
Fax: (815) 363-1246
Email: customer.service@medela.com

Medela Canada, Inc.
4090B Sladeview Crescent, Unit 2
Mississauga, Ontario L5L 5Y5
Phone: (905) 608-7272 or (800) 435-8316
Fax: (905) 608-8720
Email: info@medela.ca

SoftFeeder™

- ✓ Instructions for use
- ✓ Mode d'emploi
- ✓ Instrucciones de uso

Made in the USA and Switzerland/Fabrique aux Etats-Unis et Suisse/Hecho en EE.UU y Suiza.
Softfeeder is a trademark of Medela and Medela is a registered trademark of Medela, Inc.
Medela est une marque déposée et Softfeeder est une marque de commerce de Medela, Inc.
Softfeeder es una marca comercial y Medela es una marca registrada de Medela Inc.

Printed in the U.S.A./Imprimé aux Etats-Unis/Imprimado en los EE.UU.

SoftFeeder™

Medela supports breastfeeding as the best method of nutrition for infants. When a healthcare professional determines that the baby cannot currently be fed at the breast, the Medela SoftFeeder is an ideal alternative. The Medela SoftFeeder allows the baby to be fed without developing nipple confusion/preference until the transition to breastfeeding can be made. If the use of an electric breastpump to express milk is indicated, call 1-800-TELL YOU or go to www.medela.com for the name and phone number of a Medela Rental Station or for the name of a Breastfeeding Specialist.


Components

The SoftFeeder includes:
1-80 mL Bottle, 1-Silicone reservoir, 1-Disk, 1-Valve membrane and 1-Collar.

Assembly


1. Press valve/membrane onto the upper side of the disk so that the stud goes completely through the center hole.
2. Fill bottle half full, or with amount baby is presently taking, with the breastmilk recommended by your physician. Warm breastmilk to room temperature before pouring it into the bottle. The feeder is designed for use with room temperature breastmilk and should not be used with hot liquids.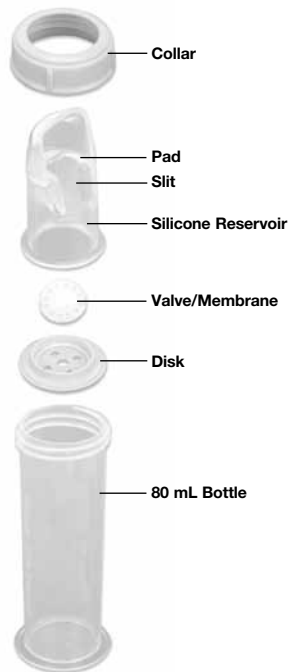

3. Slip the silicone reservoir into the collar.
4. Put assembled valve into the reservoir, making sure that the valve and high rim of the disk are facing the inside of the reservoir.
5. Place the reservoir assembly over the bottle and screw all the parts together using the collar. Tighten the collar securely to ensure a good seal.

Filling The Reservoir

1. Hold the feeder upright and remove some air from the reservoir by squeezing below the pads.
2. Keep squeezing and tip the feeder upside down.
3. Release the pads and some breastmilk will enter the reservoir.
4. Repeat filling steps 1-3 until the reservoir is almost filled, approximately up to the bottom of the pads. As baby is fed, the reservoir will automatically refill.

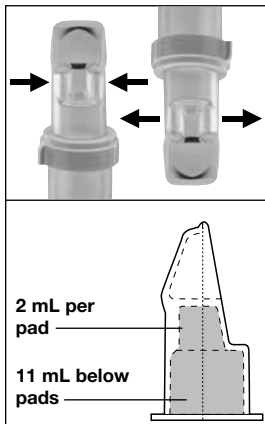Feeding Baby

1. Hold feeder horizontally, with a slight angle upward on the bottom of the bottle. Positioning is very similar to holding a pencil. Angle the feeder so the air that remains rises to the end of the bottle. Angling the bottle in this manner is essential to the automatic refill feature of the reservoir.
2. Begin the flow of breastmilk by gently squeezing the two pads on either side of the reservoir. This will pool the breastmilk in the feeding tip of the reservoir.
3. Use the tip of the reservoir to tickle baby's mouth open. Lay the rim of the reservoir on baby's lower lip.
4. Tip the feeder so that the breastmilk pooling in the reservoir is fed to baby. Or, allow the baby to lap up breastmilk with his tongue.

Cleaning & Sanitizing the Feeder

If there is some breastmilk left in the feeder, hold it nearly horizontal over a sink before unscrewing the collar. The disk should remain in the reservoir, to prevent breastmilk from spilling.

After Each Feeding:

Take the feeder apart, making sure that the valve is separated from the disk. Flush cold water through the slit in the pads of the reservoir to remove as much breastmilk residue as possible. Wash all parts in lukewarm, soapy water. Rinse thoroughly with cold water. Do not turn the reservoir inside out or push anything through the slit in the reservoir.

Each Day:

The feeder should be sanitized by boiling, preferably in distilled water, for 10 minutes. (Unless you are instructed differently by your hospital, physician or other medical professional). If hard water deposits occur, use distilled water or occasionally boil in a mixture of 3 parts water to 1 part vinegar. All disassembled parts can be autoclaved to a maximum of 272 °F for 3 minutes at 29 PSIG or 250 °F for 15 minutes at 15 PSIG. To avoid damage during sterilization, sterilize parts without applying external pressure on parts.

Helpful Hints

Reservoir Not Pooling Properly

Use stronger finger pressure to help the flow of breastmilk.

Reservoir Not Refilling Automatically

Check that the air groove in the disk has not become blocked if thickening additives have been added to the breastmilk.

Angle the feeder so the air that remains in the bottle rises to the end. Angling the bottle in this manner is essential to the automatic refill feature of the reservoir. Air may enter the reservoir through the slit causing reservoir to empty. Simply follow “Filling The Reservoir” instructions to resume feeding.

Feeder Leaking

If the feeder leaks, check that the feeder has been correctly assembled. The stud of the valve must be properly inserted through the hole in the disk. The collar must be firmly screwed onto the bottle. Make sure the breastmilk is not too hot. The feeder is designed for use with room temperature breastmilk and should not be used with hot liquids.

Check to see if either the valve membrane or reservoir is visibly damaged and needs replacing.



Bigger Bottles

The Medela SoftFeeder is designed to work with the Medela 80 mL BPA-free bottle. Other bottles may be used, but you may experience some slight leakage around the collar.


Troubleshooting

If the SoftFeeder is not releasing fluid

1. If the SoftFeeder is not releasing fluid, check the slit located on one side of the pad. If the slit becomes sealed, it needs to be manually opened.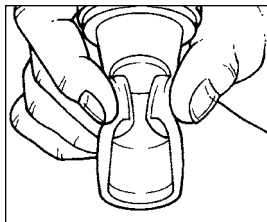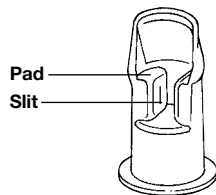

2. To break the seal, try squeezing the pads tightly, both toward the upper and lower parts of the pads.

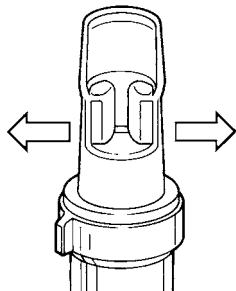3. Or, use both thumbs to pull the pads outward, as if you were trying to turn the lip inside out.

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