Homedics

women's health

vibration massage belt + soothing heat

3 YEAR GUARANTEE

WMH-200H

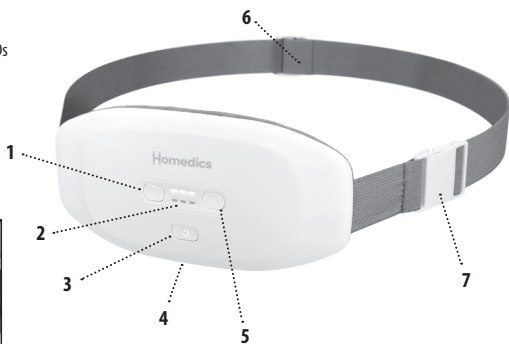
PRODUCT FEATURES:

1. Vibration level control
2. Vibration massage and heat level indicator LEDs
3. Power button
4. USB-C charge port
5. Heat level control
6. Adjustable belt
7. Belt clip

fig.1

fig.2

INSTRUCTIONS FOR USE:

1. Be sure the unit is fully charged before its first use.
2. This unit comes with a USB-C type cord. Do not attempt to use a cord that is not a USB-C type cord.
3. DO NOT use this device while it is charging.
4. When the unit is plugged in, the power button will be flashing in either green, yellow, or red. When you see a solid green light illuminated/the light shuts off completely, the device is fully charged.
5. Once unplugged, you can check the battery level on the device by quickly pushing and releasing the power button. If the device is fully charged, you will see the LED light on the power button illuminate in green.

Battery Level LED Indicator

99%-66%: green
65%-33%: yellow
32%-10%: red
<10%: flashing red

6. The power button will stay on while the device is on. The colour will change to the respective level/colour noted above while the device is in use.
7. To turn on the device, hold the power button until you see the red LED light illuminate on the bottom row. The lowest

- heat level will turn on with 1 red LED light. (fig. 1).
8. To adjust the heat levels, press the heat button. There are 3 heat levels: low, medium, and high. You can completely turn the heat off by pushing the heat button 3 times until you don't see any red LED lights on the bottom row.
9. To change the vibration massage levels, press the vibration button to turn it on. It will start at the lowest vibration setting with 1 white LED light showing on the top row. Push it a second time for medium intensity, and push it a third time for high intensity. To turn off the vibration massage, push it a fourth time and the vibration will stop.
10. Attach and centre the elastic belt around abdomen or other area (estimated belt size: 41cm-163cm) (fig. 2).
11. To turn off the device, push and hold the power button until the unit has turned off and no LED lights are illuminated.

NOTE: The heat and vibration massage can be used together or independently.

NOTE: This device is equipped with a 30-minute auto-off feature for your safety.

NOTE: A full charge will take up to 3.5 hours.

Cleaning your device.

- Make sure the device is unplugged from any power supply and allow to cool before cleaning. Clean only with a soft, slightly damp cloth.
- Never allow water or any other liquid to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass/furniture polish, paint thinner, etc to clean.

READ ALL INSTRUCTIONS BEFORE USE. SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

.....

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Do not place or store appliance where it can fall or be pulled into a bath or sink. Do not place in or drop into water or other liquid.
- DO NOT reach for an appliance that has fallen in water or other liquids.

Keep dry – DO NOT operate in wet or moist conditions.

- NEVER insert pins, metallic fasteners or objects into the appliance or any opening.
- Use this appliance for the intended use as described in this booklet. DO NOT use attachments not recommended by HoMedics.
- NEVER operate the appliance if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to the HoMedics Service Centre for examination and repair.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT attempt to repair the appliance. There are no user serviceable parts. All servicing of this appliance must be performed at an authorised HoMedics Service Centre.
- Never block the air openings of the

- appliance. Keep air openings free of lint, hair etc.
- Please ensure that all hair, clothing and jewellery are kept clear of moving parts of the product at all times.
 - If you have any concerns regarding your health, consult a doctor before using this appliance.
 - Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your GP.
 - Pregnant women, diabetics and individuals with pacemakers should consult a doctor before using this appliance. Not recommended for use by individuals with sensory deficiencies including diabetic neuropathy.
 - DO NOT use on an infant, invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation.
 - This appliance should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
 - Do not use for longer than the recommended time.
 - Only gentle force should be exerted against the mechanism in order to eliminate risk of injury.
 - Failure to follow the above may result in the risk of fire or injury.
 - Batteries (battery pack or batteries installed) shall not be exposed to excessive heat such as sunshine, fire or the like. Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type.
 - Keep small cells and batteries which are considered swallowable out of the reach of children.
 - Swallowing may lead to burns, perforation of soft tissue and death. Severe burns can occur within 2 hours of ingestion.
 - In case of ingestion of a cell or battery, seek medical assistance promptly.
 - The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

3 YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 3 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU.

A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase).

Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre.

Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights.

For your local HoMedics Service Centre, go to www.homedics.co.uk/servicecentres

Battery replacement

Your product includes a rechargeable battery designed to last the lifetime of the product. In the unlikely event that you should require a replacement battery, please contact Customer Services, who will supply details of the warranty and out-of-warranty replacement battery service.

Battery directive

 This symbol indicates that batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please dispose of batteries in designated collection points.

WEEE explanation

 This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland