EN ASSEMBLY AND USE

SYMPHONY PUMP KIT

- Carefully push the white membrane onto the yellow valve head until it engages and is laying flat.
- Push yellow valve head onto inner ring of connector bottom.
- Screw connector onto the bottle/container.
- Insert the tubing into the corresponding opening in the connector.
- Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.
- Insert the tubing adapter into the opening in the membrane cap.
- Open the lid of the breast pump. Press the oval button on the top of the Symphony® and lift the handle.
- Position the membrane cap firmly on the breast pump membrane to form a seal. Center the tubing connector in the recess in the pump housing.
- Make sure that the membrane cap engages and lies flush to the surface.
- Center the tubing connector in the recess in the pump housing.
- Make sure that the membrane cap engages and lies flush to the surface.
- Close the lid.
- Center breast shields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow.)

Note: If single pumping, only seat one membrane cap.

Refer to Symphony® breast pump Instructions for Use for pump operation information.

HARMONY KIT

- Carefully push the white membrane onto the yellow valve head until it engages and is laying flat.
- Push yellow valve head onto inner ring of connector bottom.
- Screw connector onto the bottle/container.
- If the handle components have been disassembled, place the oval shaped base of the stem into the diaphragm.
- Make sure the oval shape is lined up correctly in the diaphragm.
- Place silicone diaphragm/stem into pump handle and press down firmly around entire edge to ensure proper seal. The lines on the diaphragm should line up with the yellow dots on the handle.
- Insert stem of handle assembly into opening on top of pump body and push until you feel it snap in.
- Insert breast shield into top opening on pump body.
- Center breast shield over nipple. Caution: pushing shield too firmly into breast may actually reduce milk flow.
- The adjustable handle can be positioned for your maximum pumping comfort. The level of vacuum is controlled by the compression and the speed of the handle stroke. You decide the pumping rhythm that is best for you.
- Start pumping in the stimulation mode by pressing down on the milk drops symbol on top of the pump handle. Press and release quickly until you see milk flow begin. This mode mimics baby's initial rapid sucking and is designed to stimulate milk flow/let-down.
- Once let-down has begun, switch to the expression mode by pressing the bottom of the pump handle. This will give you a longer stroke that mimics baby's deeper, more relaxed sucking and is designed to maximize milk flow.

WASH BEFORE FIRST USE & AFTER EACH USE*

KIT PART CLEANING

Breast shield assemblies came assembled. Separate all parts that will come into contact with the breast and breast milk before cleaning.

- Separate parts for washing.
- Rinse all separated parts that came in contact with breast and breast milk in cool water in order to remove breast milk residue.

Wash in a Dedicated Wash Basin

- Once rinsed, parts that have touched the breast or breast milk should be placed in a clean wash basin, dedicated to only cleaning these items. Do not place parts directly in the sink.
- Soak all separated parts in warm, soapy water for 5 minutes.
- Wash each part with a clean dishcloth or soft brush.
- Rinse all separated parts with clear water.
- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Alternative, wash in dishwasher

- Wash all separated parts on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts in a clean, cool place when not in use.

Note: If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.

Note: Store the pump kit in a clean bag/container until next use.

Note: Only wash tubing if milk is present or tubing is soiled. You may also wash your kit parts by following the instructions on the Quick Clean™ BreastMilk Removal Soap bottle (sold separately).

HARMONY® HANDLE ASSEMBLY CLEANING

Care for your Harmony handle assembly after every few manual pumping sessions or when soiled.

- Wash the Harmony handle assembly, which includes the Harmony handle, diaphragm stem, and o-ring with warm soapy water.
- Pay special attention to the o-ring during assembly and cleaning to avoid losing or damaging.
- Do not use Quick Clean™ Micro-Steam™ bag to sanitize the Harmony handle assembly.

SANITIZE BEFORE FIRST USE AND ONCE PER DAY*

Medela pump kit parts can be sanitized by boiling or using the Medela Quick Clean™ Micro-Steam™ bag (sold separately).

- Separate all parts that come in contact with breast milk.
- Wash hands thoroughly.
- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.
- Allow water to cool and gently remove parts from water with tongs.
- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note: Store the pump kit in a clean bag/container until next use.

Note: If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommended that you boil parts with distilled water to prevent substantial mineral build-up over time which may compromise your parts.

If sanitizing your pump kit parts with the Quick Clean Micro-Steam bag, separate all parts that come in contact with your breast or breast milk and follow the instructions on the Quick Clean Micro-Steam bag.

Not for use with the Harmony handle assembly.

TUBING CARE

Inspect tubing after each pumping session.

Tubing should be washed if milk is present or tubing is soiled.

- If tubing becomes moldy, discontinue use and replace tubing. Replacement parts can be purchased at www.ShopMedela.com.

- Remove tubing from breast shield.
- Rinse tubing in cool water to remove breast milk.
- Once rinsed, place in a clean wash basin, dedicated to only cleaning breastpump parts. Do not place parts directly in the sink.
- Soak tubing in warm soapy water for 5 minutes.
- Rinse tubing with clear water.
- Hang to air dry.
- Or shake out any drops of water from the tubing and let the Symphony breast pump run with tubing attached for 1-2 minutes or until dry.

Note: If there is condensation in the tubing: Shake water droplets out of the tubing and hang it up to dry or let the Symphony breast pump run with tubing attached for 1–2 minutes or until dry.

WARNING:

- Close supervision is necessary when this product is used near children.
- Use the product only for its intended use as described in this manual.
- Never drop or insert any object into any opening or tubing.
- Use Medela original accessories only.
- Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.
- Inspect the breast pump kit components prior to each use.
 - Do not use the components if they are damaged.
 - Do not use the components if dirt, mold or other contamination is visible.
- Inspect all appropriate pump components before each use.
- Clean and sanitize all parts that come in contact with your breast

and breast milk prior to first use. Sterile kits are ready to use and do not require washing prior to first use.

- Do not modify any breast pump kit components
- Wash all parts that come in contact with your breast and breast milk after every use.
- If problems or pain occur, consult your lactation consultant or physician.

CAUTION:

- Wash hands thoroughly with soap and water before touching breast pump, kit and breasts. Avoid touching the inside of containers or lids.
- Pump only with the breast pump in an upright position.
- Separate and wash all parts that come in contact with breast and breast milk immediately after use to avoid dry up of breast milk residue and prevent growth of bacteria.
- Do NOT shorten tubing between breast shield and pump.
- Do NOT use abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Do not store wet or damp parts as mold may develop.
- Inspect the tubing for condensation or milk after each use. Wash or replace the tubing immediately after use, if milk is observed in the tubing.
- If tubing becomes moldy, discontinue use and replace tubing.
- Do not use the breast pump kit with a non-compatible breast pump.
- Do not use the breast pump kit for any other purpose than expressing breast milk.
- Only use drinking-quality tap or bottled water for cleaning.
- While some discomfort may be felt when first using a breast pump, using a breast pump should not cause pain.

NOTE

- *Sterile kits are ready to use and do not require washing or sanitizing prior to first use.
- Wash breast pump kit parts after each use.
- Take care not to damage parts of the pump kit during cleaning.
- It is not necessary to wash Harmony® handle assembly, tubing, membrane cap or protective membrane prior to first use.
- It is not necessary to wash tubing, membrane cap or protective membrane prior to first use.
- Only wash tubing if milk is present or tubing is soiled.
- Store the pump kit in a clean bag/container until next use.
- Sanitize breast pump kit parts once daily.

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EN INTENDED USE
The Symphony® & Harmony® Breast Pump Kit is intended to use with Symphony & Harmony Breast pump by lactating women to express and collect milk from their breast.

PLEASE SAVE THESE INSTRUCTIONS.
Use Symphony® & Harmony® breast pump kit Instructions in association with Symphony & Harmony breast pump Instructions for Use

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

EN SYMBOL GLOSSARY	
	Manufacturer (ISO 15223-1, Clause 5.1.1)
	Date of Manufacture (ISO 15223-1, Clause 5.1.3)
	Use-by-date (ISO 15223-1, Clause 5.1.4)
	Batch code (ISO 15223-1, Clause 5.1.5)
	Catalogue number (ISO 15223-1, Clause 5.1.6)
	Sterilized using ethylene oxide (ISO 15223-1, Clause 5.2.3)
	Do not use if package is damaged and consult Instructions for Use (ISO 15223-1, Clause 5.2.8)
	Single sterile barrier system (ISO 15223-1, Clause 5.2.11)
	Do not re-use (ISO 15223-1, Clause 5.4.2)
	Consult Instructions for Use (ISO 15223-1, Clause 5.4.3)
	Caution (ISO 15223-1, Clause 5.4.4)
	General Warning Sign

See Medela website or contact Medela Customer Service at 1-800-435-8316 for Breast Shield Sizing, Breast milk storage guidance, Supplemental information and Warranty

WARRANTY
Limited 90-day warranty. For complete warranty information visit the product page at Medela.us or Medela.ca.

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- EN PARTS**
1. Breast Shields
 2. Connectors/Pump body
 3. Yellow Valves
 4. White Membranes
 5. Breast milk bottle/collection containers (bottle size may vary based on breast pump kit)
 6. Harmony handle
 7. Lids (not shown)
 8. Tubing and membrane caps (not shown)

Kit components may vary from pictures in this instruction manual.

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<https://www.medela.us/symphony-breast-milk-initiation-support>

EN Instructions
Symphony® and Harmony® Breast Pump Kit
► For use with Symphony & Harmony breast pumps
► **Symphony: for double or single pumping**
► **Harmony: for single pumping**

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