Homedics

**hot & cold
reusable gel sleeve**

3 YEAR GUARANTEE

SR-GLB800

PRODUCT FEATURES:

- One size fits MOST
- These items are non-electrical

INSTRUCTIONS FOR USE:

- Zipper bag is included to protect the item. Using the bag will prolong the life of the item so it can be used more. Store item inside the bag when not in use.
- Place the item inside the bag and close the zip before placing in the freezer for up to 2 hours – this will provide up to 30 minutes cold relief. Gel does not freeze – so the items remain flexible and wearable.
- Remove the sleeve from the zipper bag and place in the microwave for 30 seconds and test for comfort. If it is too hot, allow it to cool to desired temperature before use. If it is not warm enough, reheat at 10 second intervals until a comfortable temperature is reached. If the heat feels uneven, flip it over and reheat for an additional 10 seconds or until the desired temperature is achieved. All microwaves may vary.
- **IMPORTANT:** When fixing the wrap to the body, the Velcro should be attached directly to the elastic band. Do not attach Velcro to the fabric as this could damage the fabric.

Care instructions:

- Use damp cloth to wipe down. Lay flat and air dry.
 - ☒ Do not wash
 - ☒ Do not bleach
 - ☒ Do not tumble dry
 - ☒ Do not iron
 - ☒ Do not dry clean

READ ALL INSTRUCTIONS BEFORE USE. SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

- This item can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the item in a safe way and understand the hazards involved. Children shall not play with the item. Cleaning and user maintenance shall not be made by children without supervision.
- Do not place or store item where it can fall or be pulled into a bath or sink. Do not place in or drop into water or other liquid.
- NEVER use directly on swollen or inflamed areas or skin eruptions.
- Do not use on irritated, sunburned or chapped skin, or on any area with dermatological problems.
- Do not apply over warts, moles or varicose veins.
- This item should NEVER be used by any individual suffering from any physical or mental condition that would limit the user's capacity to use the item in a safe manner.
- DO NOT use on an infant, invalid or on a sleeping or unconscious person.

- DO NOT use on insensitive skin or on a person with poor blood circulation.
- Pregnant women and diabetics should consult a doctor before using this item. Not recommended for use by individuals with sensory deficiencies including diabetic neuropathy.
- Use of these items should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your GP.
- This is a non-professional item, designed for personal use and intended to provide a soothing experience. DO NOT use as a substitute for medical advice.
- Persons insensitive to heat must be careful when using the product.

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3 YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 3 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase). Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre. Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights. For your local HoMedics Service Centre, go to www.homedics.co.uk/servicecentres

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FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
FKA Brands Ltd, 29 Earlsfort Terrace, Dublin D02AY28, Ireland