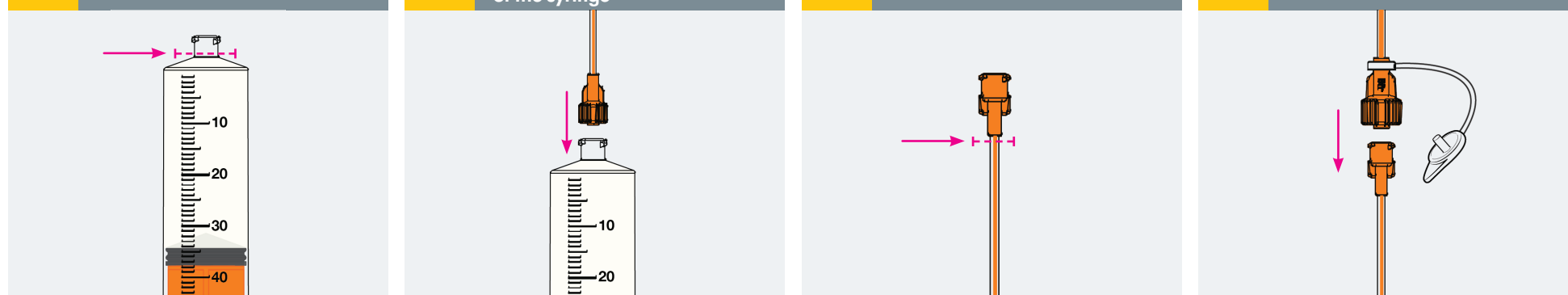


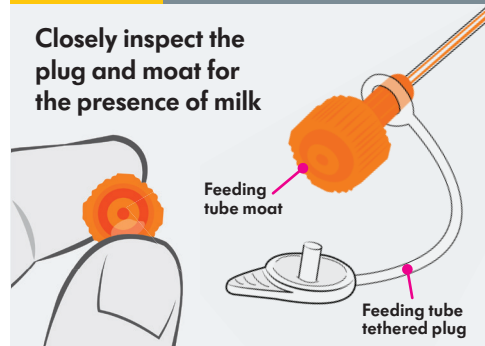
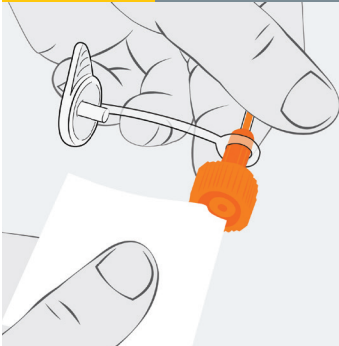


ENFit® Best Practices for Clean Connections

1	Prime to the bottom of the syringe tip	2	Bring the moat tip end of the extension set over the top of the syringe	3	Prime extension set just before orange hub on patient end	4	Bring feeding tube over the top of the extension set
							

How to prevent milk from accumulating in the feeding tube moat*

Inspect feeding tube hub before each connection and after each disconnection.

	INSPECT		CLEAN
<p>Closely inspect the plug and moat for the presence of milk</p> 		<ul style="list-style-type: none"> • Clean feeding tube plug and/or moat per protocol and when milk is visible • Saturate gauze with sterile water and use friction to remove any accumulated milk 	

Scan to learn more and view the product in-service video.

