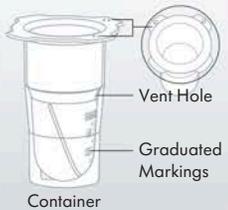


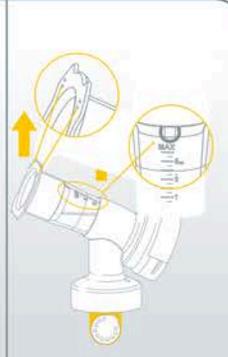
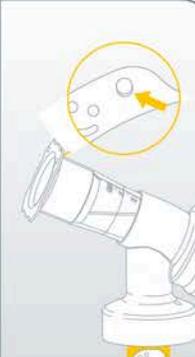
## Colostrum Protector Quick Start Guide

Step by step guide on how to assemble and use the Colostrum Protector

### How to assemble your Colostrum Protector

Parts & Features	1	2	3	4
<p><b>Lid</b></p>  <p>Smiley Faces</p>  <p>Vent Hole</p> <p>Graduated Markings</p> <p>Container</p>	<p>Attach the collector to Symphony® breast pump kit connector.</p> 	<p>Ensure correct orientation* to allow for venting and prevent colostrum from spilling from the vent hole into the container below.</p> 	<p>Attach the breast shield to the top of the Colostrum Protector.</p> 	<p>Assemble the breast pump kit.</p> 

### \*Correct Orientation Tips:

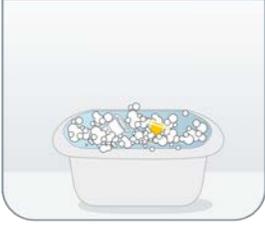
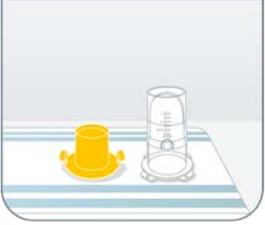
<p>Ensure the side with the smiley faces, vent hole, and graduated markings are facing up ↑.</p> <p><b>NOTE:</b></p> <p>Theside with nothing on it should be facing down ↓.</p> 	<p>Low visibility – feel for the bumps on the back of the smiley faces.</p> 	<p><b>Correct Assembly</b></p> 	<p><b>Incorrect Assembly</b></p> 
---	---	---	--

### How to disassemble your Colostrum Protector

<p>1</p> <p>Remove the Colostrum Protector from the connector, leaving the breast shield attached.</p> 	<p>2</p> <p>Place on a flat surface.</p> 	<p>3</p> <p>Remove the breast shield.</p> 	<p>4</p> <p>Secure the lid.</p> 	<p><b>More information</b></p> <p>Scan the QR code for further information about your Colostrum Protector</p> 
--	--	---	---	---

## Wash after each use\*

Wash in a dedicated wash basin

<p>1</p> <p>Rinse with clear water.</p>	<p>2</p> <p>In a dedicated wash basin for breast pump parts, soak in warm, soapy water.</p> <ul style="list-style-type: none"><li>• Do not place directly in the sink.</li></ul>	<p>3</p> <p>Wash with a clean dishcloth or soft brush.</p>	<p>4</p> <p>Rinse with clear water.</p>	<p>5</p> <p>Place on a clean surface and/or towel and allow to air dry.</p>
				
<p>6</p> <p>Store in a clean, cool place when not in use. Do NOT store wet or damp parts.</p>	<p>7</p> <p>Discard or recycle after 24-hours of use according to local regulations or hospital protocol.*</p>			
				

\*The Colostrum Protector can be reused for multiple pumping sessions for up to 24 hours or stored in the Freezer for up to 12 weeks. Follow your local regulations protocol.

**Content appearance may vary. Refer to product instructions for use for complete instructions.**

Medela wordmark and logo and Symphony are registered in the U.S. Patent and Trademark Office and the Canadian Intellectual Property Office. Science of Care is a trademark of Medela.  
15400097 C 0825© 2025 Medela