

# Colostrum Protector

Every Drop Counts™



Colostrum is a powerful therapy for the NICU infant and can lead to improved patient outcomes.<sup>1</sup> Every drop of colostrum is valuable and should be protected from loss. The Colostrum Protector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance. It catches all expressed drops of colostrum without getting lost in the breast pump parts, making it more effective than most traditional methods.

## Inspired by insights from nurses

- **Simple and easy to use:** The Colostrum Protector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance.
- **Minimizes loss of colostrum:** The Colostrum Protector minimizes loss of colostrum by catching it prior to reaching the pump parts; ensuring baby can get every drop.
- **Effective:** 84% of nurses find the Colostrum Protector to be more effective at collecting colostrum than pumping without it, allowing you to collect colostrum with confidence.<sup>2</sup>
- **Easy to transport:** The Colostrum Protector's snap on lid makes it easier to transport and protects from contamination, allowing you to focus on patient care.
- **Increases mom's sense of accomplishment:** The Colostrum Protector is perfect for holding the small amounts of colostrum produced in those first few days. 78% of nurses felt mom had a greater sense of accomplishment using the Colostrum Protector.<sup>2</sup>

## How does the Colostrum Protector work?\*

- 1 Colostrum Protector is attached to the Symphony® kit, between the breast shield and the connector.
- 2 Mom uses the Symphony breast pump to initiate breast milk production.
- 3 Colostrum is collected in the Colostrum Protector before reaching the rest of the pump parts, minimizing loss.  
**Tip:** In the event mom comes to volume sooner than expected and milk volumes exceed the 5 mL capacity; it will overflow into the container below.
- 4 After pumping, remove the Colostrum Protector from the Symphony kit keeping the breast shield on and set on a flat surface. Remove the breast shield and snap the lid onto the Colostrum Protector.
- 5 Can be frozen for up to 12 weeks.



## Product Specifications:

Product Name	Item Number	Description	Unit (Each Information)		Shipper Information		
			Dimensions (H x W x D Inches)	Weight (lbs)	Qty/Case	Dimensions (H x W x D Inches)	Weight (lbs)
Colostrum Protector 2-Pack	101046344	Includes 2 Colostrum Protectors and lids	2 x 5.5 x 4	0.06	40 packs of 2	16 x 10.875 x 9.25	2.4

Content appearance may vary.

<sup>1</sup>Meier PP et al. J Pediatr. 2017;180:15-21. <sup>2</sup> Medela 2022 Colostrum Collector Evaluation Survey  
\*Refer to Instructions for Use for complete instructions.

Medela LLC, 1101 Corporate Drive, McHenry, IL 60050 | Phone: (800) 435-8316 or (815) 363-1166 [www.medela.us](http://www.medela.us)

Medela wordmark and logo, Symphony and Every Drop Counts are registered in the U.S. Patent and Trademark Office and the Canadian Intellectual Property Office. Science of Care Waterless Milk Warmer is a trademark of Medela. 1549948E 0124 © 2024