




Nursery Care Health Kit IH474

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

Wrapping Clutch Case

The wrapping clutch case lays flat when open and has two roomy compartments with clear-view front panels for easy location of stored items. Each compartment closes securely with a generous strip of hook and loop material. To close the case, press the hook and loop strips together to contain the items within the compartments; bring the two halves of the case together with the clear-view panels facing each other; wrap the top of the two compartments over so that D-rings are as shown in the image at right; and use the grosgrain ribbon and D-rings to secure the wrapping clutch case.

Use and Care: Spot clean. Wipe with a damp cloth.

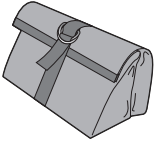

Bottle Medicine Dispenser

The Bottle Medicine Dispenser has easy-to-read dosage markings and measures up to 3 tsp. (15 ml).

To Use:

- 1 Remove top from measuring cup and fill cup to the prescribed dosage.
- 2 Snap top securely onto the measuring cup.
- 3 Bottle-feed child as normal, pausing as necessary so that child may swallow comfortably.

To Clean: Wash before first use and after every use. Disassemble top ring, nipple and measuring cup. Wash with warm, soapy water and rinse thoroughly. Alternatively, it is dishwasher safe. TOP RACK ONLY.



CAUTION: Inspect bottle nipple regularly for tears or cracks and discontinue use if any are noted. Made without BPA.

Easy Fill Medicine Syringe

Our Easy Fill Medicine Syringe is a convenient way to administer medicine to infants and toddlers. The curved, Soft Touch grip allows for comfortable finger positioning and easy dispensing.

- 1 Insert syringe tip into medicine bottle and draw plunger back slowly until medicine reaches the prescribed dosage marking.
- 2 Hold baby upright and then insert the tip of the dispenser into your child's mouth. Point tip toward child's cheek.
- 3 Push plunger slowly to help release liquid.

To Clean: Wash all parts separately. Wash with warm, soapy water and rinse thoroughly. Repeat after administering each dosage.

CAUTION: Keep all medications out of child's reach. We recommend that you use a child resistant top to seal the medicine bottle and store it in a locked cabinet or other location that is not accessible to a child. Made without BPA.

Nasal Aspirator

To Use:

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean: Aspirator is top-rack dishwasher safe or can be washed in warm soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

WARNING: NEVER push the aspirator too far into the nostril and always wash and dry thoroughly after each use. NEVER use in baby's mouth or ears. When storing, keep away from heat and bright light.



10 Digital Thermometer Covers

NOTE: Use thermometer covers when taking temperature rectally; not for oral use (see Warning section).

- 1 Hold edge of cover and slip over end of thermometer. Pull the outside protective cover off. Make sure no air remains between the probe and the cover.
- 2 **Rectal Use:** Lubricate the thermometer cover with a water-soluble lubricant or petroleum jelly. Follow instructions under "Rectal Use." Discard thermometer cover after use. Do not reuse covers.

NOTE: To purchase thermometer covers call Consumer Relations at (800) 544-1108, and ask for part number 49520.

Fingertip Toothbrush/Gum Massager

To Use:

- 1 Place on adult's fingertip.
- 2 Use the gum massager (nubby side) to help soothe baby's sensitive gums and the infant toothbrush side to clean emerging teeth and soothe tender gums.

Use and Care: Rinse thoroughly before first use and after each use.



CAUTION: Do not allow children to play with it. When not in use, keep out of reach of children. ADULT SUPERVISION REQUIRED.

Gentle Care Toddler Toothbrush

The Gentle Care Toddler Toothbrush can be placed on the table or sink and the convenient, roll-resistant handle keeps it from falling off.

To Use:

- 1 Begin use once all primary teeth are in or when recommended by your dentist.
- 2 Apply a pea-sized amount of toothpaste to the brush and have your child brush for 2-3 minutes using an up and down motion.
- 3 Continue to help and supervise your child during brushing - do not allow child to swallow toothpaste.

NOTE: Toothbrushes should be replaced every 2-3 months.

To Clean: Rinse each brush thoroughly before first use and after each use.

Important Note: Questions about your child's oral health should be answered by your family dentist.

CAUTION: Adult supervision required. Do not allow child to run or walk while carrying or using toothbrush. Do not let children play with toothbrushes.



12 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.



FOR EXTERNAL USE ONLY.

Emergency Information Card

Record important phone numbers and medical information for quick reference.

WARNING: These products are not toys. **DO NOT** allow children to play with them. When not in use, keep out of reach of children. **ALWAYS** supervise your child when using this kit. **NEVER** allow your child to use these products alone or without close adult supervision. We recommend that you store this kit in a locked cabinet or other location that is not accessible to a child. **FOR INDOOR HOME USE ONLY.**

3-in-1 Digital Thermometer

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.




TH093

Fahrenheit / Celsius

While thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This will take approximately 5 seconds.

How to Measure Temperature

Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

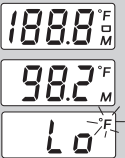

- Press and release ON/OFF button. All segments display briefly as the thermometer powers on.
- The last temperature measured displays with the **M** icon on screen for a few seconds. Then the thermometer will show an average temperature (37.0.9°C or 98.5.9°F) for 0.5 seconds before entering ready mode.
- **Lo** displays and °F flashes when the thermometer is ready to take measurement.

Specifics for each Temperature Mode

ORAL—For children 4 years and older.

1. Ensure that °F is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.



RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position baby so that the back and legs form a 90-degree angle (L-shape).
2. Ensure that °F is still flashing, then insert thermometer tip gently into the rectum, taking care not to insert past the insertion gauge (ring on the probe tip). Do not force the tip into rectum if resistance is encountered.



- When °F stops flashing, the measuring is complete. Approximate measurement times are: 60 seconds for oral and rectal temps, 90 seconds or longer for underarm temp.
 - Remove probe tip from measurement site. Read and record the temperature and time for reference.
 - Press button to shut off or thermometer will shut off automatically after 8-9 minutes.
- NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included. DO NOT allow child to walk or run during temperature taking. Probe could present a hazard.

Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or inconsequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

©2018 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. Safety 1st™ y el logotipo de Safety 1st y su diseño son marcas comerciales de Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com. Styles and colors may vary. Los estilos y los colores pueden variar. Made in CHINA. Hecho en CHINA. Distributed by (distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2855 Argenta Road, Unit 4, Mississauga, ON L5N 8G6. Importado y Distribuido en Chile por: Comercial e Industrial Silfa S.A. - San Ignacio 0201 / Parque Industrial Portezuelo Quilicura, Santiago Chile. Fono (562) 2339 9000. En Perú por: Comexa S.A. - Los Libertadores 455 - San Isidro, Lima - Perú. Fono (511) 4469014. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Via 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle, Benito Juárez C.P. 03100 México, Ciudad de México R.F.C. DJG140305GN9 Tel. (55) 67199202 10/25/18 4358-7679

Dorel

Technical Specifications

Measurement Range: 90° F to 109.9° F
Measurement Accuracy: +/- 0.2° F for temps 96° F to 107.6° F
+/- 0.4° F for temps below 96° F or above 107.6° F.
Operation Environment: 60.8° F to 104° F
Storage Environment: -13° F to 144° F
Test Protocol:
1 Put the tip of thermometer into water bath, approximately 2 cm below the surface.
2 Wait for 10 seconds, then turn on the thermometer by pressing the ON/OFF button. The thermometer is now in standard mode.
3 When the measurement is completed (approximately 60 seconds) record the temperature reading.

UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

1. Wipe underarm with a dry towel.
2. Ensure that °F is still flashing, then place probe tip at center of underarm. Bring arm down against the body. Probe must touch skin.



