# Invacare® Mattress Cleaning Instructions and Warranty

Invacare®, Invacare Essentials™, and Invacare Solace® Mattresses

en

**Therapeutic Support Surface**

User Manual

---

## General

---

### Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

**DANGER!**

-Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

**WARNING!**

-Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION!**

-Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

Gives useful tips, recommendations and information for efficient, trouble-free use.

### Symbols on the Labels

References to International Organization for Standardization (ISO) symbols are listed if applicable.

Symbol	Description	ISO Reference
	Indicates the user should refer to the product documentation.	ISO 7010:2019(E) Section M002
	Indicates the product Unique Device Identifier (UDI).	ISO 15223-1:2021 Section 5.7.10

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, this manual **MUST** be read and saved for future reference.

Yes, you can.

## Invacare® Mattress Cleaning Instructions and Warranty

Symbol	Description	ISO Reference
	Indicates the product reference number.	ISO 15223-1:2021 Section 5.1.6
	Indicates the manufacturer name and address.	ISO 15223-1:2021 Section 5.1.1
	Indicates the device manufacture date.	ISO 15223-1:2021 Section 5.1.3
	Indicates the foot end of the bed.	n/a

### Usage

1. Remove all packaging.
2. Allow the mattress to rest for a minimum of one (1) hour before use.  
 After removing the plastic bag, contents may take up to 24 hours to fully expand.
3. Place the mattress directly in the bed frame.

### Safety

#### CAUTION!

Risk of Injury

- Caretakers should assess patients for adverse reactions such as redness (erythema), swelling (edema), irritation, sensitization (delayed Type IV hypersensitivity), allergy, immune response, or other reactions on the skin where the device has contact.

### Cleaning

#### WARNING!

Risk of Injury or Damage

Mattresses **MUST** be cleaned regularly and after each patient use.

- Because of the potential risk of infectious exposure, cleaning with the patient on the bed is not recommended.
- All equipment should be inspected. Any item that is visibly soiled with the patient's blood or other body fluids should be properly cleaned or removed.
- Treat all soiled bedding as if it were contaminated with pathogenic microorganisms.
- All cleaning solutions must be properly diluted according to the manufacturers' instructions.
- Follow standard institutional wipe-down and infection-control procedures.

**WARNING!****Risk of Injury or Damage**

- DO NOT open the zippered mattress top cover during cleaning or infection-control procedures or during use.
- DO NOT autoclave mattress covers.

**CAUTION!****Risk of Injury or Damage**

To avoid fabric breakdown and loss of waterproofing, use only the following properly diluted standard cleaning solutions:

- Quaternary wipe down
- Phenolic wipe down
- 8–10% bleach and water wipe down

Iodophor-type disinfectants (for example, Betadine®) are not recommended and will stain fabrics.

To clean the mattress:

1. Remove the bedding.
2. Clean the top and bottom covers with organic cleaner or soapy water.
3. Use a standard cleaning solution as previously listed to clean the top and bottom covers.

---

## Maintenance

---

### Inspection

**WARNING!****Risk of Injury or Damage**

Proper patient assessment and monitoring and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions and mattress thickness, size, or density could increase the risk of entrapment.

- Refer to the Bed Rail Entrapment Risk Notification Guide (part number 1171780) included with the mattress.
- Visit the FDA website at [fda.gov](http://fda.gov) to learn about the risks of entrapment.
- Refer to the owners manuals for beds and rails for additional product and safety information.
- Note the mattress **MUST** snugly fit the bed frame and bed rails to reduce the risk of entrapment.

It is recommended that internal and external parts of the mattress be inspected every three (3) months for damage and wear. Remove the mattress from service and replace all damaged parts.

**Inspection Record**

Inspection Date	Inspector Name	Damage Found (Y/N)?	Removed from Service Date	Repair Date	Returned to Service Date

**Warranty**

**Limited Warranty**

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the mattress and cover when purchased new and unused to be free from defects in materials, workmanship, and assembly for the time periods listed in the following table:

Product	Foam	Cover
Solace Prevention, Therapy, Resolution mattresses	Five (5) years	One (1) year
Invacare Essentials mattresses	Five (5) years	One (1) year
Invacare standard foam mattress	Six (6) months	Six (6) months

If within such warranty periods any such product shall be proven to be defective, such product shall be repaired or replaced with new or refurbished product components, or a refund corresponding to the proper service time shall be offered, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; DAMAGE CAUSED BY ANY EVENTS OUTSIDE THE CONTROL OF INVACARE INCLUDING, BUT NOT LIMITED TO: DAMAGE FROM CARELESSNESS, ACCIDENTAL DAMAGE, DAMAGE FROM SHARP OBJECTS, TEARS, NEGLIGENCE (WHETHER WILLFUL OR NOT), TAMPERING, DAMAGE FROM FIRE, DAMAGE OCCURRING IN TRANSIT AFTER DELIVERY BY INVACARE OR EXCESSIVE WEAR AND TEAR, INCLUDING BUT NOT LIMITED TO THE FAILURE TO FOLLOW CLEANING INSTRUCTIONS, USE OF INAPPROPRIATE CLEANING COMPOUNDS AND FAILURE TO ENSURE THAT ANY PRODUCT IS THOROUGHLY DRIED BEFORE TURNING OR FLIPPING IT OVER; DAMAGE RESULTING FROM CONTAMINATION FOLLOWING DAMAGE TO THE COVER AND FAILURE TO REPLACE IT, CONTAMINATION DUE TO INCORRECT STORAGE OR FAILURE TO INSPECT THE INTERIOR OF THE MATTRESS AS STATED HEREIN, OR DAMAGE DUE TO UNUSUAL CONTAMINATION OR INFECTION, SPILLAGE OR STAINING, OR USE OF ABRASIVE, CORROSIVE OR ANY SUCH SIMILAR DAMAGING SUBSTANCE; DAMAGE DUE TO UNDERTAKING ANY ACTION OR PROCEDURE USING THE PRODUCT OUTSIDE ITS NORMAL AND INTENDED USE; DAMAGE RESULTING FROM FAILURE TO REPLACE DAMAGED PARTS PROMPTLY OR DAMAGE DUE TO INTERIOR PARTS BEING USED WITHOUT THE COVER, BEING USED WITH A DAMAGED COVER, OR WITH THE COVER BEING INCORRECTLY FITTED; DAMAGE WHEN ANY PARTS OF THE MATTRESS ARE USED AS PART OF, OR IN CONJUNCTION WITH, ANY OTHER PRODUCT OR PIECE OF EQUIPMENT EXCEPT AS SPECIFICALLY PROVIDED FOR BY INVACARE; DAMAGE NOT PROVEN TO THE SATISFACTION OF INVACARE TO BE DUE TO DEFECTIVE MATERIALS, WORKMANSHIP OR ASSEMBLY; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS OR PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE.

IT IS A CONDITION OF THIS WARRANTY THAT THE FOLLOWING IS COMPLIED WITH:

- 1) ALL INSTRUCTIONS FOR CLEANING AND MAINTENANCE OF THE MATTRESS ARE ADHERED TO.
- 2) THE INTERNAL AND EXTERNAL PARTS OF ANY PRODUCT ARE INSPECTED AT LEAST ONCE EVERY THREE (3) MONTHS FOR ANY DAMAGE AND WEAR. ALL DAMAGED PARTS ARE REPLACED IMMEDIATELY WITH THE APPROPRIATE REPLACEMENT PART, OR THE MATTRESS IS IMMEDIATELY TAKEN OUT OF SERVICE UNTIL THE REPLACEMENT PART HAS BEEN OBTAINED AND THE DAMAGED PART HAS BEEN REPLACED.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Some states do not allow the exclusion or limitation of incidental or consequential damage, or limitation of how long an implied warranty lasts, so the above exclusion and limitation may not be applicable.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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Invacare Corporation • One Invacare Way, Elyria, Ohio 44035 •
Invacare Corporation • One Invacare Way • Elyria, OH 44035, USA