Hospital's
Choice®
TESTED
FOR CHILDREN

Nursery Care Deluxe Health and Grooming Kit IH452

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

Quick Trip Diaper Bag, Changing Mat & Diaper Wipes Container

The Quick Trip Diaper Bag includes 3 compartments: two easy-view pockets and one on the bottom. Wipes case and changing mat can be stored easily in any of the three pockets.

To Clean: Wash with a damp cloth.

Bottle Medicine Dispenser

The Bottle Medicine Dispenser has easy-to-read dosage markings and measures up to 3 tsp. (15 ml).

To Use:

- 1 Remove top from measuring cup and fill cup to the prescribed dosage.
- 2 Snap top securely onto the measuring cup.
- 3 Bottle-feed child as normal, pausing as necessary so that child may swallow comfortably.

To Clean: Wash before first use and after every use. Disassemble top ring, nipple and measuring cup. Wash with warm, soapy water and rinse thoroughly. Alternatively, it is dishwasher safe. TOP RACK ONLY.

CAUTION: Inspect bottle nipple regularly for tears or cracks and discontinue use if any are noted. Made without BPA.

Easy Fill Medicine Syringe

Our Easy Fill Medicine Syringe is a convenient way to administer medicine to infants and toddlers. The curved, Soft Touch grip allows for comfortable finger positioning and easy dispensing.

To Use:

- 1 Insert syringe tip into medicine bottle and draw plunger back slowly until medicine reaches the prescribed dosage marking.
- 2 Hold baby upright and then insert the tip of the dispenser into your child's mouth. Point tip toward child's cheek.
- 3 Push plunger slowly to help release liquid.

To Clean: Wash all parts separately. Wash with warm, soapy water and rinse thoroughly. Repeat after administering each dosage.

CAUTION: Keep all medications out of child's reach. We recommend that you use a child resistant top to seal the medicine bottle and store it in a locked cabinet or other location that is not accessible to a child. Made without BPA.

Nasal Aspirator

To Use:

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean: Aspirator is top-rack dishwasher safe or can be washed in warm soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

WARNING: NEVER push the aspirator too far into the nostril and always wash and dry thoroughly after each use. **NEVER** use in baby's mouth or ears. When storing, keep away from heat and bright light.

! WARNING: Keep small parts away from children. These products are not toys. **DO NOT** allow children to play with them. When not in use, keep out of reach of children. **ALWAYS** supervise your child when using this kit. **NEVER** allow your child to use these products alone or without close adult supervision. We recommend that you store this kit in a locked cabinet or other location that is not accessible to a child. **FOR INDOOR HOME USE ONLY.**

Nail Clippers & 6 Emery Boards

These clippers have been specially designed for convenient adult use.

To use:

- 1 Lift arm from hand grip and rotate it 180°.
- 2 Position nail between cutting edges and squeeze handle to cut nail.
- 3 Emery board use is recommended to smooth any remaining sharp nails after clipping and also to smooth minor rough or jagged fingernail edges at other times.

To Clean: Wipe clippers with a damp cloth and dry after each use. Clippers may be sterilized by wiping with rubbing alcohol. **Note:** It is best to trim nails after a bath when they are soft, or while baby is sleeping.

CAUTION: Be careful not to trim too close to baby's sensitive skin.

Fingertip Toothbrush/Gum Massager

To Use:

- 1 Place on adult's fingertip.
- 2 Use the gum massager (nubby side) to help soothe baby's sensitive gums and the infant toothbrush side to clean emerging teeth and soothe tender gums.

Use and Care: Rinse thoroughly before first use and after each use.

WARNING: Do not allow children to play with this product. When not in use, keep out of reach of children. **ADULT SUPERVISION REQUIRED.**

Easy Grip Toddler Toothbrush

The contoured, ergonomic handle of the Toddler Toothbrush makes it easy for small hands to hold and is roll-resistant. Place it on the table or sink and the convenient, roll-resistant handle keeps it from falling off.

To Use:

- 1 Begin use once all primary teeth are in or when recommended by your dentist.
- 2 Apply a pea-sized amount of toothpaste to the brush and have your child brush for 2-3 minutes using an up and down motion.
- 3 Continue to help and supervise your child during brushing - do not allow child to swallow toothpaste.

NOTE: Toothbrushes should be replaced every 2-3 months.

To Clean: Rinse each brush thoroughly before first use and after each use.

Important Note: Questions about your child's oral health should be answered by your family dentist.

CAUTION: Adult supervision required. **DO NOT** allow child to run or walk while carrying or using toothbrush. **DO NOT** let children play with toothbrushes.

Brush & Comb Set

The Brush & Comb Set features large handles so parents and kids get a secure and comfortable grip. The hairbrush's soft bristles are extra gentle on baby's sensitive scalp while the comb's rigid teeth are perfect for getting through thicker hair.

To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

Cradle Cap Comb

Many babies develop the temporary dry skin condition called "Cradle Cap." The dry skin usually appears on the scalp. Use the Cradle Cap Comb to gently remove the loosened skin from baby's scalp. The teeth on this comb may also be great for gently combing a newborn baby's extra-fine hair.

To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

10 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.

FOR EXTERNAL USE ONLY.

Emergency Information Card

Record important phone numbers and medical information for quick reference.

3-in-1 Digital Thermometer

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.

TH093

Fahrenheit / Celsius

While thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This will take approximately 5 seconds.

! WARNING: Before using digital thermometer, please read carefully. • This is **NOT** an ear thermometer. **NEVER** put probe into ear. • Take temperature measurements Underarm, Orally or Rectally. • **ALWAYS** supervise your child when using thermometer and thermometer covers. **NEVER** allow your child to use it alone or without close parental supervision. • **NEVER** allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision. • **DO NOT** expose the thermometer to extreme temperatures, humidity, direct sunlight or shock. Avoid sudden impact and dust. • Keep battery out of child's reach. Swallowing a battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode. • Used batteries should be disposed of properly. • **POTENTIAL CHOKING HAZARD.** Do not leave child unattended when a thermometer cover is in use. • Thermometer covers are not recommended for oral use. • Keep thermometer covers out of reach of children. • Thermometer covers are intended for single use only.

Technical Specifications

Measurement Range: 90° F to 109.9° F

Measurement Accuracy: +/- 0.2° F for temps below 96° F or above 107.6° F.

Operation Environment: 60.8° F to 104° F

Storage Environment: -13° F to 144° F

Test Protocol:

- 1 Put the tip of thermometer into water bath, approximately 2 cm below the surface.
- 2 Wait for 10 seconds, then turn on the thermometer by pressing the ON/OFF button. The thermometer is now in standard mode.
- 3 When the measurement is completed (approximately 60 seconds) record the temperature reading.

How to Measure Temperature

Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

- 188.8°F M**
- Press and release ON/OFF button. All segments display briefly as the thermometer powers on.
 - The last temperature measured displays with the **M** icon on screen for a few seconds. Then the thermometer will show an average temperature (37.0.9°C or 98.5.9°F) for 0.5 seconds before entering ready mode.
 - **Lo** displays and °F flashes when the thermometer is ready to take measurement.

Specifics for each Temperature Mode

ORAL—For children 4 years and older.

RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

1. Ensure that °F is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.
2. Ensure that °F is still flashing, then insert thermometer tip gently into the rectum, taking care not to insert past the insertion gauge (ring on the probe tip). Do not force the tip into rectum if resistance is encountered.

• When °F stops flashing, the measuring is complete. Approximate measurement times are: 60 seconds for oral and rectal temps, 90 seconds or longer for underarm temp.

• Remove probe tip from measurement site. Read and record the temperature and time for reference.

• Press button to shut off or thermometer will shut off automatically after 8-9 minutes.

NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included. DO NOT allow child to walk or run during temperature taking. Probe could present a hazard.

Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +


How to Replace the Batteries

Dead Battery Indicator:

- When the **□** icon appears, the battery is exhausted and needs to be replaced.
- Battery is to be inserted with the correct polarity.
- Check that all contact surfaces are clean and bright before installing battery.

Replace Battery:

- 1 Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
- 2 Use a toothpick to push the battery out of the metal housing.
- 3 Insert a new battery (LR41 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration.
- 4 Place the battery cover on the thermometer and secure the cover.

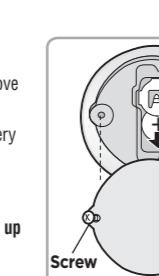Care and Maintenance

Cleaning Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

Cleaning Thermometer Body:

- Use a soft, dry cloth to clean thermometer body.

Notes:

- DO remove battery if stored for a long period of time.
- DO store thermometer in a dry place at room temperature with the protective cap in place.
- DO take special care not to scratch the display window.
- DO NOT allow electronics to get wet.
- DO NOT use abrasive cleaning agents, thinners or benzene for cleaning.
- DO NOT drop thermometer.
- Install new battery and retry function before calling Consumer Relations on electronics.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

© 2017 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com Styles and colors may vary. Made in CHINA

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