
Light Up Tongue Depressor


User Guide IH576

Thank you for purchasing this Safety 1st product. Read these instructions completely before using this product.

Parts and Features

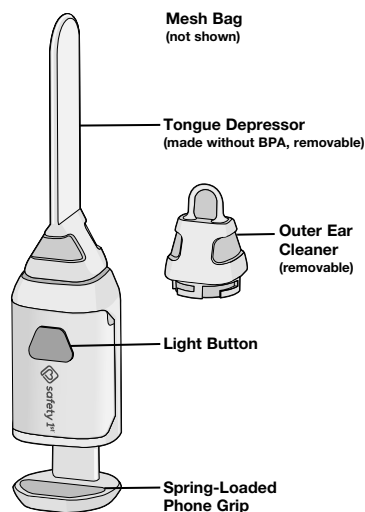How to Replace Batteries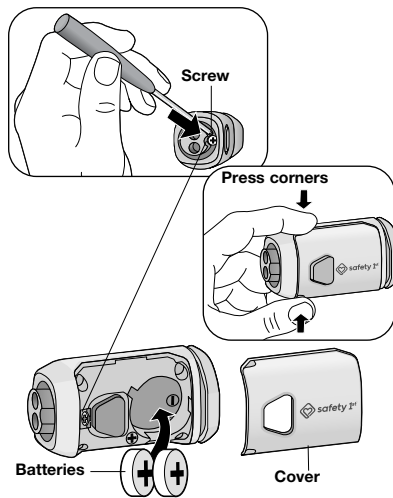

- 1 Use a small, flathead screwdriver to loosen battery cover screw. When you remove cover, the screw remains in the unit.
- 2 Press corners to remove cover.
- 3 Remove batteries.
- 4 Insert two new 3-volt batteries (CR2032 or equivalent), correctly positioned with the positive side facing up as shown.
- 5 Place the battery cover on the unit and secure with the screw.


One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Light Up Tongue Depressor), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship, Dorel Juvenile Group, Inc. will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Care Department at the address noted at the top of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse. This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages. **SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.**

How to Use


Please read this section carefully and all the way through before using this product.

NOTE: You can use the tongue depressor in hand or attached to a phone.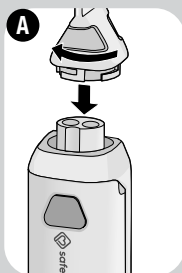

To use tongue depressor on a phone:

1. Insert attachment and twist clockwise to secure (Figure A).
2. Pull on the spring-loaded phone grip, insert edge of phone, and gently let go. Align the product with the phone so it is right underneath the camera (Figure B). **NOTE:** You can orient the tongue depressor horizontally or vertically so it lays flat on the tongue.

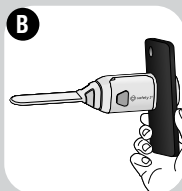
3. Press the light button (Figure C) and open the phone's camera.

4. Instruct the person to sit still and extend tongue.

5. Insert tongue depressor (but not so far that you cause the person to gag). Gently press down on tongue. Look at screen and take picture. **NOTE:** Adjust unit up or down on phone until the picture is in view. You may need to zoom in to get a clear view of the back of the mouth.

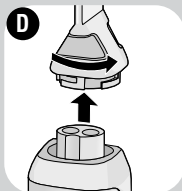6. Remove from person's mouth. Turn tip counter-clockwise to disassemble (Figure D). See Care instructions.

7. Consult your physician and share photos via email or health portal.

To use outer ear cleaner:

1. Insert attachment and twist clockwise to secure.

2. Press the light button (Figure C), then hold tip near ear for illumination. The tip lights up for better visibility. Use outer ear cleaner or swab to scrape any dirt in the outer ear. **DO NOT** use outer ear cleaner inside the ear canal.

This device should not be used as a substitute for a medical diagnosis. Only a medical professional can make a diagnosis of a medical condition. It is important that you always seek medical treatment from a qualified medical professional.

FCC Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Troubleshooting

- Install new batteries and retry function before calling Consumer Care.
- To address any issues, please visit us at www.safety1st.com to chat with us there. You can also text us at (812) 652-2525, call Consumer Care at (800) 544-1108, or email us at support@djgusa.com.
- The lightbulb is not replaceable.

DO NOT use any replacement parts not provided and approved by Dorel Juvenile Group. **DO NOT** substitute parts or try to modify the product in any way. Parts not provided and approved by Dorel Juvenile Group may result in an unsafe product.

- Keep these instructions as they contain important information.

⚠ WARNING:

- **NEVER** push the attachment too far into the mouth or ear.
- **DO NOT** use outer ear cleaner inside the ear canal.
- This product contains small parts. Keep away from children at all times.
- This product contains a Button or Coin Cell Battery. A swallowed Button or Coin Cell Battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.

⚠ CAUTION:

Adult Use Only. Not intended for use by children. This product is not a toy. When not in use, keep out of reach of children. • This is a sensitive electronic device and may be damaged if not used properly under parental supervision. **TO AVOID BATTERY LEAKAGE:** • Always follow the instructions carefully. Use only specified batteries. Make sure to insert batteries with the correct polarity. • Remove exhausted batteries from the product. • Do not short-circuit the supply terminals • Remove batteries when stored for long periods of non-use. • Do not mix old and new batteries. • Do not mix alkaline, standard (carbon-zinc), or rechargeable batteries. **RECHARGEABLE BATTERIES:** Rechargeable batteries are to be removed from the product before being charged. Rechargeable batteries are only to be charged under adult supervision. Do not recharge non-rechargeable batteries.

- Recycle or dispose pictograms following by statement: Recycle or dispose of batteries safely according to federal, state, and local laws.

Care and Maintenance

- Always wash and dry attachment thoroughly before and after each use.

Cleaning Tongue Depressor and Outer Ear Cleaner:

- Remove attachment from unit to clean.
- Wash in warm soapy water and rinse well.
- Air dry thoroughly before storing.

Cleaning Base Unit:

- **DO NOT** submerge base unit with electronics in water.
- **DO NOT** place in microwave.
- Wipe with damp cloth.

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