
INSTRUCTIONS FOR USE

Filling Your Milk Storage Bag:

1. Wash your hands thoroughly; there is no need to wash milk storage bags prior to use.
2. Remove the tamper-evident seal along the top of the milk storage bag **A**; dispose of torn-off seal immediately and keep out of the reach of children.
3. Place thumbs inside the top of the bag above the zipper and gently pull both sides apart to open the bag.
4. Pour your expressed milk into the bag; do not fill beyond the 5oz/150ml level – your breast milk will expand when frozen and the additional space at the top of the bag will be necessary to accommodate this expansion.
5. Carefully remove excess air by flattening the part of the bag above the milk and seal the bag by closing the zipper from one side, sliding along the entire zipper to the other side.
6. Run your fingers back across the zipper to ensure a secure seal.
7. Use a pen or permanent marker to record the time and date of expression, name, and volume included on the label area above the zipper **B**; do not write on the filled part of the bag to prevent puncturing the bag.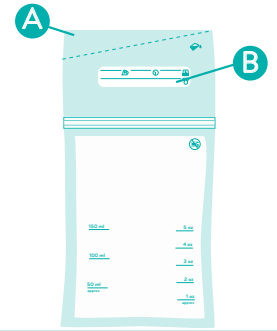

Storing Your Milk:

1. Ensure dual track zipper is completely sealed.
2. Dry the exterior of the bag before placing in refrigerator or freezer.
3. Lay bag flat in the refrigerator or freezer; if using the freezer, place in the coldest part of your freezer. Avoid placing in the freezer door where temperature fluctuates more. See the Breast Milk Storage Guidelines chart for storage time limitations.

Note: Always cool breast milk before placing into the freezer next to already frozen milk. Storing warm, fresh milk next to already frozen milk in the freezer could potentially start the defrosting process for the previously stored milk.


Defrosting Your Frozen Milk:

1. Defrost milk slowly overnight by placing the filled milk storage bag in the refrigerator. Avoid letting milk sit out at room temperature to thaw. For quicker thawing hold sealed bag under running water. Start with cool water and gradually increase the temperature to warm. NEVER thaw the milk in a microwave or boiling water.
2. Your milk may separate from fat – this is normal. With the bag sealed, gently tilt the bag back and forth until the milk is blended; do NOT shake.
3. Transfer into a clean feeding container. Previously frozen milk can be stored up to 24 hours in the refrigerator after it has finished thawing; NEVER refreeze thawed breast milk.

Transferring Your Milk:

1. Ensure your milk is defrosted completely and wash your hands thoroughly.
2. Place thumbs inside the top of the bag above the zipper and gently pull both sides apart to open the bag.
3. Holding the bag by the zipper area on the opposite side of long pour spout **C**, pour your expressed milk into a feeding container.
4. Discard milk storage bag after use; DO NOT REUSE.
5. To heat the milk, place the feeding container with the defrosted milk in a bottle warmer or in a bowl of hot water.

NOTE: If baby does not finish all of the milk at first feeding, milk can be refrigerated and offered at next feeding. Do not refreeze thawed milk.

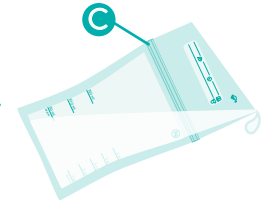CAUTION: NEVER THAW OR WARM BREAST MILK IN THE MICROWAVE OR IN BOILING WATER. These milk storage bags are intended for one-time use only; NEVER reuse storage bags. Dispose of the bag after use. Keep the milk storage bags out of the reach of children. Thoroughly wash your hands before you fill and empty milk storage bags. The actual amount of milk in the bag may deviate slightly from the level indications on the bag. Never refreeze breast milk or add fresh breast milk to already frozen breast milk. Always check the temperature of the milk before you feed your baby. Always consult your physician for proper breast milk feeding practices.

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1. After sealing your milk storage bag securely, label the bag with the date and time the milk was expressed, as well as the volume.

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2. Use the chart below as a guide for storing your breast milk for future use. Note: In hot weather, chill milk immediately to prevent spoilage.

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Note: Breast milk should be stored in the main refrigerator or freezer compartment, away from the door, to avoid changes in temperature.

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* Human Milk Banking Association of North America. Best practices for expressing, storing and handling human milk in hospitals, homes and child care settings. Raleigh, NC: 3rd Edition, 2011