



Allergy Emergency Kit Case

Case Contains:

- 1 sheet of epinephrine auto-injector instructions
- 1 sheet of wall mounting instructions
- 1 sheet of paper marked "Template"
- For wall mounting: 1 plastic bag containing 2 strips of adhesive discs (10 discs total)
- For medication mounting: 1 plastic bag containing 1 strip of red adhesive discs (6 discs total)

Instructions for Wall Mounting

1. Temporarily tape the sheet marked "Template" to the wall where you intend to mount the case.
2. Remove an adhesive disc from one of the included strips. Press the sticky side against the wall in one of the circular cutouts in the template.
3. Continue with 4 more adhesive discs until all five of the hole cutouts have a disc in them with the sticky side pressed to the wall.
4. Remove the template from the wall and close the case.
5. The Emergency Kit Case has an adhesive sheet on the back side. This will allow the case to adhere to the adhesive discs that you have placed on the wall.
 - a. Pick up the case with the printed front side of the case facing you so you can read the printing.
 - b. Press the back of the case against the wall over the adhesive discs you installed on the wall. Make sure the case looks level.
6. If you wish to adjust the level, pull the case by its handle away from the wall and the adhesive sheet will release. Then reinstall the case in the position you prefer.
7. If more adhesion is needed between the case and the wall, press more discs onto the wall within the empty spots inside the pattern.



Not made with
natural rubber latex.

Manufactured for: Dynarex Corporation
11 Dynarex Drive • Middletown, NY 10941
USA • www.dynarex.com

Symbol Glossary: dynarex.com/symbols.php

Made in China

R241002

dynarex® 
we care like family™