

## 3-in-1 Thermometer

User's Guide TH114

Please be sure to read all warnings, instructions, and notations before using this product. Please keep these instructions as they contain important information.

### Basics of Your Thermometer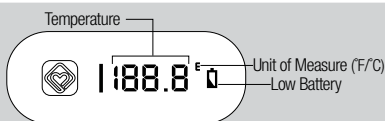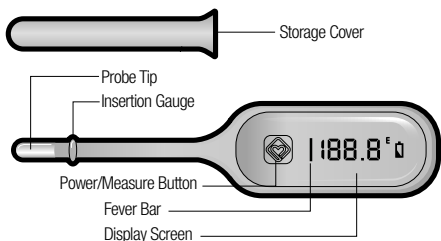

### Care and Maintenance

#### Cleaning the Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

#### Cleaning the Thermometer Body:

- Use a soft, dry cloth to clean the thermometer body.

#### Notes:

- **DO** remove the battery if stored for a long period of time.
- **DO** store the thermometer in a dry place at room temperature with the protective cap in place.
- **DO** take special care not to scratch the display window.
- **DO NOT** allow electronics to get wet.
- **DO NOT** use abrasive cleaning agents, thinners, or benzene for cleaning.
- **DO NOT** disassemble for any reason other than battery replacement.
- **DO NOT** drop the thermometer.

### Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example, oral temperatures tend to be comparable to ear temperatures and approximately 1°F (0.5°C) lower than rectal temperatures for the same individual. Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading, and technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well so that you know what is normal for your own child. On average, temperature ranges are as follows:

TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° to 99.6°F (36.1° to 37.6°C)	99.6°F + (37.6°C +)
Rectal	97.3° to 100.4°F (36.3° to 38°C)	100.4°F + (38°C +)
Underarm	95.4° to 98.6°F (35.2° to 37°C)	98.6°F + (37°C +)

# WARNING

## BEFORE USING DIGITAL THERMOMETER PLEASE READ CAREFULLY.

• This is **NOT** an ear thermometer. **NEVER** put probe into ear. • Take temperature measurements underarm, orally, or rectally. • **ALWAYS** supervise your child when using thermometer. **NEVER** allow your child to use it alone. • **NEVER** allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision. • **DO NOT** expose the thermometer to extreme temperatures, humidity, or direct sunlight. Avoid sudden impact and dust. • Keep battery out of child's reach. • This product contains a Button or Coin Cell Battery. A swallowed Button or Coin Cell Battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.

**IMPORTANT INFORMATION** • Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique. • **DO NOT** allow child to walk or run during temperature taking. Probe could present a hazard. • High, prolonged fever requires medical attention, especially for young children. Please contact your physician. • Fever chart included on package and back of thermometer.

## How to Measure Temperature

Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

#### Ready Display:

- Remove the probe tip cap. Press and release the Power/Measure button. The Fever Bar illuminates amber and all segments display briefly as the thermometer powers on.
- The thermometer will store the last temperature reading and display it automatically for 3 seconds.
- The thermometer is ready to take a measurement when and °F or °C appears (see the example at left).

## Specifics for each Temperature Mode


### ORAL—For children 4 years and older.

1. Ensure that °F (or °C) is still flashing, then place the probe tip under the tongue. Instruct your child not to bite down on the probe.


### RECTAL—Highly accurate for measuring core body temperature. Pediatrician-recommended mode from birth to age 1 and beyond as tolerated by your child.

1. Lubricate the probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position the baby so that the back and legs form a 90-degree angle (L-shape).
2. Ensure that °F (or °C) is still flashing, then insert probe tip no more than 1/2 inch (1.3 cm) into rectum. **DO NOT** force the tip into rectum if resistance is encountered. **DO NOT** insert past insertion gauge.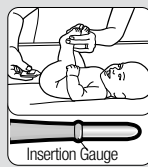

### UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

1. Wipe your child's underarm with a dry towel.
2. Ensure that °F (or °C) is still flashing, then place the probe tip at the center of your child's underarm. Bring their arm down against their body. The probe must touch the skin.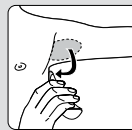

- The measuring is complete when °F (or °C) stops flashing, the thermometer beeps, and the fever bar lights up. Approximate measurement time is 8 seconds.
- Remove the probe tip from the measurement site and check the temperature.
- The Fever Bar illuminates amber for any temperature above 99.6°F (37.5°C) regardless of whether the site of measurement is oral, rectal, or underarm. The Fever Bar illuminates green for any temperature below 99.6°F (37.5°C). Consult the Fever Chart to note whether the site of measurement indicates an actual fever.

#### FEVER CHART

Under Arm	98.6°F + 37°C +
Oral	99.6°F + 37.5°C +
Rectal	100.4°F + 38°C +

## Troubleshooting

Display	Meaning	Recommended Action
	<ul style="list-style-type: none"> <li>• The thermometer may have been stored in a hot environment.</li> <li>• The measured temperature is above this thermometer's range.</li> </ul>	<ul style="list-style-type: none"> <li>• Wait 30 minutes to allow the thermometer to reach room temperature, then try again.</li> <li>• See the Technical Specifications section for Measurement Temperature Range.</li> </ul>
	<ul style="list-style-type: none"> <li>• The thermometer may have been mishandled and may be irreversibly damaged.</li> <li>• The temperature reads below 32°F/0°C or above 122°F/50°C.</li> </ul>	<ul style="list-style-type: none"> <li>• Shut off and restart before discarding.</li> <li>• Wait 30 minutes to allow the thermometer to reach room temperature, then try again.</li> </ul>
		<ul style="list-style-type: none"> <li>• Batteries are low. Replace them. (See the How to Replace the Batteries section).</li> </ul>

- Do not return this product to the place of purchase. If you have any questions about this product, email support@djgusa.com or chat with us at www.safety1st.com. You can also call Consumer Care at (800) 544-1108 or text us at (812) 652-2525. Have ready the model number (TH114) and date code (manufacture date located on package).
- Install a new battery and retry function before calling Consumer Care on electronics.

### Memory Recall

The thermometer stores the last temperature reading and displays it automatically for 3 seconds when the thermometer is powered on.

98.2° F

### Fahrenheit / Celsius

#### To switch from one unit of measurement to the other:

While the thermometer is off, press and hold the Power/Measure button for about 3 seconds. You will see a flashing F° or C° and hear a short beep. Press the Power/Measure button to toggle between F° and C°.

### Sound

To turn Sound on/off: Press and hold the Power/Measure button for about 5 seconds. The display will flash to confirm sound being turned on/off.

### FCC Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.


### How to Replace the Batteries

#### Dead Battery Indicator:

When the "D" icon flashes, the battery is exhausted and needs to be replaced. **NOTE: DO NOT** press or hold down the power button at any point while changing the battery or replacing the battery door.

#### To Replace the Battery:

- 1 Use a miniature Phillips head screwdriver to loosen the battery cover screw. Remove the cover and the old battery.
- 2 Insert a new 3 volt battery (CR2032 or equivalent) correctly positioned with the **positive side facing up** as shown in the illustration.
- 3 Place the battery cover on the thermometer and secure the cover.

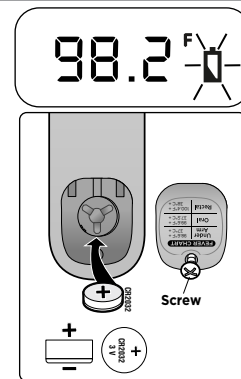#### CAUTION

#### TO AVOID BATTERY LEAKAGE

- Always follow the instructions carefully. Use only specified batteries. Make sure to insert batteries with the correct polarity.
- Remove exhausted batteries from the product.
- Do not short-circuit the supply terminals
- Remove batteries when stored for long periods of non-use.

Recycle or dispose of batteries safely according to federal, state, and local laws.

### Technical Specifications

**Measurement Temperature Range:** 89.6°F to 111°F (32°C to 43.8°C)  
**Measurement Accuracy:** ±0.3°F (±0.2°C) for temps 96.4° to 97.9°F (35.8° to 36.6°C)  
±0.2°F (±0.1°C) for temps 98° to 102°F (36.7° to 38.9°C)  
±0.3°F (±0.2°C) for temps 102.1° to 106°F (38.9° to 41.1°C)  
**Operation Environment:** 50°F to 104°F (10°C to 40°C)  
**Storage Environment:** -13°F to 140°F (-25°C to 60°C)

### One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

**SOME PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.**

**THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY PROVINCE STATE TO PROVINCE.**

©2026 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com. Styles, colors, and labeling may vary. Les styles, les couleurs et l'étiquetage peuvent varier. Los estilos, colores y etiquetado pueden variar. Made in CHINA. Fabriqué en CHINA. Hecho en CHINA. Distributed by (distribué par/distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2233 Argenta Road, Suite 110, Mississauga, ON L5N 2X7. Representante exclusivo en Chile: DOREL JUVENILE CHILE S.A. Av. del Parque 4161, oficina 602 A, Huechuraba, Santiago, Chile - Fono: 56 2 24971500 - www.infanti.cl - www.dorel.cl. Importado y distribuido en Perú por: COMEXA COMERCIALIZADORA EXTRANJERA S.A. SUCURSAL DEL PERÚ - Los Libertadores 455, San Isidro, Lima 27, Lima, Perú RUC 20431062870 - Fono: 4413029 - www.infanti.com.pe. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Via 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle, Benito Juárez C.P. 03100 México, Ciudad de México R.F.C. DJG140305GN9 Tel. (55) 67199202 03/26 4358-8838

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