MOBILITY DESIGNED

# FUSION™

## 2 in 1 Rollator and Transport Chair

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VER.A.9.25

# Parts List

- |                              |                           |                        |
|------------------------------|---------------------------|------------------------|
| <b>1</b> Hand Grip           | <b>5</b> Armrests         | <b>9</b> Wheels        |
| <b>2</b> Brake Handle        | <b>6</b> Seat             | <b>10</b> Folding Clip |
| <b>3</b> Folding Strap       | <b>7</b> Removable Basket | <b>11</b> Curb Climber |
| <b>4</b> Reversible Backrest | <b>8</b> Footrests        |                        |


# Assembly Instructions

- 1** Remove contents from carton.

# Assembly Instructions

- 2** Unfold the frame. Push down on the seat supports to expand and open the frame. The safety lock will click into place when the rollator is fully unfolded.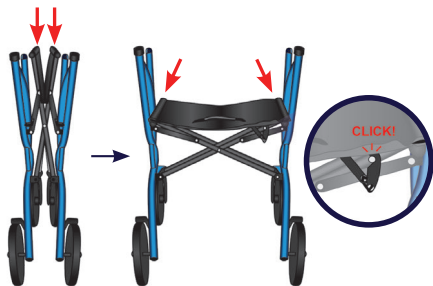

- 3** Insert the handlebars. Ensure the brake cables are on the outside of the frame and that they pass through the cable guide.

- 4** For height adjustment, stand upright and set the handgrips even with your wrists. Hand tighten the triangular hand screws.

# Assembly Instructions

- 5** Install the footrests. Match the letters on the frame (L and R) to the same letters on the footrests. Press the push-buttons on the footrests tubing and insert into the Fusion's frame. The footrests can be locked in two different positions (front, for transport chair use, and side, for rollator use). To rotate the footrests, press the push-buttons and rotate.

When not in use, the footrests can be conveniently stored by pushing the push-button and swinging them to the side of the frame where another push-button will hold them securely in place. This allows you to lift and store your Fusion safely. The footrests can also be removed when not in use.

TIP: the black cap on the footrest tubing can be removed and used to cover the open tubing where the footrests slot into.


- 6** Install the backrest and the basket. Insert the backrest so that the logo faces the front wheels. Slide one end of the backrest into the fan bracket. Insert the screw from the inside of the unit out and repeat. Install the basket by sliding it into the mounts starting from the back and then pulling to the front.


# Directions

**Folding:** Pull the folding strap on the seat to release the safety lock and initiate folding. Use the folding clip to lock the rollator in the folded positions.

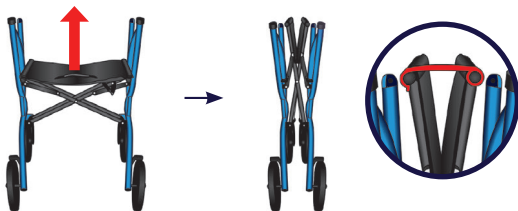
**Using the brake handles:** Squeeze up to slow down. To lock, push down until they click. To release, pull back up. ALWAYS lock the brakes before using the seat.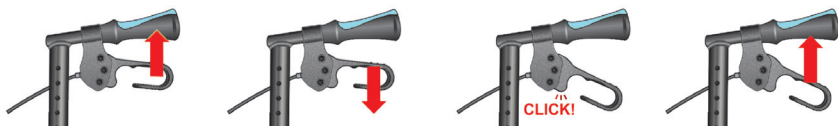

**Brake Adjustment:** Push down the adjustment knob and rotate. Clockwise will loosen the brakes, counter-clockwise will tighten them. Your brakes must be checked periodically for safety.

**IMPORTANT:** Before using your rollator again, ensure the brakes are working properly. DO NOT use if the wheels rotate while the brakes are locked.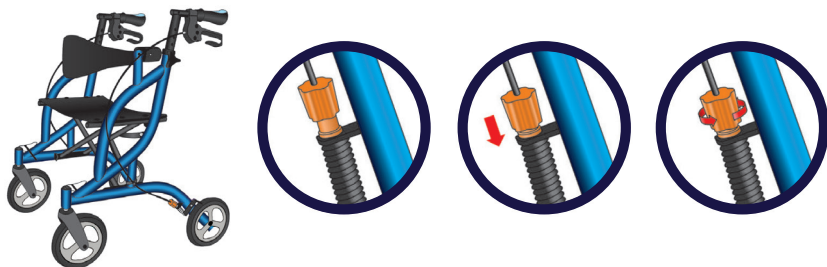


# Directions


## Rollator & Transport Chair Mode:

**ROLLATOR MODE** – Flip the backrest away from the handlebars so that it is positioned towards the front wheels. Before entering or exiting the seat always ensure the brakes are in the locked position by pushing down on both brake handles until you hear them click.

**TRANSPORT CHAIR MODE** – Flip the backrest toward the handlebars so that it is positioned to the back of the Fusion. Then, for each footrest, press on the push-buttons above the front wheel and rotate the footplate toward the front of the Fusion until it locks into place. Before entering or exiting the seat, always ensure that the footrest plates are folded up and that the brakes are locked. Once the user is seated, deploy the footrest plates. Their height can be adjusted using the push-buttons on their tubing.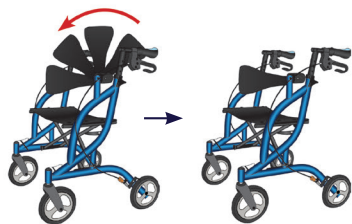

**Transport Chair Mode**

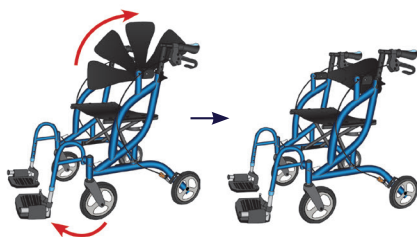**Rollator Mode**

# Safety Instructions

## **WEIGHT CAPACITY: 300 LB / 136 KG**

- Use only on even and solid surfaces.
- Maximum user weight: 300 lb.
- Regularly check the screws and fasteners and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lb.
- Do not expose the product to temperatures above 104°F for an extended period as this can cause parts to overheat and burn the user.
- Please make sure the Fusion is fully opened, seat is down and the folding bar under the seat is locked in the down position before using or sitting in the Fusion.
- Be sure both hand grips are at equal heights. Test Fusion for stability before applying full weight to hand grips.

# Safety Instructions

- Test the brake lever before operating the Fusion. When wheels are locked, the brake pad should prevent the wheels from turning. When the brakes are released, the wheels should turn freely.
- Replace tires when significantly worn or damaged.
- When in rollator mode, do not move rollator while seated.
- Do not navigate independently while seated. Do not attempt to sit on the seat while on an incline. Do not lean when you are seated. Doing so may cause the walker to tip and a serious risk of injury may result.
- Falls are possible if you use the seat without installing the backrest.
- Do not use your rollator while folded. The stability of any rollator is compromised when the frame is folded, increasing the likelihood of a fall.
- Do not exceed the following weight limit or serious injury could result:  
Rollator Mode – 300 lb      Transport chair mode – 300 lb

## Cautions:

- Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
- Before each use, make sure that all parts are secure and that the frame is fully opened.
- Risk of fall and serious injury may result if brakes are not locked while the user attempts to sit down.
- Do not exceed 300 lbs weight limit or serious injury could result.

**Caution when seated:** Leaning out of the Fusion without proper assistance could cause tipping. To reduce the risk of an accident, NEVER reach or lean when you are seated in your Fusion. Instead, have someone present to help you.

**Caution when pushing:** When pushing someone in the Fusion, do not roll across small uneven surfaces. Doing so can cause the unit to tip forward. To safely cross curbs, uneven surfaces or use on large includes, the person sitting on the Fusion must get out of the seat.

# Maintenance

M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. Clean regularly by hand with a damp cloth and mild detergent only. Do not use aggressive detergents such as chlorine, bleach or abrasive cleaners. If you experience problems, please contact the provider that sold you the product or visit [www.drivemedical.com/help](http://www.drivemedical.com/help) for more information.

# Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.