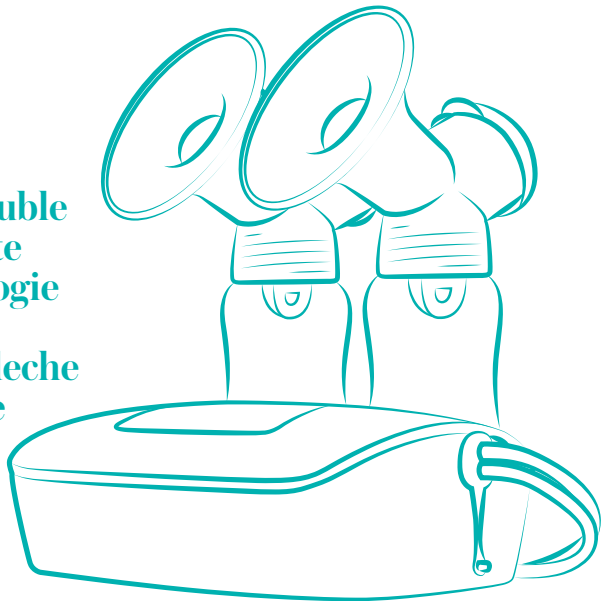
# Advanced Double Electric Breast Pump

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For video instructions about how to use and care for your pump,  
visit: [www.evenflofeeding.com](http://www.evenflofeeding.com)

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Thank you for choosing Evenflo Feeding to support you on your breastfeeding journey. If you have questions about the performance or operation of this pump, please contact our Parent's Ally Consumer Care Team.

For more information about breastfeeding or pumping, contact a lactation expert, your physician, or call us at USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at [www.evenflofeeding.com](http://www.evenflofeeding.com).

Please find the most up-to-date instruction manual at [www.evenflofeeding.com](http://www.evenflofeeding.com).

This pump is a single user medical device and should never be shared between mothers, resold, or given from one mother to another. Once the package has been opened, this product cannot be returned.

Please note that this pump should not be serviced or disassembled by the consumer. If you have any issues or concerns with the product, please contact our Parent's Ally Consumer Care Team.

Evenflo Feeding has established a robust support program for using your pump, including 1:1 pumping support by phone and video with the goal of making your pumping success our top priority.

After working with our Parent's Ally support team, our experts will determine what solutions we can offer, in accordance with our policies. For warranty information please visit our website at [www.evenflofeeding.com/warranty](http://www.evenflofeeding.com/warranty).

Important Safety Information ..... 2-3  
 Identification of Parts ..... 4  
 Before First Use ..... 5-6  
 Assembling the Flange ..... 7  
 Single Versus Double Pumping ..... 8  
 Attaching Flange Assembly to Pump Unit ..... 8  
 Power Operations ..... 8  
 Using the AC Adapter ..... 9  
 Using Batteries ..... 9-10  
 Check the Fit of the Flange ..... 10-11  
 Using Your Breast Pump ..... 11-13  
 After Every Use ..... 14  
 Storing Your Breast Milk..... 14-15  
 Using Your Breast Milk ..... 15  
 Maintenance ..... 16  
 Troubleshooting ..... 17-19

### Read all instructions before using and save for future use.

When using this pump and other electrical products, basic safety precautions should always be maintained, including the following:

### DANGER: To reduce the risk of burns, electrocution, fire or injury to persons:

#### ▲ GENERAL USAGE & SAFEGUARDS & WARNINGS

- **ALWAYS** unplug this product immediately after using.
- **DO NOT** use while bathing.
- **DO NOT** place or store this product where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.
- **DO NOT** reach for an electrical product that has fallen into water. Unplug the product immediately.
- **NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven.
- Use this product only for its intended use as described in these instructions.
- Breast pumps are personal care items, for one user only.
- This product should never be left unattended when plugged in.
- **NEVER** use while sleepy or drowsy.
- Close supervision is necessary when this product is used near children.
- **DO NOT** allow small children or pets to play with the pump unit, flange, AC adapter, etc. These parts may pose a choking or strangulation hazard.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- **DO NOT** use attachments not recommended by the manufacturer.
- **DO NOT** use this product if it is broken or any parts are missing.
- **NEVER** drop or insert any object into any opening of the pump unit.

#### ▲ FLANGE USAGE SAFEGUARDS & WARNINGS

- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening. See page 11 for proper flange fit.

#### ▲ PUMP POWER SAFEGUARDS & WARNINGS

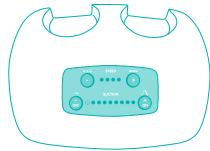
- **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, dropped into water, or if any parts are broken or missing.
- Keep the cord away from heated surfaces.
- **DO NOT** use outdoors with the AC adapter or operate where aerosol spray products are being used, or oxygen is being administered.
- Operate pump using ONLY 6 "AA" name-brand batteries (not included) or with the Evenflo AC adapter provided, and plug the adapter into a standard wall outlet.
- Operate the AC adapter on 100 – 240 volts 50/60 Hz AC only. Connect only to household outlet power supplies of this description. For use in countries outside North America, a plug converter (not included) may be required.
- Protect power cord and AC adapter cord. Route cords so that they are not walked on or pinched by items placed upon them.
- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump.
- **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon-zinc), or rechargeable batteries; leakage, explosion, and/or fire could occur.
- Remove batteries before storing unit for a prolonged period of time.

#### ▲ BREAST MILK SAFEGUARDS & WARNINGS


- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds. See page 15 for proper use of breast milk.
- **DO NOT** re-freeze breast milk. See page 14 for proper milk storage.
- **ALWAYS** test temperature of liquid before feeding.
- Mothers should NOT breastfeed or feed expressed milk to their infants if the infant is diagnosed with galactosemia or the mother is diagnosed with HIV, HTLV-1/2, Ebola, has untreated brucellosis or tuberculosis, active HSV lesions on breast, or uses illicit street drugs.

**Indications for Use:** The Evenflo Advanced Double Electric Breast Pump is an electrically powered suction device intended to express and collect milk from a lactating woman's breasts. This pump is intended for a single user.

**SAVE THESE INSTRUCTIONS FOR FUTURE USE.**

Pump Unit

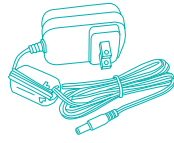
AC Adapter

Tubing Adapter

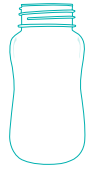Tubing


Bottle

Bottle Cap Ring

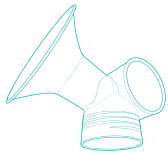Sealing Disc

Flange  
(30.5 mm)

Valve

Membrane

Diaphragm  
Cap

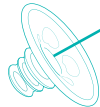
Diaphragm

24.5-mm Flange Insert

28.0-mm Flange Insert

Included flange inserts vary by kit.  
Additional sizes available separately, see pages 10-11.

Universal Shoulder Bag  
(Some Models Only)


Ice Pack  
(Some Models Only)

Cooler Bag  
(Some Models Only)


**IMPORTANT**

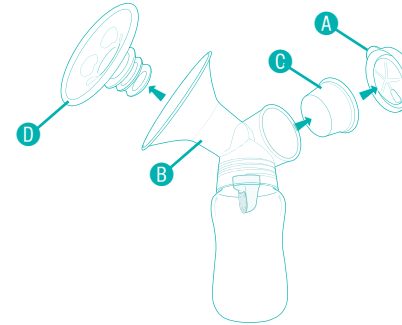Before using your breast pump, disassemble, clean and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.


**1**

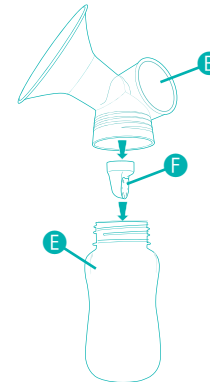Remove diaphragm cap **A** from flange assembly **B** by pulling up on cap tab. Remove diaphragm **C** and flange insert **D** from flange assembly.

**2**

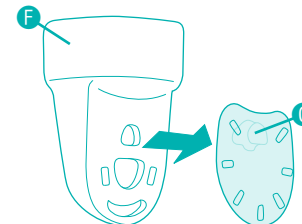Unscrew bottle **E** from bottom of flange **B**. Remove valve **F** from the bottom of the flange **B**.

**3**

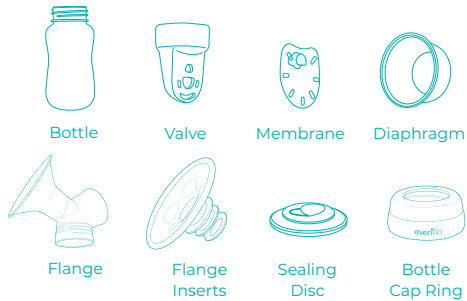
Remove membrane **G** from valve **F**.

Repeat steps 1 through 3 for second flange disassembly.

## BEFORE FIRST USE

**Clean Your Pump Parts:** To prevent warping, do not place anything on top of pump parts during sanitizing, washing, or drying. Do not assemble pump until parts are cool and dry.

### PARTS THAT MAY BE SANITIZED:

#### To Sanitize Parts on the Stove:

- Place disassembled pump parts in a pan of boiling water for 5 minutes. Stir frequently.
- **DO NOT** allow parts to rest on bottom of pan as they may melt or become damaged.

#### To Sanitize Parts in the Microwave:

- Use Evenflo Feeding Silicone Steam Sanitizing Bag according to instructions

### PARTS THAT ARE HAND WASH ONLY:

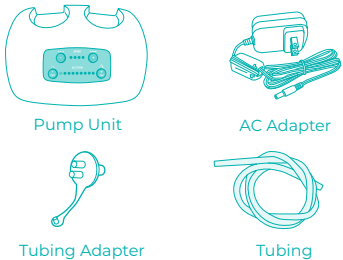
Diaphragm Cap

#### ⚠ IMPORTANT

**DO NOT BOIL OR SANITIZE IN MICROWAVE.** Boiling or sanitizing in the microwave will damage these parts and the **PUMP WILL NO LONGER WORK.**

### DO NOT WASH:

Pump Unit

AC Adapter

Tubing Adapter

Tubing


#### ⚠ WARNING

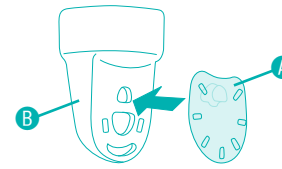**NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. Do not boil or wash tubing or tubing adapter. Wipe the pump unit with a clean, damp cloth as needed.

### Before Every Use

- Wash your hands with soap and water before handling your pump.
- Ensure parts are clean and dry. Refer to cleaning instructions on page 14.
- Use care when handling the valve/membrane assembly to avoid damage.
- Visually inspect the individual components for cracks, chips, tears, or deterioration. In the event that damage to the device is observed, please discontinue use and purchase replacement parts.


## ASSEMBLING THE FLANGE

1


Attach the membrane **A** to the valve **B**. Ensure membrane can move by gently lifting the membrane away from the valve. The membrane should return to lie flat against the valve.

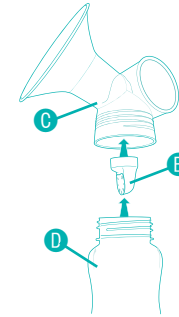**Note:** If the membrane is not lying flat against the valve, the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this too will cause decreased suction and the pump not to work properly.

2

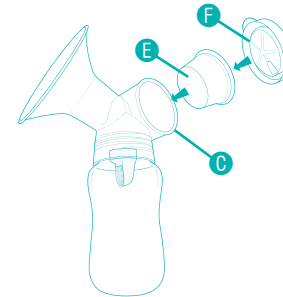
Push valve assembly **B** onto the bottom of the flange **C**. Screw bottle **D** into bottom of flange.

3

Install diaphragm **E** and firmly snap diaphragm cap **F** on to flange assembly **C**.

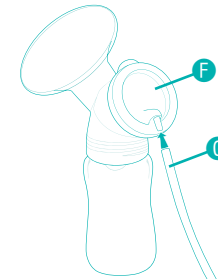**Note:** Orient diaphragm cap as desired before snapping into place.

4


Attach tubing **G** to diaphragm cap **F**.

**Note:** If you intend to pump from both breasts at the same time, repeat steps 1 through 4 for the second flange.

## SINGLE VERSUS DOUBLE PUMPING

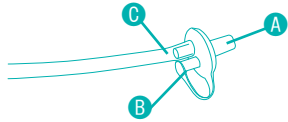
The tubing adapter can be used to pump with one or both flanges.

### To Pump One Breast at a Time:

Plug one side of the tubing adapter (A) with the attached cap (B). Connect one tube (C) to the other side.

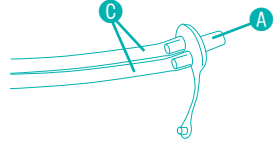
**Note:** The open side must be completely capped to ensure proper suction.

### To Pump Both Breasts at the Same Time:

Ensure that both tubes (C) are properly connected to the tubing adapter (A).
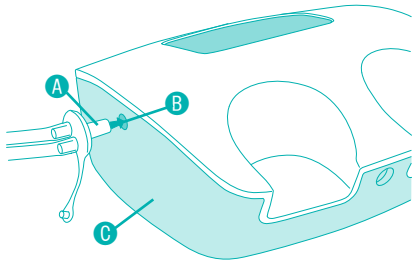
## ATTACHING FLANGE ASSEMBLY TO PUMP UNIT

Insert tubing adapter (A) into the port (B) on the side of the pump unit (C).

**Note:** Always operate with a flange assembly attached to minimize damage to the pump unit.


## POWER OPTIONS

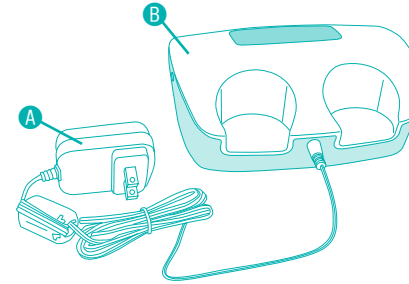This pump can be powered by: the supplied Evenflo AC adapter, six "AA" brand name batteries, or the Evenflo Advanced Car Adapter (sold separately).

**Note:** When batteries are first installed or when adapter is first plugged in, the pump unit will briefly turn on then off; this is normal.

## USING THE AC ADAPTER

### ⚠ WARNING

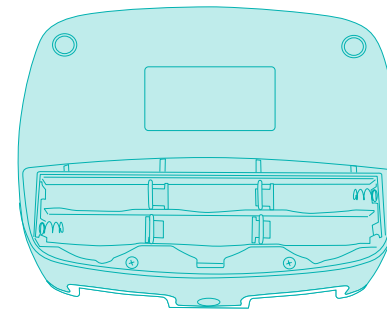
- **DO NOT** under any circumstances, use this AC adapter with any other equipment. Likewise, **DO NOT** use the adapter from any other product on this breast pump. If you lose the AC adapter, contact Parent's Ally Consumer Care.
- **ALWAYS** remove plug from product and wall by pulling on the plug/housing. **DO NOT** pull on the cord as this could damage the cord.
- **NEVER** operate this product if it has a damaged plug or cord.

**Note:** AC adapters are not interchangeable between Evenflo Models


To use the AC adapter (A), be sure there are no batteries installed. Plug AC adapter into the pump unit (B) then plug the adapter into 100 – 240V AC power only (standard household outlet). For use in countries outside of North America, a plug converter (not included) may be required. Unplug the adapter from the outlet after each use.

## USING BATTERIES

### ⚠ WARNING

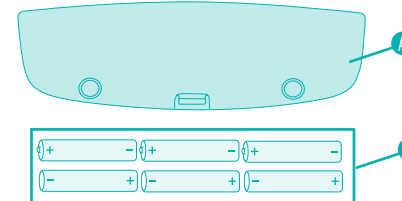- **DO NOT** mix old and new batteries.
- **DO NOT** mix battery types; leakage, explosion and/or fire could occur.
- Remove batteries before putting product into storage.


## USING BATTERIES

To use batteries, lift tab on battery door **1** and insert six alkaline “AA” batteries **2** (not included). Replace battery door. For best performance, brand name non-rechargeable batteries are recommended. Generic batteries may impact pump performance.

### Low Battery Indication

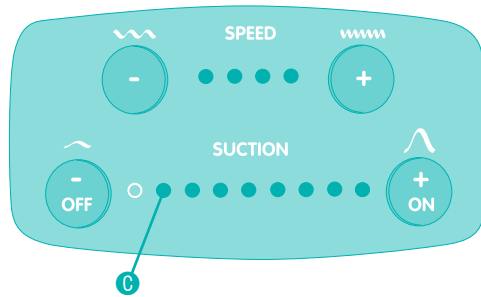When batteries are low, the first suction level indicator **3** will blink, but the pump will still be usable.

Once battery level is too low to pump, the pump will stop and not turn on until the batteries are replaced or the AC adapter is used. During this time the first suction level indicator **3** will blink, but the pump will not turn on.

## CHECK THE FIT OF THE FLANGE

The Advanced Double Electric Pump works with the Evenflo AdvancedFit™ Flange System. Additional sizes are available separately for purchase, allowing you to find your best fit.

### THE ADVANCEDFIT™ FLANGE SYSTEM AVAILABLE SIZES INCLUDE:

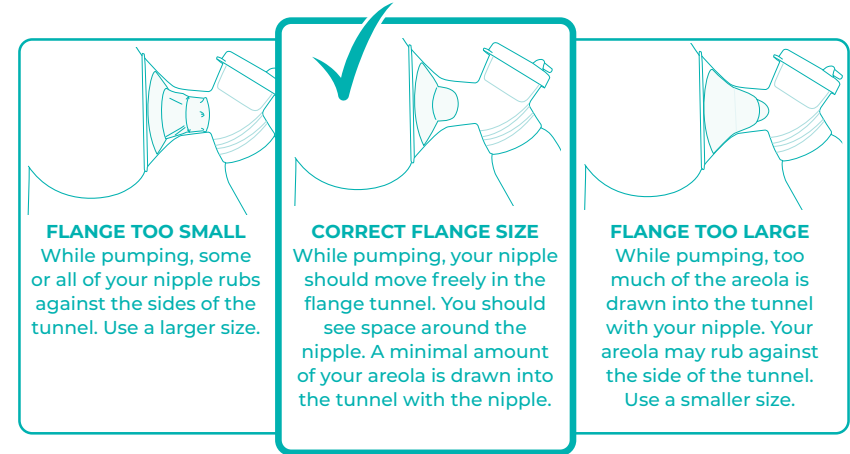
- 21.0mm (insert)
- 24.5mm (insert)
- 28.0mm (insert)
- 30.5mm (no insert)
- 33.0mm (insert)
- 36.0mm (no insert)

To use silicone flange insert, press it into the tunnel of the hard flange firmly. Then ensure that the outside edges of the insert wrap around the hard flange.

## CHECK THE FIT OF THE FLANGE

### Correct Flange Size vs. Too Small or Too Large

A proper flange size is needed for comfort and to maintain your milk supply. Your flange size can change several times throughout your journey. Recheck your flange fit frequently to ensure you are using the proper flange size.

**FLANGE TOO SMALL**  
While pumping, some or all of your nipple rubs against the sides of the tunnel. Use a larger size.

**CORRECT FLANGE SIZE**  
While pumping, your nipple should move freely in the flange tunnel. You should see space around the nipple. A minimal amount of your areola is drawn into the tunnel with the nipple.

**FLANGE TOO LARGE**  
While pumping, too much of the areola is drawn into the tunnel with your nipple. Your areola may rub against the side of the tunnel. Use a smaller size.

### IMPORTANT

- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort. Additionally, each breast may require a different flange size.
- Included flange sizes may not fit all breast and nipple sizes, but additional sizes can be purchased separately. If you are having issues after trying all included sizes, please contact our Parent's Ally Consumer Care for assistance.

## USING YOUR BREAST PUMP

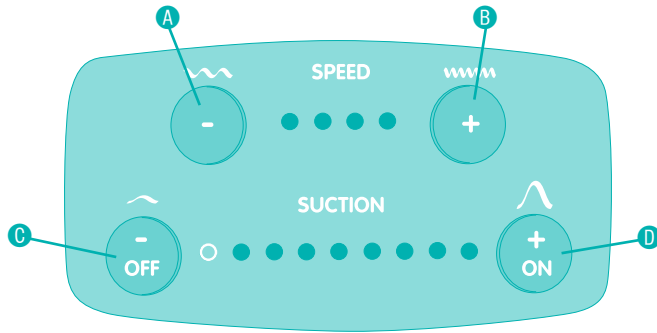
1

To ensure a good seal, place the flange **1** gently, but firmly, against your breast, with your nipple directly in the center, as shown. Hold the bottle upright to ensure that milk flows into the bottle. Use the flange or appropriate flange insert that is the most comfortable and seals against your breast the best.

**Note:** Use only one soft flange insert per flange. Be sure the outside edge of the flange has a complete seal against the breast without excessive pressure.

2

Using The Controls

**Turn on the pump** by pressing the “ON” button **D**.

The pump will turn on at the highest speed and lowest suction setting to help stimulate your first letdown (milk release).

When milk starts to flow, **decrease speed** **A** and **increase suction** **D** to the highest comfortable setting. If uncomfortable, turn suction down slightly.

When milk flow slows, **increase speed** **B** and maintain suction setting. When milk begins to flow again, **decrease speed** **A** halfway. If desired, repeat this step for additional letdowns.

**Turn off the pump.** After breast milk has been expressed, **decrease suction** **D** to the lowest setting. Release button, then press and hold the “OFF” button **C** until pump turns off (approximately 1 to 2 seconds). If necessary, slide a finger between the flange and your breast to break any remaining suction, then remove the flange.

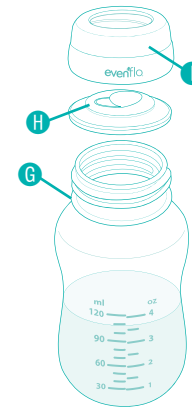
IMPORTANT

If you feel pain in your breast or nipple, turn the pump off (see above) or slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort at every suction level, turn the pump off and contact your healthcare provider for assistance.

*TIP: Some users find that using the highest speed with the highest comfortable suction throughout the pumping session yields the most amount of milk in the shortest amount of time.*


**NOTE:** Pump will automatically shut off after 40 minutes. When preparing to store milk, do not overfill the 5-ounce collection bottle; when freezing your milk for storage, space is needed in the bottle for the milk to expand.

3

Carefully unscrew bottle **G** from flange. You may now transfer milk into your preferred milk storage container, such as breast milk storage bags, or you can store your milk directly in the milk collection bottle. Place the sealing disc **F** into the cap ring **H** and screw onto the bottle securely. (see page 14 for information on storing breast milk)

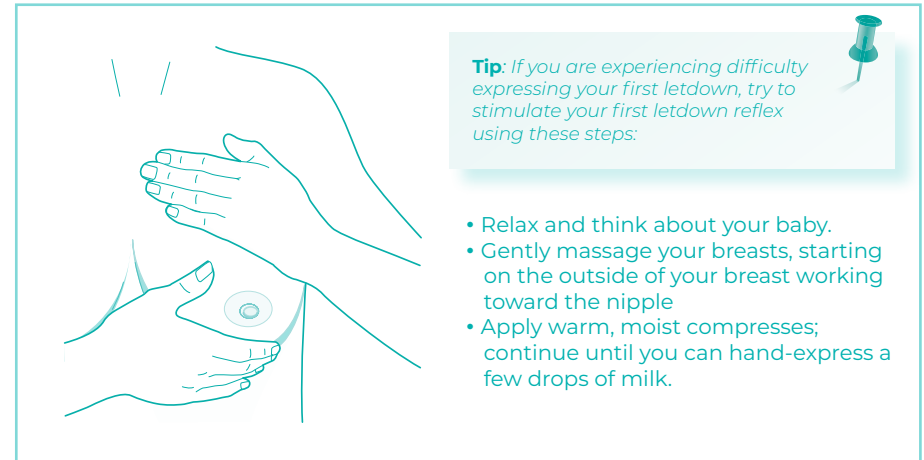**Note:** When using as a single pump on one breast at a time, repeat steps 1 through 3 on your other breast.

**Tip:** If you are experiencing difficulty expressing your first letdown, try to stimulate your first letdown reflex using these steps:

- Relax and think about your baby.
- Gently massage your breasts, starting on the outside of your breast working toward the nipple
- Apply warm, moist compresses; continue until you can hand-express a few drops of milk.

## AFTER EVERY USE

For optimal performance, follow these steps after every use:

- **Completely disassemble the flange assembly, see page 5.**
  - Rinse bottles, diaphragms, valves, membranes, flanges, and flange inserts in cool water to remove milk residue.
  - Hand wash with warm, soapy water (approx +30°C / +86°F). Never soak parts in hot water.
  - To prevent warping, do not place anything on top of pump parts during sanitizing, washing, or drying. Do not assemble pump until parts are cool and dry.
  - Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
  - Use care when handling the valve/membrane assembly during cleaning to avoid damage. Before each use, visually inspect the individual components for cracks, chips, tears, or deterioration. In the event that damage to the device is observed, please discontinue use and purchase replacement parts.
- Wipe the pump unit, tubing, and diaphragm cap with a clean, damp cloth as needed.
- Inspect the tubing for condensation or milk after each use. If condensation is observed in the tubing, contact Evenflo for replacement parts.

### ▲ WARNING

**NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. **DO NOT** boil or wash tubing or tubing adapter.

## STORING YOUR BREAST MILK

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart below as a guide for storing your breast milk for future use.

**Note:** In hot weather, chill milk immediately to prevent spoilage.

### Breast Milk Storage Guidelines\*

	Room Temperature	Refrigerator	Freezer
Freshly Expressed Breast Milk	Up to 4 hours	Up to 4 days	Within 6 months is best Up to 12 months is acceptable
Thawed Breast Milk (frozen)	1 - 2 hours	24 hours	<b>NEVER</b> re-freeze milk after it has thawed

\* CDC. *Proper Storage and Preparation of Breast Milk, Revised 2020.* [https://www.cdc.gov/breastfeeding/recommendations/handling\\_breastmilk.htm](https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm)

\* ABM Clinical Protocol #8: *Human Milk Storage Information for Home Use for Full-Term Infants, Revised 2017.* <https://abm.memberclicks.net/assets/DOCUMENTS/PROTOCOLS/8-human-milk-storage-protocol-english.pdf>

## STORING YOUR BREAST MILK

**Note:** Breast milk should be stored in the refrigerator or freezer compartment away from the door to avoid changes in temperature.

**For models that include a cooler bag:**

1. Place frozen ice packs in the included cooler bag.
2. Place the sealed bottle(s) inside and zip the bag closed.
3. When frozen ice packs are used and the cooler bag is kept at room temperature of 68 – 77° F (20 – 25° C), breast milk can be safely stored for up to 10 hours.

**Note:** Avoid storing the cooler bag in a hot closet, cabinet, or car.

## USING YOUR BREAST MILK

1. Thaw frozen milk slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, you may warm refrigerated milk:
  - Hold the sealed bottle under warm, running water, or
  - Use a bottle warmer to the manufacturer's instructions, or
  - Heat water in a cup or other small container, then place sealed bottle in the water to warm.
  - **DO NOT** heat breast milk for more than 20 minutes.
3. The milk fat may separate while stored. Briefly shake or swirl milk to mix before offering it to baby.
4. If feeding from the collection bottle, you must use **Evenflo Balance + Standard Nipples**. Other brands or types of nipples will not work effectively. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the **TOP** of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care not to over or under tighten. *Evenflo Balance + Standard Nipples sold separately.*
5. Always test the temperature of the milk before feeding.

### IMPORTANT

- **NEVER** microwave human milk or heat directly on the stove.
- If baby does not finish breast milk at one feeding, unfinished amount must be discarded within 2 hours.
- Cool freshly pumped breast milk before adding to refrigerated or frozen breast milk.

For optimal performance, keep spare parts on hand. Replace at the first sign of wear or if you pump less milk than expected. It is normal for pump parts to lose elasticity and wear over time with use, washing, and sanitizing. Here is a guide for pump maintenance.

**Breast Pump Part Replacement Guide**

Part	Pump exclusively or 4+ times a day	Pump 1-3 times a day or less
 Membranes	2-3 weeks	2-3 months
 Diaphragm	6-8 weeks	3-6 months
 Valve, Diaphragm cap, and flanges (hard, non- moving pieces),	6 months	6 months

Problem	Possible Cause	Action
Pump won't turn on	AC adapter not connected properly	Check adapter connections ( <b>PAGE 9</b> ).
	No batteries or low batteries	Make sure fresh, brand-name, alkaline, non-rechargeable batteries are installed correctly (p. 9-10). If suction level 1 light is flashing when the pump is off, the batteries are too low to effectively pump.
Milk won't flow into the bottle	The membrane is stuck to the valve	Clean the valve and reassemble, per step 1 in Assembling Your Breast Pump section ( <b>PAGE 7</b> ).
	Fast milk flow	Use a slower speed or remove flange briefly from breast to allow milk to flow into bottle.
Little to no suction	Not using Evenflo AC adapter	Only use provided Evenflo AC adapter.
	Tubing or tubing adapter is loose or not connected	Tubing adapter should be pushed firmly into pump unit. Tubing should be pushed completely onto the tubing adapter and diaphragm caps. ( <b>PAGE 8</b> ).
	Tubing or tubing adapter is damaged	Inspect both pieces of tubing for any holes or cracks. Inspect tubing adapter for any damage or obstruction.
	Diaphragm not inserted into flange	Properly install diaphragm. ( <b>STEP 3 PAGE 7</b> )
	Diaphragm is damaged or worn	Check diaphragm for signs of wear, holes, cracks, etc. Replace damaged components.
	Diaphragm cap(s) not firmly snapped on	Attach diaphragm cap firmly to flange, ( <b>STEP 3 PAGE 7</b> ). You should feel a slight snap when installed properly.
	Valve not assembled to flange	Attach the valve firmly to flange ( <b>PAGE 7, STEP 2</b> ).
	Membrane worn, torn or not assembled to valve	Replace membrane or ensure that it is connected properly to the valve ( <b>PAGE 7, STEP 1</b> ).
Dried milk in pump	Follow cleaning instructions ( <b>PAGE 6</b> ). Pay extra attention to the valve and membrane.	

*Continued on next page...*

Problem	Possible Cause	Action
Little to no suction	Membrane not sealing	Make sure valve and membrane are clean, with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve <b>(PAGE 7)</b> .
	Flange insert not fully seated into flange	Properly install flange insert <b>(PAGE 10)</b> .
	Pump is not in desired single or double pumping mode	If pump is set up to be in double mode (both flange tubes are attached directly to tubing adapter) both flanges must be attached to the breast with a good seal. When single pumping, make sure the unused tubing is correctly plugged with the tube cap <b>(PAGE 8)</b> .
	Poor seal between breast and flange	Reposition the flange to get a better seal. Make sure flange seals directly onto breast with no clothing between breast and flange. <b>(PAGE 11)</b> .
	Tubing kinked	Avoid crushing or kinking the tubing. Reposition flanges if tubing kinks during pumping.
	Suction setting too low	Choose a higher suction level by pressing the suction increase button <b>(PAGE 12)</b> .
	Not using Evenflo brand parts	Use provided Evenflo flange kit.
	Milk entered motor unit through adapter port	When used properly, milk should not enter motor unit. Inspect all parts, particularly diaphragms, for damage prior to each use. Only use Evenflo flange kit.
High suction	Suction settings too high	Choose a lower suction level by pressing the suction decrease button <b>(PAGE 12)</b> .
	Not using Evenflo brand parts	Use provided Evenflo flange kit.
	Pumping in single mode	Suction levels in single and double mode are not the same. Choose a lower suction level when pumping in single mode.
	Damaged diaphragm	Inspect all parts before each use. Replace all damaged components.

Problem	Possible Cause	Action
Moisture or milk in tubing	Using non-Evenflo brand parts	Use provided Evenflo flange kit.
	Damaged diaphragm	Inspect all parts before each use. Replace all damaged components.
Discomfort	Damaged diaphragm	Inspect diaphragm before each use, replace damaged components.
	Suction settings too high	Select a lower suction level by pressing the suction decrease button <b>(PAGE 12)</b> .
	Using incorrect flange size	Using a flange that is too small or too large can cause discomfort. See Flange Sizing <b>(PAGE 11)</b> .
Expressing little to no milk when using the pump	Not stimulating milk release before expressing	Use low suction with high speed until milk begins to flow, then increase suction to highest comfortable setting.
	Tubing not properly connected	If pump is set up to be in double mode (both flange tubes are attached directly to tubing adapter) both flanges must be attached to the breast with a good seal. When single pumping, make sure the unused tubing is correctly plugged with the tube cap. <b>(PAGE 8)</b> .
	Missing parts	Ensure diaphragm, valve, and membrane are properly assembled.
	Pumping after breastfeeding	Wait 45-60 minutes after breastfeeding before pumping.
	Changing from a different pump brand	Try massage and hand expression to stimulate milk release until your breast becomes accustomed to the feel of a new brand.
	Incorrect pump settings selected	Adjust suction and speed to levels that are comfortable for you.

**For warranty information please contact us at USA/Canada: 1-855-EFI-BABY (334-2229), by mail at Evenflo Feeding Inc., Attn. Consumer Resources, 9277 Centre Pointe Drive, Suite 160, West Chester, OH 45069, or visit our website at [www.evenflofeeding.com/warranty](http://www.evenflofeeding.com/warranty).**

**Still having problems? Our Parent's Ally Consumer Care Team can help.**

*Remember, this is a personal care item for one user only. For hygienic reasons, **ONCE OPENED, THIS PRODUCT CANNOT BE RETURNED.***

*If you have any questions, comments, or concerns about our Evenflo breast pumps, please contact our Parent's Ally Consumer Care Team.*

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