
One-Way Nasal Aspirator

User's Guide IH198

Thank you for purchasing this Advanced Solutions™ product. We're proud to be your choice to help you keep your child healthy and safe. Please read these instructions completely before using this product.

1st To Use

1. Hold baby upright and grasp aspirator as shown (Figure A). Gently place aspirator tip in nostril, angled so that it does not point directly toward center of nostril.
2. Keeping tip in the nostril, pump with thumb on bottom of bulb (6+ times in rapid succession) to create suction. Air flows into tip and out vent (Figure C), while mucus collects just inside the tip.
3. When aspiration is complete, remove tip from nostril. Repeat in other nostril.
4. Clean aspirator after use.

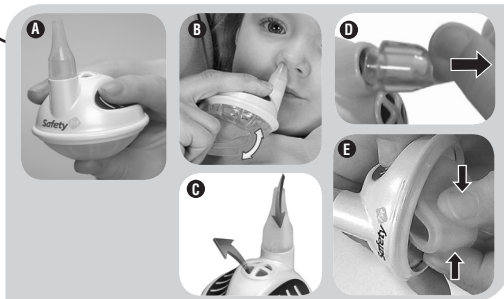CAUTION: NEVER push the aspirator too far into the nostril. Wash and dry thoroughly before first use and after each use. NEVER use in baby's mouth or ears. When storing, keep away from heat and bright light. This product is not a toy. **DO NOT** allow children to play with it.

1st To Clean

1. Separate bulb into three parts: top of bulb, bottom of bulb and silicone tip (Figures D and E). **DO NOT** attempt to remove valves.
2. Wash all parts in warm soapy water and rinse thoroughly. Run warm water through the valves in the same direction as air flow (arrows in Figure C). Avoid using abrasive pads or cleansers.
3. Sterilize silicone tip (optional): Microwave 90 seconds on high. Tip will be hot. Allow to cool completely before touching or using it. **DO NOT** put other aspirator parts in microwave.
4. Air dry, then be sure to install tip completely during reassembly.

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