Yes, you can.®

Walkers Marchettes Anadores

Safety You Can TrustSM

Walker Models

6291-A	6291-3F	6281-JR
6291-JR	6291-JR3F	6281-3F
6241-A	6291-5F	6291-HDA
6241-JR	6281-A	

Refer to page 2.

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6291-A	6291-3F	6281-JR
6291-JR	6291-JR3F	6281-3F
6241-A	6291-5F	6291-HDA
6241-JR	6281-A	

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6291-A	6291-3F	6281-JR
6291-JR	6291-JR3F	6281-3F
6241-A	6291-5F	6291-HDA
6241-JR	6281-A	

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Invacare Walkers have a Lifetime Limited Warranty (see pages 10 and 11).

Invacare Walkers provide assistance for an individual weighing up to:

300 lbs. (136 kg.)- Models 6291-A, 6291-JR, 6291-I, 6291-3F, 6291-JR3F, 6291-5F, 6281-A, 6281-JR, 6281-3F, 6241-A, and 6241-JR

500 lbs. (226 kg.)- Models 6291-HDA

DUAL-RELEASE WALKERS

* Easy to open and close from a seated or standing position.

Model No.	Width		Depth		Height Adjustment	Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6291-A	17"	21-1/2"	17-1/2"	4"	30-3/8" - 37-3/8"	5'3" - 6'4"
6291-JR	17"	20-1/2"	16-1/2"	4"	25-3/8" - 32-3/8"	4'4" - 5'7"
6241-A	17"	19-1/2"	17"	4"	30-3/8" - 37-3/8"	5'3" - 6'4"
6241-JR	17"	18-1/2"	16"	4"	25-3/8" - 32-3/8"	4'4" - 5'7"

DUAL-RELEASE WALKERS WITH FIXED WHEELS AND GLIDE TIPS

* 3-inch or 5-inch fixed wheels roll freely over smooth surfaces.

Model No.	Width		Depth		Height Adjustment	Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6291-3F	17"	21-1/4"	17-1/2"	4-1/2"	31" - 37"	5'3" - 6'4"
6291-JR3F	17"	20-1/4"	16-1/2"	4-1/2"	26" - 32"	4'4" - 5'7"
6291-5F*	17"	21-1/2"	17-1/2"	4-1/2"	33" - 39"	5'3" - 6'4"

* with 13-inch leg extensions.

SINGLE-RELEASE WALKERS

* Ergonomic release button allows walker to be folded with one hand.

Model No.	Width		Depth		Height Adjustment	Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6281-A	17"	21-1/2"	17-1/2"	4-1/2"	30-3/8" - 37-3/8"	5'3" - 6'4"
6281-JR	17"	20-1/2"	16-1/2"	4-1/2"	25-3/8" - 32-3/8"	4'4" - 5'7"

SINGLE-RELEASE WALKER WITH FIXED WHEELS AND GLIDE TIPS

* 3-inch fixed wheels roll freely over smooth surfaces.

Model No.	Width		Depth		Height Adjustment	Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6281-3F	17"	21-1/4"	17-1/2"	4"	31" - 37"	5'3" - 6'4"

HEAVY DUTY ADULT DUAL RELEASE WALKER

* Wider walker provides ambulatory assistance for an individual weighing up to 500 lbs.

Model No.	Width		Depth		Height Adjustment	Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6291-HDA	20"	24"	17-1/2"	4"	30-3/8" - 37-3/8"	5'3" - 6'4"

Specifications are subject to change without notification.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the proper use of Invacare Walkers, these instructions MUST be followed:

GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

STABILITY WARNINGS

If walker is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips do not twist on side frame otherwise damage or injury may occur.

Check to make sure rubber tips and/or plastic glide tips are not ripped, worn or missing. Replace **BEFORE** using walker.

All leg extensions, must be adjusted so the walker is **LEVEL**. When using wheeled attachments, extensions **MUST** be adjusted so side frames are level.

DO NOT use rubber tips on rear leg extensions when using front wheel or brake options.

Rear extension legs with rubber tips or plastic glide tips **MUST** be in contact with floor surface at all times.

Glide tips are used on rear leg extensions **ONLY** to allow walker to roll easily over indoor carpeted areas without the need to lift the walker off the ground.

Glide tips on rear leg extensions should only be used with front wheel attachments.

Glide tips are not recommended for use on rough surfaces (concrete, gravel, etc.).

Glide tips are not recommended for use on all four (4) leg extensions at once.

DO NOT use glide tips and rubber tips at the same time.

Glide tips do not grip floor the way rubber tips grip flooring, therefore, caution should be observed when in use.

When using glide tips on walker, short steps should be taken to assure weight is distributed evenly and directly over Walker legs. These precautions will prevent uncontrollable movement of walker.

Always use caution when using walker on wet or icy surfaces.

STABILITY WARNINGS (CONTINUED)

Invacare Walkers can provide ambulatory assistance for an individual weighing up to 300 lbs. (136 kg.) for Single-Release (6281 Models), Paddle (6291 Models) and Blue-Release™ (6241 Models) Walkers, and 500 lbs. (226 kg.) for the Heavy Duty Walker (6291-HDA Model).

Invacare Walkers are designed to provide support, increased stability and assistance to an individual while walking; it is not intended to support the full weight of the user.

FOLDING/ASSEMBLY WARNINGS

After unfolding or assembling walker, ensure walker is securely locked in OPEN position and level to the ground BEFORE using. **DO NOT** hang anything on the front of the Blue-Release Walker. This may depress the release mechanisms and prevent walker from locking into place when fully opened.

DO NOT hang anything on either the left or right side frames (area that includes dual-release paddles) or the left and right slide tubes (area that includes palm release button). This will cause side frames or slide tubes to bend downward and prevent walker from locking into place when fully opened. This may also cause walker to tip, resulting in injury or damage.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

Wheeled Accessories, if used, are **ONLY** to be used on the front walker legs with the exception of Model Nos. 6264 - 10-inch Rear Wheel Brakes and 6265 - 13-inch Rear Wheel Brakes.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

Always test to see that the walker and attachments are properly and securely locked in place **BEFORE** using.

OPENING WALKERS (FIGURE 1)

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this manual.

1. To OPEN the walker, pivot the left and right side frames outward.
2. Perform one (1) of the following:

SINGLE-RELEASE WALKER (6281) - Grip the side frames and swing them fully outward while listening for an audible "click" which indicates that the walker is fully opened and ready for test. Refer to DETAIL "A" in FIGURE 1.


NOTE: The palm release button on the pivot slide tube will pop up when the click is heard.

BLUE-RELEASE WALKER (6241) - Swing the side frames outward until an audible "click" is heard. Refer to DETAIL "B" in FIGURE 1.

PADDLE WALKER (6291) - Swing the side frames outward until the dual-release paddles activate (**engage upward**). Refer to DETAIL "C" in FIGURE 1.


3. Before using, test walker by grasping the side frames and, with some force, attempt to fold them inward. See DETAIL "C" in FIGURE 1. The walker will remain locked in the OPEN position if it is engaged properly. Test Walker side frames before using. See DETAIL "D" in FIGURE 1.

DETAIL "A"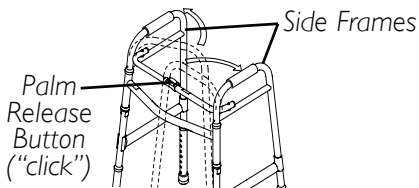


**SINGLE-RELEASE
WALKER (6281)**

DETAIL "B"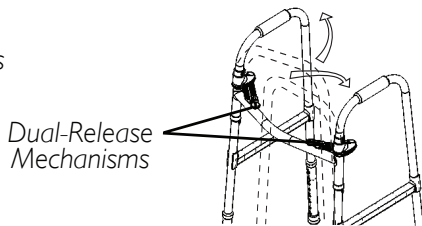


**BLUE-RELEASE
WALKER (6241)**

DETAIL "C"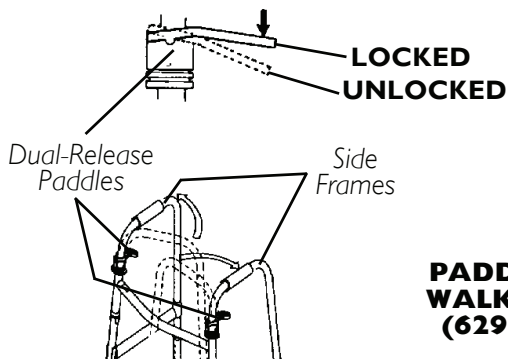

DETAIL "D"

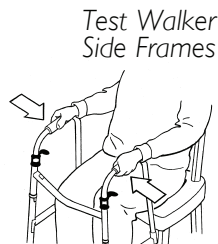PADDLE WALKER (6291)

FIGURE 1 - OPENING WALKERS

FOLDING WALKERS (FIGURE 2)

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this instruction sheet.

I. Perform one (1) of the following to fold the walker:

SINGLE-RELEASE WALKER (6281) -

- A. Press **DOWN** on the palm release button located in the center of the pivot slide tube. Refer to DETAIL "A" in FIGURE 2.
- B. Pivot the right and left side frames inward until they meet the front crossbrace.


BLUE-RELEASE WALKER (6241)

- A. Depress either release mechanism and pivot the corresponding side frame inward toward the front crossbrace. Refer to DETAIL "B" in FIGURE 2.
- B. Repeat the same procedure for the remaining side frame.


PADDLE WALKER (6291) -

- A. Depress either release paddle and pivot the corresponding side frame inward toward the front crossbrace. Refer to DETAIL "C" in FIGURE 2.
- B. Repeat the same procedure for the remaining side frame.


DETAIL "A"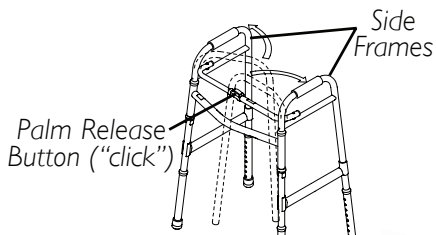

DETAIL "B"

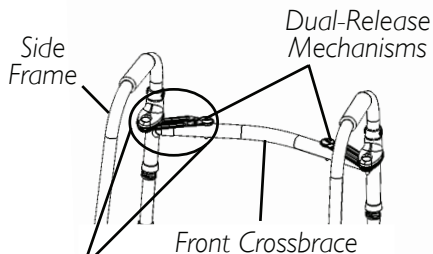SINGLE-RELEASE WALKER (6281)

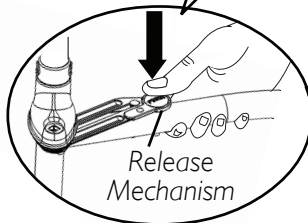BLUE-RELEASE WALKER (6241)

DETAIL "C"

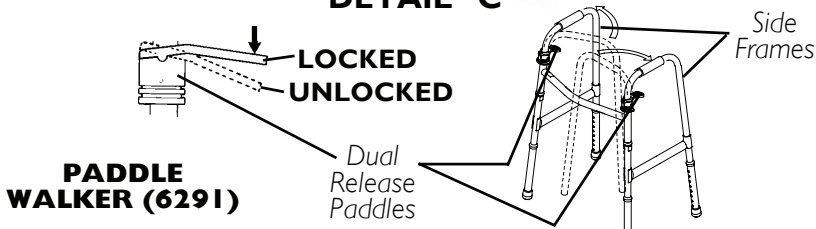
FIGURE 2 - FOLDING WALKERS

INSTALLING/ADJUSTING LEG EXTENSIONS (FIGURE 3)


NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove leg extensions from carton and install the leg extensions by depressing the snap buttons on the lower legs of the walker and inserting the leg extension. Refer to FIGURE 3.
2. Depress the snap button and slide the leg up or down to the proper height. Ensure that the snap button protrudes through the appropriate adjustment hole on the leg extension.

NOTE: The leg extension adjustment holes are numbered for convenience.

3. Repeat this procedure for each leg extension.
4. Adjust leg extensions to ensure that the walker is LEVEL.
5. Adjust the walker so that when the user's arm is extended downward the handle is at wrist height.

NOTE: MODEL NO. 6291-3F, 6291-5F, 6291-JR3F, AND 6281-3F: Ensure that fixed wheel attachments are positioned to the outside of the walker to help maintain stability.

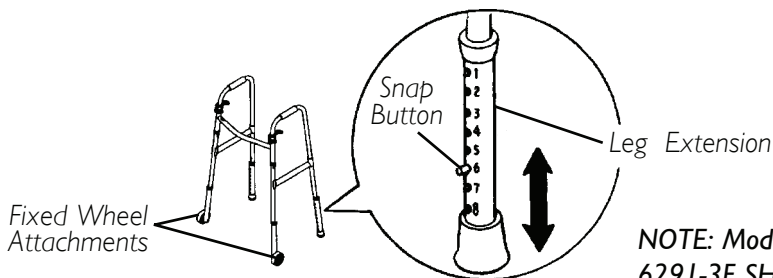*NOTE: Model No.
6291-3F SHOWN*

FIGURE 3 - INSTALLING/ADJUSTING LEG EXTENSIONS

CARE AND MAINTENANCE

1. Ensure that **ALL** attaching hardware is secure at **ALL** times.
2. Make sure Snap Buttons release and re-engage properly.
3. Replace any broken, damaged or worn items immediately.

CARE OF WHEELED ACCESSORIES

1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
2. Lubricate with 3-in-1[®] oil periodically or if wheels begin to squeak.

LIFETIME LIMITED WARRANTY

Invacare warrants this product to be free from defects in materials and workmanship.

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in material and workmanship for the lifetime of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

LIFETIME LIMITED WARRANTY CONTINUED

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/ PROVINCIAL LAWS AND REQUIREMENTS.

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Yes, you can.[®]

Find out about other products that make your life easier at:
www.invacare.com

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É.-U. : 1-800-333-6900

Canada: 1-800-668-5324

"Invacare says 'Yes, you can,' and I couldn't have said it better myself!"

— Arnold Palmer

USA

One Invacare Way

Elyria, Ohio 44036-4028 USA

Canada

570 Matheson Blvd. E., Unit 8

Mississauga, Ontario L4Z 4G4 Canada

©2005 Invacare Corporation. Invacare, the Blue Medallion Design and Yes, you can. are registered trademarks and IVC and I-Fit are trademarks of Invacare Corporation. Safety You Can Trust is a service mark of Invacare Corporation. 3-in-1 is a registered trademark of American Home Products Corporation.