
Handsfree LED Magnifier

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Lupa manos libres con LED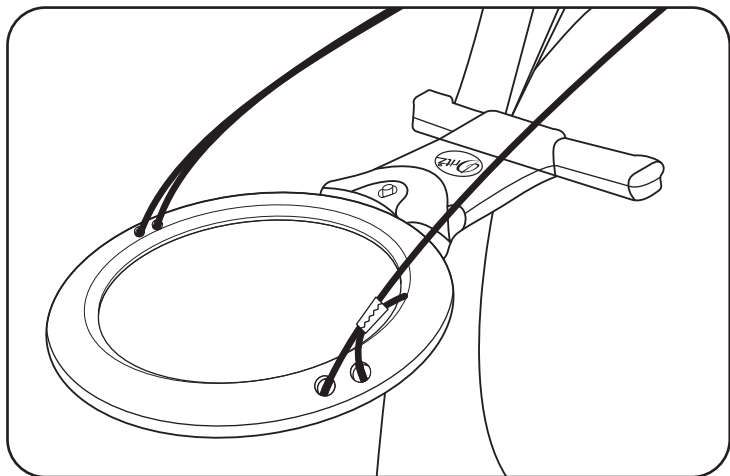

To install batteries:

1. Release stand on back side of handle by placing fingertips on small pads at top sides of handle (A) and from the back use fingertips to pull stand down and away from magnifier.
2. Grasp battery cover at top side indentations. Pull cover down (B).
3. Remove old batteries. Install new batteries as indicated by markings in battery compartment.

To lower the stand for table top use:

1. Stand is at back of handle. Place fingertips on small pads at top sides of handle (A).
2. Working from the back, use fingertips to pull stand down and away from magnifier.
3. Hold onto handle while folding down magnifier to desired position. Stand has two angled positions and magnifier has four stops for multiple angles.

Care instructions:

- Do not view sun with lens.
- Wipe gently with soft cloth to clean.
- Keep away from direct sunlight.
- Only use the specified battery type and size.
- Observe proper +/- polarities, properly aligning the anodes and cathodes.
- Do not mix old and new batteries. Do not mix different types of batteries such as alkaline, standard (carbon-zinc), or rechargeable (Ni-Cad, NiMH, etc.) batteries.
- Properly dispose of used batteries.

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